BackBoardStraps

System **RoMedic**

Instructions for use – English

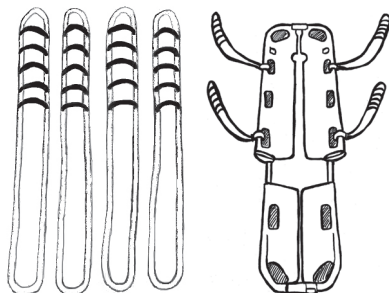
Bruksanvisning – Svenska

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REF


49482000

SWL: 160 kg/ 350 lbs

Table of contents

English	3
Svenska	5
Norsk	7
Danske	9
Deutsch	11
Français	13
Symbols	15

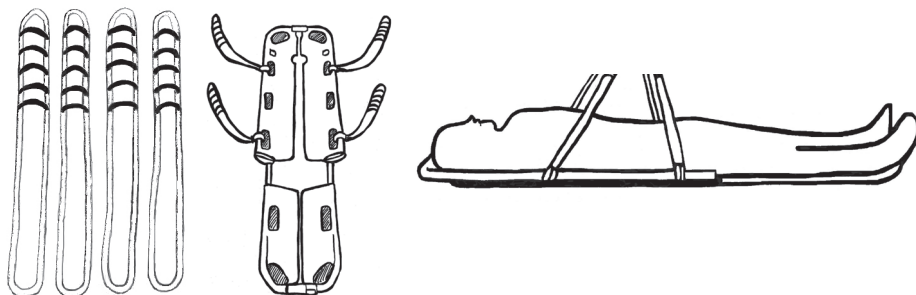BackBoardStraps have been designed for use in combination with the Ferno Scoop Stretcher 065 EXL to enable mechanical lifting with a manual stretcher in emergency care. The straps are mounted on the Ferno Scoop Stretcher according to these instructions. BackBoardStraps consist of four independent lift straps. There are 6 alternative lift loops on one side of each strap to facilitate balancing of the stretcher. The BackBoardStraps are looped securely through the handle holes (two on each side), and then connected to the slingbar hooks.

BackBoardStraps is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.

Direct Healthcare Group's products together form the SystemRoMedic concept, which takes a holistic approach to patient transfers. SystemRoMedic has four categories of transfers: transfer, positioning, support and lifting.

Functional inspection

Visual inspection

Check the condition and function of the product regularly, and always after laundering. Check to ensure that the fabric is not worn, damaged or bleached. If there are signs of wear, the product must be discarded.

Always read the instructions

Read instructions for all assistive devices used in connection with transfers.

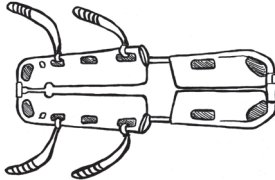
Read and follow the instructions for the Ferno Scoop Stretcher.

To prevent discomfort, exercise care and caution when using the product.

Application and use

The straps are secured through the holes on the sides of the stretcher at the position of the patient's hips and shoulders. Simply pull a strap through each hole, and then loop it through itself.

Two straps are mounted on each side of the stretcher, and then hooked to the slingbar. Adjust the slingbar to the appropriate width to prevent discomfort to the patient.

Before lifting, balance the stretcher using the six alternative lift loops.

Start lifting carefully, and then stop to ensure that the stretcher is properly balanced in a horizontal position. If it is balanced, continue lifting; if not, lower the lift and adjust the balance using the lift loops.

In each lifting situation, always perform an individual test to ensure that the lift is properly balanced.

Always make certain that the stretcher is horizontal!

Always use the safety belts according to the instruction guide for the Ferno Scoop Stretcher!

Before lifting:

Read and follow the instructions for the Ferno Scoop Stretcher.

Plan the lift and decide how many caregivers will be needed to assist to ensure safety.

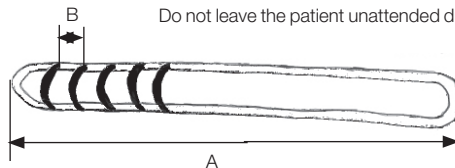
Never lift higher than necessary.

Do not leave the patient unattended during a lifting situation.

Dimensions

A: 110 cm

B: 8 cm

Care of the product:

Read the product label.

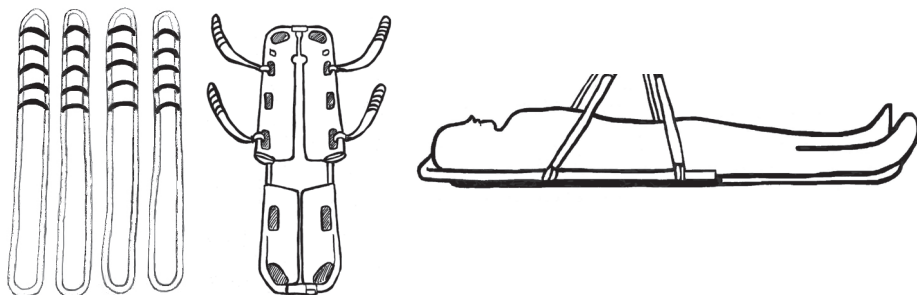
Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

When in doubt, contact your Direct Healthcare Group representative for product trials, advice or information.

If you require additional information, contact your transfer specialist or our representative,

or visit www.directhealthcaregroup.com. Make sure you always have the most recent version of the manual.

Manuals are available for downloading from our website www.directhealthcaregroup.com

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.