

# Infrared remote 30-67847-9

## UserManual




**Keep this folder with product at all times!**

# 30-67847-9 Infrared remote, one channels (IRR-DSK-SET-LIGHT)

## Preparation

Place the two batteries (R03 AAA) in the handset.

Connect the receiver to the motor control (HS – connector). The bottom side of the receiver is coated with an adhesive, for easy installation.

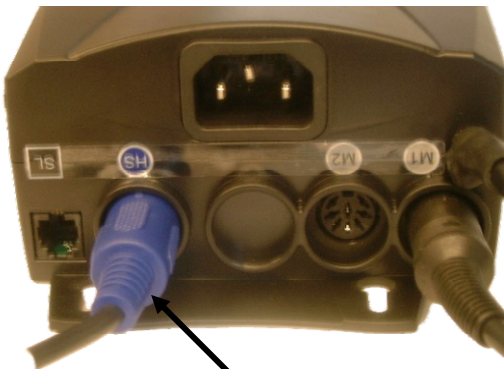

Connect receiver to HS socket



Left position CH1


## Channel selection

The handset allows operation on only one channel. Channel 1

### ***Channel Selection (Receiver):***

The switch on the receiver must be in its left position, channel 1 is selected.

## Functionality:

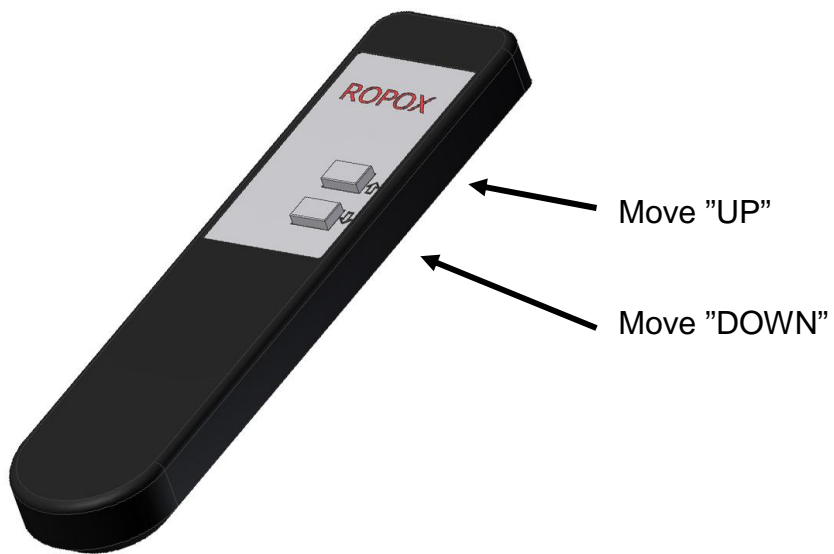

### *Driving Up/Down*

With the buttons up ( "↑ " ) and down ( "↓ " ), the motor control box starts the drives in the respective direction.

Availability and specifications are subject to change without prior notice.  
**LOGICDATA** assumes no liability for maloperation or improper use of products.  
**LOGICDATA** will at point of delivery replace/repair defective products covered by the warranty.  
No liability is assumed beyond such replacement/repair.  
Please contact **LOGICDATA** directly on inquiries or custom requirements.

Manufacturer information:

LOGICDATA Electronic & Software Entwicklungs GmbH Wirtschaftspark 18 A-8530 Deutschlandsberg	Tel.: +43 (0)3462 5198 0 Fax: +43 (0)3462 5198 530 Email: <a href="mailto:office@logicdata.at">office@logicdata.at</a> <a href="http://www.logicdata.at">www.logicdata.at</a>
---	--



**ROPOX A/S**

Ringstedgade 221  
DK – 4700 Naestved  
Tel.: +45 55 75 05 00 Fax.: +45 55 75 05 50  
E-mail: [info@ropox.dk](mailto:info@ropox.dk)  
[www.ropox.com](http://www.ropox.com)