# Loire wave 190, 40-40145-9

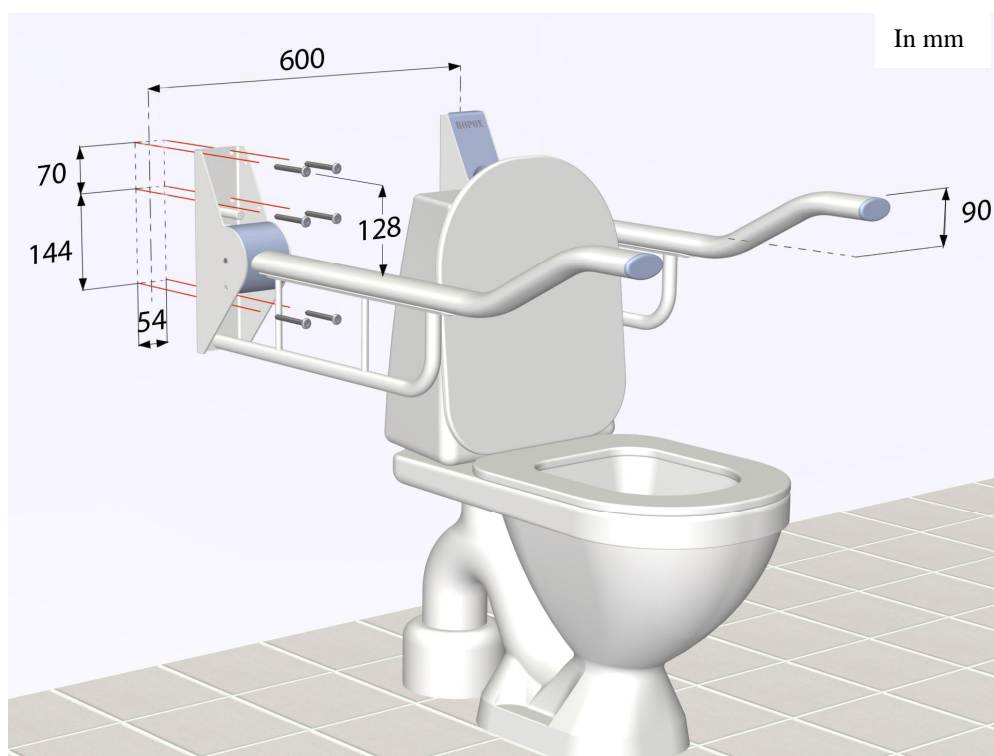
Montageanvisning

Mounting instruction

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## I henhold til / according to DS 3028 (Danish Standard)

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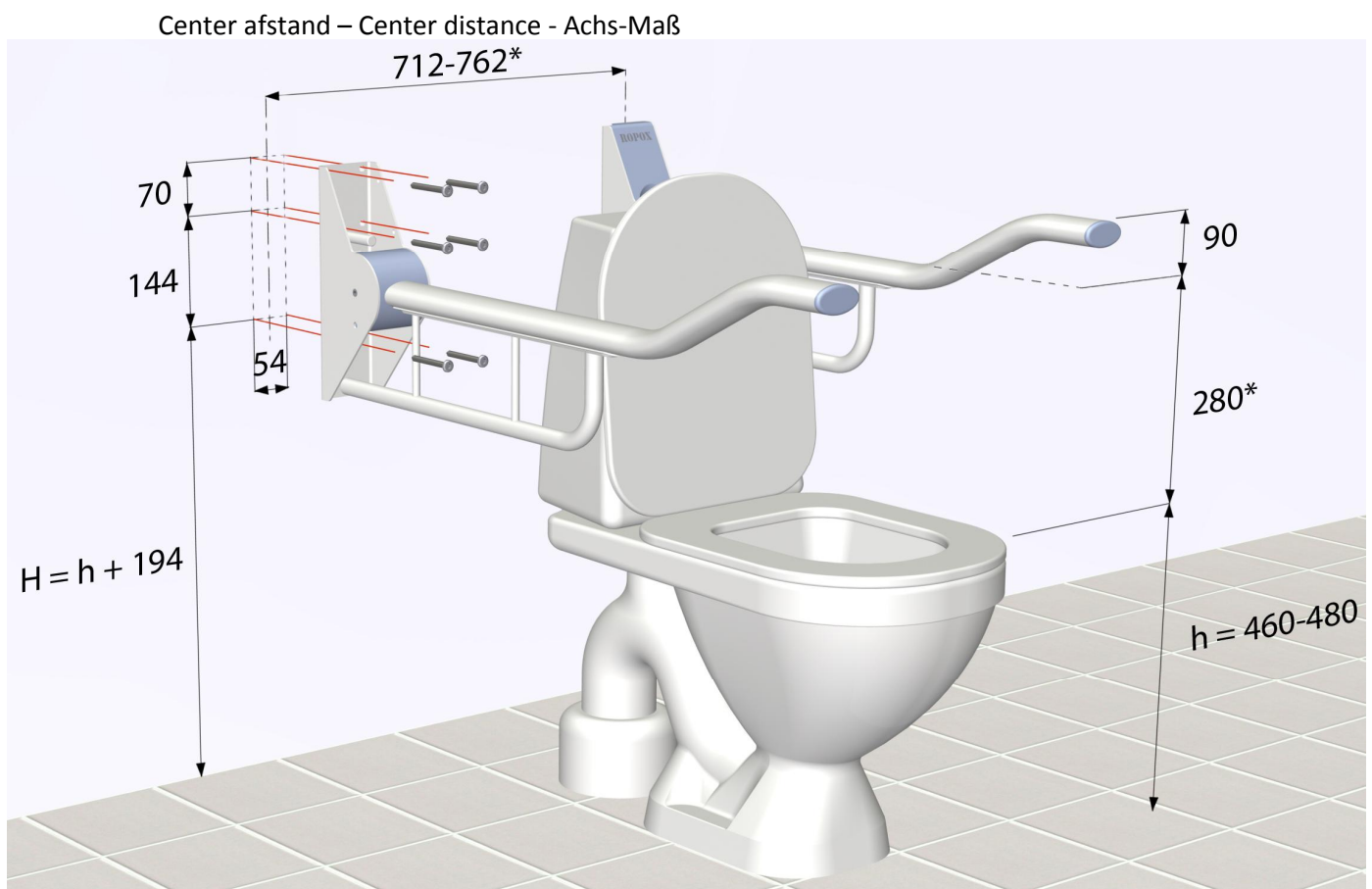
Please find attached mounting template showing position of the 6 holes for easy installation

### Recommended user heights and installation heights (upper holes in mounting template/bracket)

User height lowest part of wave (recommendation)	Installation height (upper holes)	Situation
710 mm	838 mm	Loire wave (front height + 90 mm) = 800 mm
<p>If any other specific user height is required, simply position the 2 upper holes of the bracket (or the mounting template) 128 mm above the required height of the lowest part of the support arm.</p> <p><u>Example:</u> Required user height (lowest part of wave) is 690 mm                      Installation height = 690 + 128 mm = 818 mm                      Front end height = 690 + 90 mm = 780 mm</p>		

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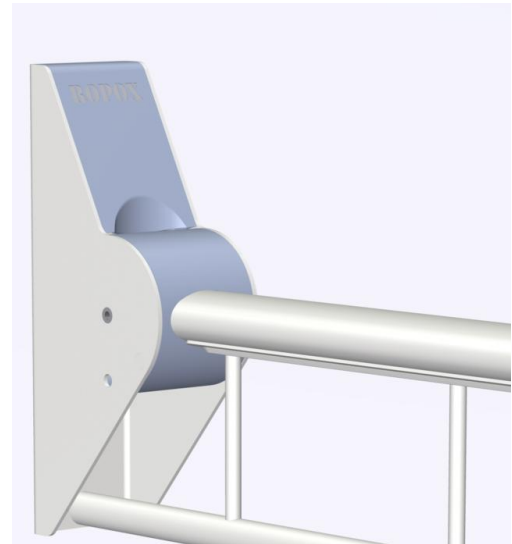
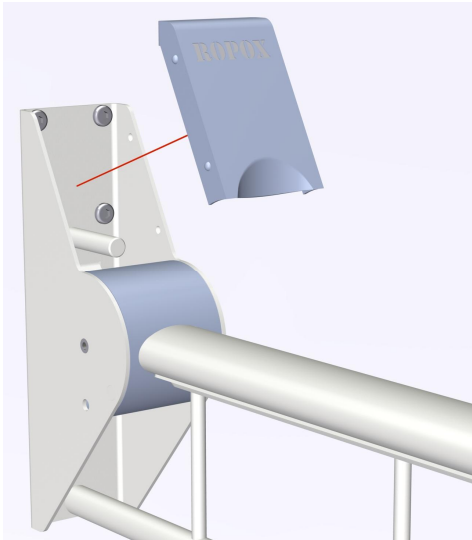
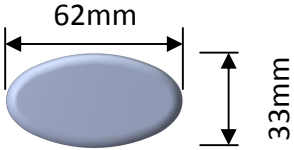
## Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

## Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable.

Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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## **Complaints**

See General Terms of Sale and Delivery on [www.ropox.com](http://www.ropox.com)

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## **ROPOX A/S**