

**Direct Healthcare  
Group**

Advancing Movement & Health®

# Bure Ortho Bure Space

User Manual





# Bure Ortho

## Bure Space

Ortho is especially designed for orthopaedics departments and situations that demand extra legroom and room for manoeuvre for the user. Ortho has a powerful electric motor for powered height adjustment. Walker heights are thus adjustable with the patient in situ and the walker can be used as a stand-up aid. Ortho is also available in a small version – Ortho S (only available in the gas version) – with a max frame width of only 65 cm. In addition to Ortho's advantages, Bure Space also has extra room to get up close to wheelchairs or similar. The base frame has been made wider and the internal width extended to 67 cm. Bure Space is available with electrically operated or hydraulic height adjustment.

### Contents

1. Raising the frame	04
2. Lowering the frame	04
3. Adjusting the handles	04
4. Left and right adjustment of armrests	04
5. Locking/unlocking castors	04
6. Maintenance	06
7. General maintenance advice	07
8. Technical specifications	07

## 1. Raising the frame

Release the gas strut by moving the handle upward (the gas strut will raise the walker). Let go of the handle at the desired height (the gas strut will lock).



## 2. Lowering the frame

Release the gas strut by moving the handle upward. Use your body weight to press the frame down. Let go of the handle at the desired height (the gas strut will lock).



NOTE: The handle may not be pushed down. There is a risk of breaking it.

## 3. Adjusting the handles

To adjust handles to the desired position; loosen screw, adjust handle, tighten screw.



## 4. Left and right adjustment of armrests

The armrests can be adjusted sideways for optimal support; loosen screw, adjust armrest, tighten screw.

## 5. Locking/unlocking castors

Depress the foot brake to set it and lock the castor.

Press down on the protruding part of the brake to release it.



## 6. Maintenance

Recommended at least once per year.

Check that	Action
<p><b>General</b></p> <p>The walker feels firm/free from play.                      The walker does not rattle during manoeuvres.                      The walker is level and all of its castors are in contact with the floor.                      The walker is not dirty</p>	<p>Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).</p>
<p><b>Armrests</b></p> <p>Armrests are intact and clean.                      Armrest widening is functional</p>	<p>Clean; replace if damaged.                      Fit lock screws/plates as required.</p>
<p><b>Handle</b></p> <p>Grips are not damaged/dirty.                      Handle adjustment is functional</p>	<p>Clean with lukewarm soap solution (no petroleum products).                      Replace handle grips.                      Fit lock screws/plates as required.</p>
<p><b>Height adjustment</b></p> <p>Raising and lowering are functional.                      The walker feels stable at maximum settings.                      There is no play between the vertical frame and bottom frame.                      Height adjustment clamps lock.                      The gas strut fitting must be free from play.                      Height adjustment by means of the handle is working. The gas strut must slide freely without squeaking.</p>	<p>Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required.                      Tighten all lock bolts.                      Replace clamps.                      Tighten the attachment fitting concerned; replace bolts/lock nuts as necessary.                      Replace gas strut, handle</p>
<p><b>Frame parts</b></p> <p>There is no mechanical damage.                      There are no scratches.                      End plugs/lower frame fitted</p>	<p>If damage is present, contact GATE's customer services.                      Touch up as necessary.                      Fit new end plugs.</p>
<p><b>Hjul/broms</b></p> <p>Castors roll easily/tread not damaged.                      The castors are firmly fastened to the lower frame.                      Castor brakes functional on all castors.                      Handbrake functional.</p>	<p>Clean or replace castors.                      NOTE: Castors are always fixed to the lower frame with thread locker or lock nuts. Do not dismantle the castors; replace the whole castor assembly instead.                      Tighten the castor bolt and fit locking nut or use thread locker (depends on model).                      Replace castor assembly.                      Adjust the brake or fit new castors.</p>

## 7. General maintenance advice

- Clean the walker and its armrests clean as necessary. Wipe using a soap solution/surface disinfectant.
- The product can be washed in a pressure washer/machine at 60°C for ten minutes. To increase lifetime, avoid aiming high pressure water jets at bearings.
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

## 8. Technical Specification

Dimensions are for walkers with 125mm wheels.

Part No.	Product	Product width	Max user weight	Length	Height	Armrest/ Internal dimensions
56-318	Ortho Gas	78cm	150kg	80cm	90-130cm	25-50cm
56-307	Ortho Space Gas	78cm	150kg	79cm	90-130cm	25-50cm
56-318-S	Ortho S Gas	65cm	150kg	79cm	90-130cm	30-45cm

# USER MANUAL

# Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group  
Withey Court, Western Industrial Estate  
Caerphilly  
CF83 1BF

T: 0800 043 0881  
F: 0845 459 9832  
E: [info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)

LIT-00139 Issue 1  
Date: January 2020

GATERD.SE