



# Bure Rise & Go

Bure Rise & Go is an all-round solution that saves space, time and money. Instead of purchasing both hoists and walkers it is probably often sufficient to acquire the flexible, cost effective Bure Rise & Go.

**Stable**

The walker has anatomical cushions that provide relief for shoulders, arms and the neck and shift the focus to larger muscle groups such as leg and chest muscles. This improves stability and makes the walker easier to steer.

**Easily reachable hand controller**

The hand controller is located such that both of the carer's or user's hands can easily reach them.

**Ergonomic handles**

Ergonomically designed handles are steplessly adjustable in every direction for optimum individual comfort. This is extremely important for stroke patients who can thus reach their "usual" hand position.

**Folding armrests**

Bure Rise & Go's anatomically designed armrests can be folded out to provide the user with an easy means of reaching the handles during the standing manoeuvre.

**Flexible stepless adjustment**

Electrically driven height adjustment makes it easy to pre-set the walker to the desired height. Once the patient is standing in position, the height can be steplessly adjusted for comfort and the proper support for an optimal walking configuration.

**Sustainable**

The walker is designed to last. For example it has extra reinforcement bands around the arm rest cushions to prevent them from splitting from impacts to the side.

**Power Rise**

Bure Rise & Go is a unique aid for assisting patients or users during the standing manoeuvre. Personnel burdens, and thus repetitive strain injuries, are minimised.

**Waterproofing**

All Bure Rise & Go electrical components Bure Rise & Go have IP 65 international protection ratings and can therefore be used in wet spaces.

**Stirrups for stability**

To provide extra safety, Bure Rise & Go is fitted with a stirrup to ensure the walker remains stationary during the standing manoeuvre.

**Open, reinforced design**

The open design provides the same stability but with greater room for manoeuvre. The patient does not feel shut in and it makes assistance with standing up and dressing easier. Bure Rise & Go is also reinforced and extended for extra stability during the standing manoeuvre.

**Leg support**

Bure Rise & Go is fitted with shin supports that make for a better standing manoeuvre; they are height adjustable and can be moved aside simply to create greater space.

**Unique power rise harness**

The power rise harness is designed to reach as far down across the bottom as possible. This provides optimum support, which results in a natural standing action.

**Lockable castors**

Bure Rise & Go has four individually lockable castors as standard.

**Handbrakes and directionally lockable castors**

Single or double hand brakes are available as accessories located so that the user can reach them from the walking position. The handbrake acts on the rear castors and the brake assembly also includes directional locks that make it easier for the user to stay on course when walking.



**Technical Specification**

Art. Nr.	Produkt	Width, external	Max user weight	Length	Height	Armrest/internal dimension
56-316	Bure Rise & Go Stand Tall Walker	77cm	150kg	92cm	97-137cm	34-67cm
56-316/75	Bure Rise & Go, 75mm	77cm	150kg	92cm	92-132cm	34-67cm
56-333-STD	Platform short		150kg			
56-383-ST	Platform long		150kg			
56-384-MULTI-S/M/L/XL	Stand-up harness, multi	-	150kg	-	-	-
56-388-KIT	Gait training kit for multi stand-up harness	-	150kg	-	-	-

\* S=95-105cm, M=105-120cm, L=110-140cm, XL=130-160cm

All walkers may be ordered with 75, 100 or 125mm castors – specify when ordering. (Unless otherwise specified, the heights shown below refer to products with 125 mm castors). Examples of possible parts combinations are shown below.