

**Direct Healthcare
Group**

Advancing Movement & Health®

Bure Standard Bure S Bure Low Bure Low S

User Manual



BURE STANDARD/S/LOW/LOW S

Bure Standard

Bure S

Bure Low

Bure Low S

Contents

1. Raising and lowering the frame	04
2. Adjusting the handles	04
3. Left and right adjustment of armrests	04
4. Locking/Unlocking castors	04
5. Control box wiring	04
6. Battery charging	05
7. LEDs on control box and battery	05
8. Maintenance	06
9. General maintenance advice	07
10. Technical specifications	07

1. Raising and lowering the frame

Adjust the height with the aid of the remote control UP and DOWN arrows



2. Adjusting the handles

To adjust handles to the desired position; loosen screw, adjust handle, tighten screw.

3. Left and right adjustment of armrests

För optimalt stöd kan armstödsplattorna justeras isidled: Ratt lossas, armstödsplattans läge justeras, ratt låses.

4. Locking/unlocking castors

Depress the foot brake to set it and lock the castor.

Press down on the protruding part of the brake to release it.



5. Control box wiring

Remove the control box cover to make the connections as described below. Use a screwdriver or similar; refer to the special labels on the control box.

1. Socket for the raise/lower actuator. NOTE: Make sure that plugs are connected to the correct sockets otherwise the actuators can be damaged.
2. Remote control socket.
3. Battery terminal socket.
4. Plug socket. NOTE: The plug must ALWAYS be connected to the control box for IP protection to be valid.



6. Charging the battery

- Connect plug to 230V wall socket. Charging begins automatically and the battery LED flashes green.
- Make sure the battery is fully charged before using for the first time. It takes around 25 h to fully charge the battery.
- We recommend that the battery plug be removed from the control box if the product will be out of use for an extended period (more than three weeks). This eliminates the risk of ruining the battery as the control box always uses a small quantity of electricity even when the walker is not in use.

7. LEDs on control box and battery

Control box – green light when power is available from battery/mains socket.

Battery:

- The LED flashes green during charging.
- The LED shows a constant green light when the battery is fully charged.
- When the battery is in normal state/in operation, the LED flashes green at REGULAR intervals of 0.5 s ON, 4 s OFF.
- When the battery requires charging (remaining charge <25%) the LED flashes orange/red at REGULAR intervals of – 0.5 s ON, 4 s OFF.



NOTE: The battery has an in-built function that automatically switches it off when remaining charge is <20%. The LED is then extinguished. The battery must now be fully charged before the electronics can be used again.

8. Maintenance

Recommended at least once per year

Check that	Action
<p>General The walker feels firm/free from play. The walker does not rattle during manoeuvres. The walker is level and all of its castors are in contact with the floor. The walker is not dirty</p>	<p>Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).</p>
<p>Armrest Armrests are intact and clean. Armrest widening is functional.</p>	<p>Clean; replace if damaged. Fit lock screws/plates as required</p>
<p>Handle Grips are not damaged/dirty. Handle adjustment is functional.</p>	<p>Clean with lukewarm soap solution (no petroleum products). Replace handle grips. Fit lock screws/plates as required</p>
<p>Battery Check that the charging cable is connected to the control box. Check that the remote control is connected to the control box. Check that connections are made according to the wiring diagram. Check that the battery, actuators and control box are not loose.</p>	<p>Fit new charging cable; this must always be connected to the control box (IP65 protection). Connect or fit new remote control as required. Refer to the wiring diagram in the Technical manual. Tighten; replace with new fasteners as required.</p>
<p>Height adjustment Raising and lowering are functional. The walker feels stable at maximum settings. There is no play between the vertical frame and bottom frame. Height adjustment clamps lock. The electric actuator fitting must be free from play. Height adjustment via remote control is functional. The electric actuator must run smoothly at a constant speed.</p>	<p>Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required. Tighten all lock bolts. Replace clamps. Tighten the attachment fitting concerned; replace bolts/lock nuts as necessary. Check that the battery/remote control/ actuator are connected in accordance with the wiring diagram. Charge the battery. Contact customer services.</p>
<p>Frame parts There is no mechanical damage. There are no scratches. End plugs/lower frame fitted</p>	<p>If damage is present, contact GATE's customer services. Touch up as necessary. Fit new end plugs</p>
<p>Castors/brakes Castors roll easily/tread not damaged. The castors are firmly fastened to the lower frame. Castor brakes functional on all castors. Handbrake functional.</p>	<p>Clean or replace castors. NOTE: Castors are always fixed to the lower frame with thread locker or lock nuts. Do not dismantle the castors; replace the whole castor assembly instead. Tighten the castor bolt and fit locking nut or use thread locker (depends on model). Replace castor assembly. Adjust the brake or fit new castors.</p>

9. General maintenance advice

- Clean the walker and its armrests clean as necessary; wipe using a soap solution/surface disinfectant.
- The product may be cleaned in a tunnel washer or similar as electrical systems have IP65 protection. However, the product is not corrosion proof which means washing by machine may reduce its lifetime.
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

10. Technical specifications

Dimensions apply to walkers with 125 mm castors

Part No.	Product	Product width	Max user weight	Length	Height	Armrest/ internal dimension
56-312T	Standard EL	78cm	150kg	80cm	90-130cm	25-50cm
56-302T	S EL	66cm	150kg	79cm	90-130cm	30-45cm
56-321	Low EL	78cm	150kg	80cm	75-100cm	25-50cm
56-321-S	Low S EL	66cm	150kg	79cm	75-100cm	30-45cm

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group
Withey Court, Western Industrial Estate
Caerphilly
CF83 1BF

T: 0800 043 0881
F: 0845 459 9832
E: info@directhealthcaregroup.com

LIT-00145 Issue 1
Date: January 2020

DIRECTHEALTHCAREGROUP.COM