# CalfStrap

System**RoMedic**

---

Instructions for use – English

Bruksanvisning – Svenska

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**REF**

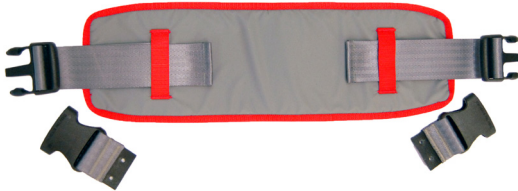
70200033

---

**SWL: 200 kg/ 440 lbs**

## Table of contents

English .....	3
Svenska .....	5
Norsk .....	7
Dansk .....	9
Suomi .....	11
Deutsch .....	13
Nederlands .....	15
Français .....	17
Polski .....	19
Symbols .....	21

CalfStrap is an accessory for MiniLift125/MiniLift200. It is a padded strap that gives the patient extra support and a sense of security when used in combination with MiniLift125/MiniLift200. CalfStrap is fastened via clasps at the side of the leg support, and the length is easily adjusted to suit the patient. On MiniLift125/MiniLift200 models, a hole at the back of the knee support is used to mount the fastener for the CalfStrap.

## Functional inspection

### Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

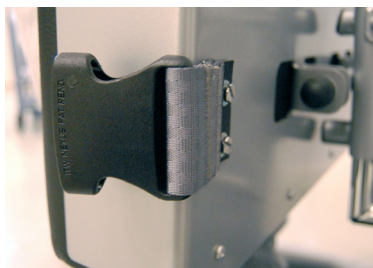
## Always read the manual

Always read the manuals for all assistive devices used during a transfer.  
Keep the manual where it is accessible to users of the product.  
Do not leave the patient unattended during a transfer situation.

## Placement and use

Screw the clasps for the CalfStrap to the back of the knee support on MiniLift125/MiniLift200. Use the accompanying spacers/washers between the leather and the screws.

Adjust the length of the strap to suit the patient, allowing sufficient support behind the calves. Then, place the CalfStrap behind the patient's calves and fasten it with the clasps, one on either side.

### Material:

Polyester (60%) with a Polyurethane layer (40%).

### Care of the product:

Read the product label.

Do not use rinsing agent.

To prolong product life,  
avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

# SystemRoMedic™

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)