



Comfortable and functional slide and draw sheets for easy transfers

All too often care staff still work with laborious, out-dated methods during bed transfers of people with reduced function. The multi-functional WendyLett family has been developed to facilitate turning, moving higher up in bed and other small transfers in bed. Together Direct Healthcare Group's three WendyLett sheets make an in-situ, well thought-through system of slide and draw sheets which work in combination with each other and contribute to a simpler everyday life for both users and caregivers.

Direct Healthcare
Group

Advancing Movement & Health®



Three sheets for all occasions

Superficially, WendyLett is just a wonderful satin sheet. Examined more closely, you will soon discover that WendyLett is also an excellent transfer device.

The unique, deceptively simple concept lies in the weave and the weft of the different fabrics' structures. The sliding surfaces of the sliding sheets, for example, are "slippery" in one direction, but not in the other. With the right material and the right combinations of sheets with different functions, it becomes possible to perform very easy transfers. Even large and heavy users can be positioned in bed with considerably less effort for staff, which provides for substantially improved ergonomics.

Furthermore, the WendyLett slide and draw sheets are always in-situ. They are products which make everyday life more pleasant for the user while reducing the load and making work easier for the caregivers. You could say that WendyLett is revolutionary bedlinen!



Slide and draw sheets

The WendyLett family consists of three different length sheets. The WendyLett base sheet has a satin sliding section which allows the fabric to slide in one direction, but not in the other, making it easier for the user to turn in bed independently.

To accompany the WendyLett base sheet there are two different draw sheets – the WendyLett2Way and WendyLett4Way. The difference between 2Way and 4Way is that while 2Way slides in two directions, 4Way, which has an anti-friction material on the underside, slides in four directions. When combining one of these draw sheets with the WendyLett base sheet, turning or repositioning becomes light work for the caregivers, regardless of which type of positioning need the user has.

Which draw sheet to be used is determined by a risk assessment including the user's needs and abilities, as well as the caregiver's ability and the environment.

The right WendyLett for each situation

All WendyLett sheets can be used in home and nursing care as well as in hospitals and other acute environments. The system is suitable for both passive or partially-active users, who experience some difficulties in turning themselves in bed.

A combination of WendyLett base sheet and WendyLett4Way can also contribute to the user's rehabilitation, as the user is able to move himself, by making micro moves and small positional changes.

To prevent the user from sliding down when sitting up in bed, it is important that the draw sheet is folded-in, about 10cm under the thighs before it is tucked into place under the mattress.

Despite the simplicity and ease-of-use, the WendyLett system is remarkably functional during all kinds of heavier bed transfers. Transfers without friction and shearing damage of the skin of the user are also an important part of prevention and treatment of pressure-related ulcers.

A fabric that speaks

The colours and patterns of the fabric in the WendyLett slide and draw sheets are designed to indicate the function of the sheet and thereby also the needs of the user. The WendyLett base sheet has, just like WendyLett2Way, stripes which signal in which direction sliding is facilitated. WendyLett4Way slides in multiple directions and, therefore, the design is checked. This unique concept was invented by us. By the distinct pattern, the caregivers can immediately see and understand the function of the sheet. The instruction is simply "embedded" in the linen. This provides for confidence in the work and saves time.

Simple maintenance

The WendyLett sheets are just as easy to wash as are regular sheets and considerably easier to maintain, compared to other types of aids. They can handle up to 90°C/194°F which fulfils strict hygiene requirements. The different patterns are easily distinguished in the laundry and they can therefore be included in the ordinary laundry loads.



WendyLett

Base sheet



A wonderful satin sliding sheet

WendyLett base sheet is a satin sliding sheet available in two colours; white and grey with narrow white stripes, and in many different sizes. The sliding takes place sideways along the direction of the stripes.

Facilitates turning in bed

WendyLett is intended for users who have difficulties turning in bed. The smooth and silky central part, enables turning with minimal effort. Sliding sideways is made easy without risk of sliding off the bed by the anti-slip material at both sides of the sheet. The material also prevents the user from sliding down in bed.

WendyLett sliding sheet is bedded in the bed like a normal sheet, with the smooth surface in the middle of the bed. For users who need more assistance, WendyLett can be combined with WendyLett2Way draw sheet or WendyLett4Way sliding draw sheet.



WendyLett2Way

Draw sheet



A draw sheet to be combined with WendyLett base sheet

WendyLett2Way is a draw sheet made of a material which is adapted to WendyLett base sheet so that sliding down in bed is much reduced. WendyLett2Way is available in two colours: white or white with narrow grey stripes, and in several sizes. Sliding takes place sideways, along the direction of the stripes.

For easy and comfortable turning of users

WendyLett2Way is placed on top of a WendyLett base sheet. The combination of these two WendyLett sheets makes it easy for the caregiver to turn the user sideways in the direction of the stripes. Fold the edge of the draw sheet and fill your hand with fabric, creating a good, strong ergonomic grip. Work methodically, using straight arms and leaning or stepping backwards to create a sliding transfer sideways, then turn the user.

After turning, the gliding movement can be inhibited by bedding WendyLett2Way in under the mattress on both sides of the bed.



WendyLett4Way

Draw sheet



A sliding draw sheet to be combined with WendyLett base sheet

The multi-directional WendyLett4Way sliding draw sheet is made of a material which is designed to work in combination with WendyLett base sheet. WendyLett4Way is grey/white checked and available in several sizes. Sliding takes place in four directions; sideways as well as upwards and downwards, as indicated by the checked pattern.

Easy and comfortable turning and positioning of users in bed
WendyLett4Way is placed on top of a WendyLett base sheet. A marking tag indicates which side should be facing upwards. WendyLett4Way has an anti-friction material on the underside and the combination of these two WendyLett sheets allows for multi-directional moves in bed. Procedures such as turning in bed, moving higher up/lower in bed or re-adjusting a user while lying becomes simple to achieve with this combination, even with the heavier or passive user. The function of WendyLett4Way also makes it possible to perform gentle and comfortable transfers of users in pain.

After turning, the gliding movement can be inhibited by bedding WendyLett4Way in under the mattress on both sides of the bed.



WendyLean

Pillow case




For easy and comfortable moving higher up in bed or into a seated position

The patented, self-retracting handles are not the only features of this pillowcase, but they are very handy when you need to help a user move higher up in bed or into a seated position at the edge of the bed.

The underside of the pillowcase is made of smooth satin which makes it very easy both to position the pillow and to pull the pillow in under the shoulders for raising the upper body of the user. The stripes indicate in which direction sliding is facilitated.

WendyLean is a very simple device which is always at hand. It fits the smaller pillows and is easy to remove and wash. It can be washed in up to 90°C/194°F which fulfils strict hygiene requirements and it can be included in the ordinary laundry loads.

The top of the pillow case is made of soft polyester/cotton material for a good night's sleep

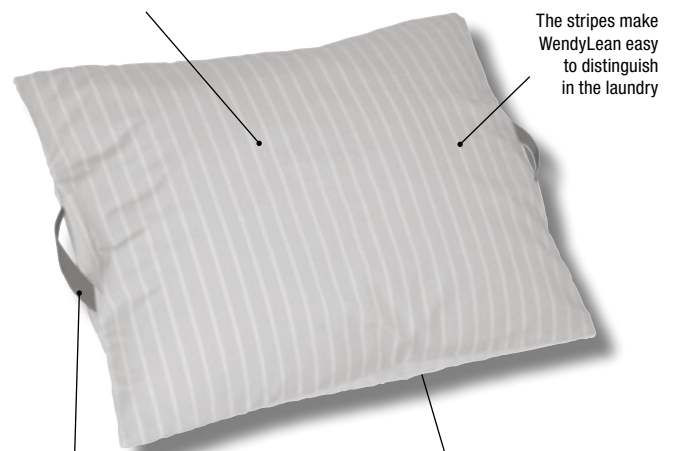


The stripes make WendyLean easy to distinguish in the laundry

Strong handles provide for a good grip and added safety

The underside is made of a satin material, making it easy to move when turning and changing position





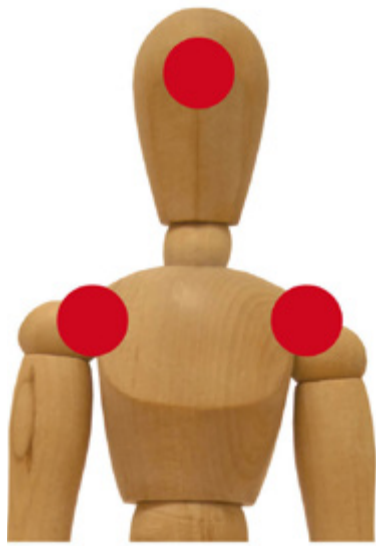

PREVENTS
pressure ulcers

A transfer aid which prevents pressure-related ulcers

Using WendyLett, small transfers, turning and positioning of users in bed can be performed without friction and shearing damage of the skin of the user. The sheets are made of a very fine material and can favourably be placed on top of a pressure-relieving mattress without compromising the pressure-relieving properties of the mattress. In order to provide the user with maximum comfort, we have also eliminated all seams across the sheets and made the joint seams of different functional materials really flat and low.



Communicating and self-instructing design

The colours and patterns of the fabric in the WendyLett slide and draw sheets are designed to indicate the function of the sheet and thereby also the needs of the user. White and grey stripes and checked patterns communicate in which direction sliding is facilitated and how the sheet should be used. The design also makes the sheets easy to distinguish in the laundry. And, on top of this, the WendyLett sheets look really good.



A comfortable functional material

The WendyLett sheets are made of a mixed material fabric. Wonderful, smooth cotton provides for comfort while strong and smart polyester ensures functionality and durability. The quality of the fabric is continuously and carefully monitored with regards to strength, durability, burls, washing properties and non-inflammability.

Product specification

	Item No.	Product	Size	Colour
WendyLett – sliding sheet (Can be combined with WendyLett2Way draw sheet or WendyLett4Way sliding draw sheet)				
Flat base sheets	1609	WendyLett, 60cm/24" sliding surface (bed 80/90cm – 32"/36")	100 x 200cm/40" x 80"	white
	1614	WendyLett, 60cm/24" sliding surface (bed 80/90cm – 32"/36")	140 x 200cm/56" x 80"	white
	1629	WendyLett, 60cm/24" sliding surface (bed 80/90cm – 32"/36")	100 x 200cm/40" x 80"	grey/white striped
	1634	WendyLett, 60cm/24" sliding surface (bed 80/90cm – 32"/36")	140 x 200cm/56" x 80"	grey/white striped
	1639	WendyLett, 70cm/28" sliding surface (bed 105cm/42")	200 x 200cm/80" x 80"	grey/white striped
	1667	WendyLett, 100cm/40" sliding surface (bed 120cm/48")	100 x 220cm/40" x 88"	grey/white striped
	1668	WendyLett, 100cm/40" sliding surface (bed 120cm/48")	140 x 220cm/56" x 88"	grey/white striped
	1669	WendyLett, 100cm/40" sliding surface (bed 120cm/48")	200 x 220cm/80" x 88"	grey/white striped
	1681	WendyLett, extra wide to allow for optional placing of sliding surface. Suitable for King and Queen sized beds	370 x 200cm/145" x 80"	grey/white striped
Fitted base sheets	1641	WendyLett, 70cm/28" sliding surface (bed 90cm/36"), elasticated fitting, UK standard single	90 x 200cm/36" x 80"	grey/white striped
	1651	WendyLett, 80cm/32" sliding surface (bed 105cm/42"), elasticated fitting, suitable for deep cell dynamic mattress	105 x 200cm/42" x 80"	grey/white striped
	1661	WendyLett, 95cm/38" sliding surface (bed 120cm/48"), elasticated fitting	120 x 200cm/48" x 80"	grey/white lined
	1671	WendyLett, 110cm/44" sliding surface (bed 120cm/48"), elasticated fitting	140 x 200cm/56" x 80"	grey/white lined
	WendyLett2Way – draw sheet (To be combined with WendyLett sliding sheet)			
	1620	WendyLett2Way	90 x 180cm/36" x 72"	white
	1631	WendyLett2Way	100 x 200cm/40" x 80"	white/grey striped
	1636	WendyLett2Way	140 x 200cm/56" x 80"	white/grey striped
WendyLett4Way – sliding draw sheet (To be combined with WendyLett sliding sheet)				
	1637	WendyLett4Way (3/4 length)	140 x 200cm/56" x 80"	grey/white checked
	1647	WendyLett4Way (full length)	200 x 200cm/80" x 80"	grey/white checked
	1648	WendyLett4Way, inco, sliding draw sheet/ incontinence protection with extra fabric on both sides	75 x 200cm/30" x 80"	grey/white checked
	1649	WendyLett4Way, inco with handles, sliding draw sheet/ incontinence protection with 7 handles on each side	140 x 120cm/56" x 48"	grey
	1652	WendyLett4Way, draw sheet for heavier users Also suitable for King and Queen sized beds	140 x 260cm/56" x 103"	grey/white checked
	1657	WendyLett4Way, with handles, sliding draw sheet	140 x 190cm/56" x 75"	grey/white checked
1680	WendyLean – pillowcase with handles WendyLean	50 x 60cm/20" x 24"	grey/white striped	

SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.

Transfer, between two locations. **Positioning**, within the same location. **Support**, for mobility. **Lifting**, both manual and mechanical.

Direct Healthcare Group

Advancing Movement & Health™

Withey Court, Western Industrial Estate
Caerphilly, United Kingdom CF83 1BF
T: +44 (0) 800 043 0881
F: +44 (0) 845 459 9832
E: info@directhealthcaregroup.com
directhealthcaregroup.com