EasyRoll Inco

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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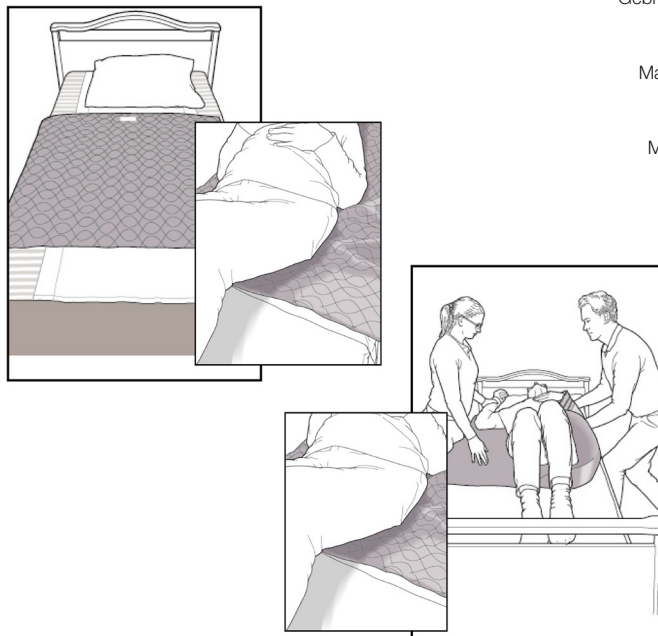
REF

10991	80018P
10992-5	80019
10990-5	80011PS
1299k	80010P
14545GR	90.200J01
16060GR	105.200J01
16090GR	120.200J01
80011-1	
80011B	
80011P	
80011PL	
80017-1	
80017P	
80018	

SWL 10991: 200 kg/440 lbs

SWL 10992-5, 10990-5: 400 kg/880 lbs

SWL 1299k, 14545GR, 16060GR, 16090GR, 80011-1, 80011B, 80011P, 80011PL, 80017-1, 80017P, 80018, 80018P, 80019, 80011PS, 80010P, 90.200J01, 105.200J01, 120.200J01: 150 kg/330 lbs

Table of contents

Product information	3
English	6
Svenska	9
Norsk	12
Dansk	15
Suomi	18
Deutsch	21
Nederlands	24
Français	27
Italiano	30
Español	33
Symbols	36

Product information/ Produktinformation/ Produktinformasjon/ Produktinformation/ Tuotetiedot/ Produktinformation/ Productinformatie/ Information produit/ Informazioni sul prodotto/ Información del Producto

Art. no.	Product/Produkt/ Tuote/ Produktname/Product/ Produit/ Prodotto/ Producto	Size/ Storlek/ Størrelse/ Koko/ Größe/ Maat/ Taille/ Dimensione/ Tamaño
EasyRoll - sliding sheet/ glidilakan/ glidelaken/ glidelagen/ liukuvan telttakankaan/ piirtää arkki/ gleittlaken/ glijlaken/ feuille de glissement/ telone scorrevole/ sábana de deslizamiento		
(Can be combined with EasyRoll 2way drawsheet or EasyRoll 4way sliding drawsheet)		

411051P	EasyRoll Sliding sheet	100cm long (for 105cm bed)
4106	EasyRoll Sliding sheet	190cm long (for 90cm bed)
4404IB	EasyRoll Sliding sheet	100cm long (for 90cm bed)
418011	EasyRoll Sliding sheet	100cm long (for 80cm bed)
418021	EasyRoll Sliding sheet	140cm long (for 80cm bed)
418031	EasyRoll Sliding sheet	200cm long (for 80cm bed)
419011	EasyRoll Sliding sheet	100cm long (for 90cm bed)
419021	EasyRoll Sliding sheet	140cm long (for 90cm bed)
419031	EasyRoll Sliding sheet	200cm long (for 90cm bed)
4110511	EasyRoll Sliding sheet	100cm long (for 105cm bed)
4110521	EasyRoll Sliding sheet	140cm long (for 105cm bed)
4110531	EasyRoll Sliding sheet	200cm long (for 105cm bed)
4112011	EasyRoll Sliding sheet	100cm long (for 120cm bed)
4112021	EasyRoll Sliding sheet	140cm long (for 120cm bed)
4112031	EasyRoll Sliding sheet	200cm long (for 120cm bed)
41701PS	EasyRoll Sliding sheet	100cm long (for 70cm bed)
41702PS	EasyRoll Sliding sheet	140cm long (for 70cm bed)
41703PS	EasyRoll Sliding sheet	200cm long (for 70cm bed)
41801PS	EasyRoll Sliding sheet	100cm long (for 80cm bed)
41802PS	EasyRoll Sliding sheet	140cm long (for 80cm bed)
41803PS	EasyRoll Sliding sheet	200cm long (for 80cm bed)
41901PS	EasyRoll Sliding sheet	100cm long (for 90cm bed)
41902PS	EasyRoll Sliding sheet	140cm long (for 90cm bed)
41904TV	EasyRoll Sliding sheet	160cm long (for 90cm bed)
41903PS	EasyRoll Sliding sheet	200cm long (for 90cm bed)
411051PS	EasyRoll Sliding sheet	100cm long (for 105cm bed)
411052PS	EasyRoll Sliding sheet	140cm long (for 105cm bed)
411053PS	EasyRoll Sliding sheet	200cm long (for 105cm bed)
411201PS	EasyRoll Sliding sheet	100cm long (for 120cm bed)
411202PS	EasyRoll Sliding sheet	140cm long (for 120cm bed)
411203PS	EasyRoll Sliding sheet	200cm long (for 120cm bed)
41902PSB	EasyRoll Sliding sheet	140x200cm (for 90cm bed)
419031D	EasyRoll Sliding sheet	200cm long (for 90cm bed)
418031R	EasyRoll Sliding sheet	200cm long (for 80cm bed)
419031R	EasyRoll Sliding sheet	200cm long (for 90cm bed)
4110531R	EasyRoll Sliding sheet	200cm long (for 105cm bed)
519031R	EasyRoll Sliding sheet	200cm long (for 90cm bed)
45901S	EasyRoll Sliding sheet Anticubitus	100cm long (for 90cm bed)
45902S	EasyRoll Sliding sheet Anticubitus	140cm long (for 90cm bed)
45903S	EasyRoll Sliding sheet Anticubitus	200cm long (for 90cm bed)
451051S	EasyRoll Sliding sheet Anticubitus	100cm long (for 105cm bed)
451052S	EasyRoll Sliding sheet Anticubitus	140cm long (for 105cm bed)

451053S	EasyRoll Sliding sheet Anticubitus	200cm long (for 105cm bed)
451201S	EasyRoll Sliding sheet Anticubitus	100cm long (for 120cm bed)
451202S	EasyRoll Sliding sheet Anticubitus	140cm long (for 120cm bed)
451203S	EasyRoll Sliding sheet Anticubitus	200cm long (for 120cm bed)
51901FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 90cm bed)
51902FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 90cm bed)
51903FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 90cm bed)
511051FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 105cm bed)
511052FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 105cm bed)
511053FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 105cm bed)
511201FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 120cm bed)
511202FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 120cm bed)
511203FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 120cm bed)
519031FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 90cm bed)
5110531RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 105cm bed)
5112031RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 120cm bed)
42901PS	EasyRoll Sliding sheet w Barrier	100cm long (for 90cm bed)
42902PS	EasyRoll Sliding sheet w Barrier	140cm long (for 90cm bed)

EasyRoll 2Way - drawsheet/ draglakan/ stikkklakenet/ træktagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera

(To be combined with EasyRoll sliding sheet)

2101	EasyRoll 2way Drawsheet	100x180cm
2101S	EasyRoll 2way Drawsheet	140x200cm

EasyRoll 4Way - drawsheet/ draglakan/ stikkklakenet/ træktagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera

(To be combined with EasyRoll sliding sheet)

4011S1	EasyRoll 4way Sliding Drawsheet	150cm
4011S	EasyRoll 4way Sliding Drawsheet	140x200cm
4011L	EasyRoll 4way Sliding Drawsheet	200x200cm
4011SSL	EasyRoll 4way Sliding Drawsheet	150x200cm
4111T	EasyRoll 4way Sliding Drawsheet	140cm long (for 90cm bed)
4112T	EasyRoll 4way Sliding Drawsheet	140cm long (for 105cm bed)
4113T	EasyRoll 4way Sliding Drawsheet	140cm long (for 120cm bed)
4111LT	EasyRoll 4way Sliding Drawsheet	200cm long (for 90cm bed)
4112LT	EasyRoll 4way Sliding Drawsheet	200cm long (for 105cm bed)
4113LT	EasyRoll 4way Sliding Drawsheet	200cm long (for 120cm bed)
4111B	EasyRoll 4way Sliding sheet	140x200cm
4112B	EasyRoll 4way Sliding sheet	140x215cm
4113B	EasyRoll 4way Sliding sheet	140x230cm
4111LB	EasyRoll 4way Sliding sheet	200x200cm
4112LB	EasyRoll 4way Sliding sheet	200x215cm
4113LB	EasyRoll 4way Sliding sheet	200x230cm

EasyRoll - pillowcase/ örngott/ putetrek/ hovedpudebetræk/ tynnyliina kahvat/ Kissenbezug/ kussensloop/ Taie d'oreiller/ federa/ unda de almohada

4303	EasyRoll Sliding Pillow case	50x60cm
4304	EasyRoll Pillow case w Barrier	50x60cm

EasyRoll 2way Inco, EasyRoll 4way Inco, EasyRoll Chair pad, EasyRoll Underpad and EasyRoll Mattress Protector

10991	EasyRoll Inco 2way Sliding sheet	185x90cm
10992-5	EasyRoll Inco 4way Sliding sheet	140cm
1299K	EasyRoll Chair pad	45x45cm
14545GR	EasyRoll Chair pad	45x45cm
16060GR	EasyRoll Chair pad	45x45cm
16090GR	EasyRoll Chair pad	45x45cm
80011-1	EasyRoll Underpad	85x90x180 cm
80011B	EasyRoll Underpad	80x180cm
80011P	EasyRoll Underpad	80x180cm
80011PL	EasyRoll Underpad	90x210cm
80017-1	EasyRoll Underpad	85x90 cm
80017P	EasyRoll Underpad	85x90 cm
80018	EasyRoll Underpad	150x90 cm
80018P	EasyRoll Underpad	150x90 cm
80019	EasyRoll Underpad	120x90x180cm
80011PS	EasyRoll Underpad w Barrier	70x190cm
80010P	EasyRoll Underpad w Sliding sheet	80x85x180 cm
90.200J01	EasyRoll Mattress protector	90x200cm
105.200J01	EasyRoll Mattress protector	105x200cm
120.200J01	EasyRoll Mattress protector	120x200cm

Product combinations, Produktkombinationer, Produkt kombinasjon, Produkt kombination, Tuoteyhdistelmä, Produktkombination, Product-combinatie, Combinaison de produit, Combinazione di prodotti, Combinación del producto

Art. no.	Combination	Art. no.	Combination
4111	4111 + 4111T	41401S	41401S + 4011S
4112	4112 + 4112T	41402S	41402S + 4011S
4113	4113 + 4113T	41403S	41403S + 4011S
4111L	4111L + 4111LT	41401LS	41401LS + 4011L
4112L	4112L + 4112LT	41402LS	41402LS + 4011L
4113L	4113L + 4113LT	41403LS	41403LS + 4011L
41402SET	41402SET + 4011S	10990-5	10990-5 + 10992-5
41403SET	41403SET + 4011L		

EasyRoll Inco

SystemRoMedic™

Instructions for use - English

EasyRoll 2way with Inco is a sliding drawsheet. EasyRoll 4way with Inco is a sliding drawsheet that is used in combination with EasyRoll sliding sheet.

EasyRoll 2way Inco, EasyRoll 4way Inco, EasyRoll Chair pad, EasyRoll Underpad and EasyRoll Mattress Protector are made from a highly absorbent material that can be laundered, is intended for users that are in need of incontinence protection.

The sliding drawsheet and underpad also has an anti-friction material on the underside, which facilitates repositioning laterally as well as upwards and downwards on the bed.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement

The EasyRoll slide sheet is placed with the shiny side in the middle of the bed and with the EasyRoll4Way, Inco on top.

EasyRoll 4way Inco with handles, Art No. 10990-5

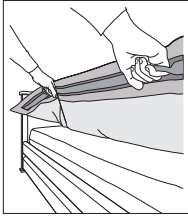
Be aware that there is a risk of the user sliding out of bed. Raise the bed rails before leaving the user.


Caution! Never attempt to lift a user with the handles on both sides of the sheet simultaneously.

Turning


Turn the user by grasping the sliding drawsheet at the level of the hip and shoulder.

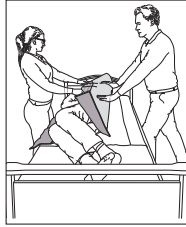Turning using the sliding drawsheet can be done by 1 or 2 caregivers. When two caregivers are assisting, one can draw the sliding drawsheet (as above) and the other can press the sliding drawsheet down against the bed at the hip and shoulder while at the same time pushing (image 2), using the so-called “press down method”. Then, the user is turned on one side (images 3 and 4) and repositioned comfortably in the right position.

Decide, in each individual case, whether the user can be left without the bed side rails raised. Remember, there is a risk that the user may slide out of bed.

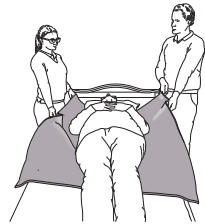Higher up in bed with


When repositioning the user higher up in bed the sliding drawsheet is grasped at the centres of gravity on either side of the user, either by grasping the fabric or the handles, depending on the model. Work in an upright stance and shift your weight while drawing the user higher up in bed.

Sitting up in bed

The user can be prevented from sliding downward in bed by tucking in the lower 10 cm. By folding the sliding drawsheet double the friction side faces downward, against the underlying surface, thereby preventing the user from sliding downward in bed.

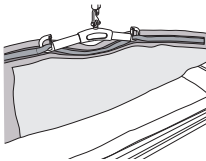Advice

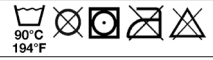
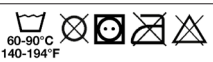
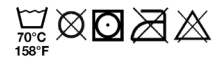
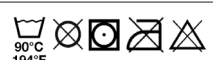
If positioning is strenuous, the EasyRoll4Way, Inco with handles (Art. No. 10990-5, 10992-5) can be used in combination with a user lift. Only connect the handles on one side of the sheet to the slingbar loops. See image. Positioning in combination with a lift reduces friction under half of the body and facilitates repositioning of the user in bed, e.g., lying on the side or higher up in bed.

Advice: Use AdjustmentLoops (Art. No. 49182006) to increase the distance between the slingbar and the sliding drawsheet.

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use. See www.directhealthcare-group.com for a complete list of distributors.

Art. No.	Material	Care of the product
90.200J01 105.200J01 120.200J01 105.200J01	CottonJersey; Barrier PU	
80011B	CottonPolyester; Barrier PU; Absorbent Polyester	
80011	CottonPolyester; Barrier PU; Absorbent PolyesterRayon	
80017 80018	CottonPolyester; Barrier PU; Absorbent PolyesterRayon	
80019	CottonPolyester; Barrier PU; Absorbent PolyesterRayon	
10991 10992-5 10990-5	Duchesse, CottonPolyester; Barrier PU; Absorbent PolyesterRayon	
80011PS 80011P 80011PL	Duchesse, CottonPolyester; Barrier PU; Absorbent PolyesterRayon Polyester; Barrier PU; Absorbent Polyester	
80017P 80018P	Polyester; Barrier PU; Absorbent Polyester	
1299k	Polyester; Barrier PU; Absorbent PolyesterRayon	
80010P	Polyester; Barrier PU; Absorbent PolyesterRayon	
14545GR 16060GR 16090GR	Velour; Barrier PU; Absorbent PolyesterRayon	

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

SystemRoMedic™

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.