EasySlide

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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1400 x 900 mm/ 55.1" x 35.4"

1420

2000 x 900 mm/ 78.7" x 35.4"

1470

REF

1420

1470

SWL: 180 kg/ 400 lbs

Table of contents

English	3
Svenska	5
Norsk	7
Dansk	9
Suomi	11
Deutsch	13
Nederlands	15
Français	17
Italiano	19
Español	21
Accessories	23
Symbols	23

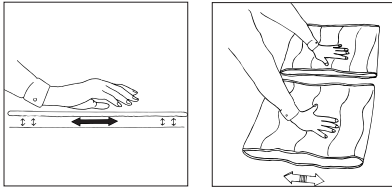
EasySlide is used for turning in bed. Since this model of EasySlide has clasps that allow it to be secured to the bed, the bed's backrest can be raised without the user sliding down. The product can be bedded into the bed.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

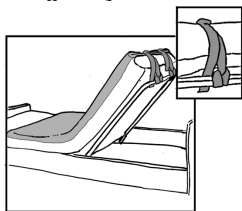
Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement of EasySlide

The EasySlide is placed with the locking clasps facing upward and towards the head of the bed. Place a sheet over the EasySlide and fold it in under the mattress on both sides of the bed for transfers to/from the bed and for repositioning.

Using EasySlide

Tighten the straps before the bed backrest is raised. Loosen the straps when the backrest is lowered.

Turn the user by pulling on the bed sheet that covers the EasySlide. When turning is complete, tuck the sheet in on both sides.

Note

Be aware that there is a risk of the user sliding out of bed. Always raise the bed side rails when the user is unattended.

Material

Polyester, microfibre, polyamide

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Vida útil estimada

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Accessories

Art. no	Product	Description
1451	Protective cover	For EasySlide 1420
1481	Protective cover	For EasySlide 1470
1599	LockingStrap	Spare part

1430	Bundle	EasySlide 1420 + Protective cover 1451
1480	Bundle	EasySlide 1470 + Protective cover 1481

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

SystemRoMedic®

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.