# FootStool

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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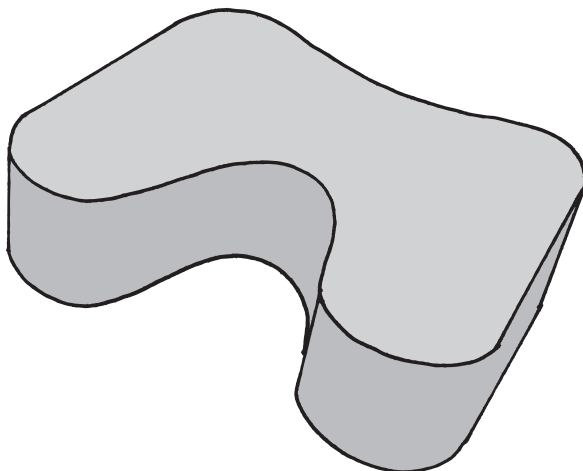
**REF**

8090  
8095

SWL: 200 kg/ 440 lbs

## Table of contents

English .....	3
Svenska .....	4
Norsk .....	5
Dansk .....	6
Suomi .....	7
Deutsch .....	8
Nederlands .....	9
Français .....	10
Italiano .....	11
Español .....	12
Český .....	13
Product information .....	14
Symbols .....	14

FootStool is used to raise the position of the knees and thereby enable safer, easier bed-to-wheelchair transfer in a seated position. FootStool also provides good support under the feet during visits to the toilet, or when repositioning farther back in a chair. FootStool is made of a material with closed cells, which makes it easy to keep clean.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly.

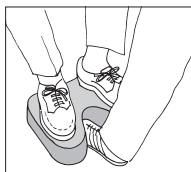
## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).



## Placement of FootStool

Place the FootStool under the user's feet when the user is sitting on the toilet or in a wheelchair. FootStool is recessed at the front to allow the caregiver to place a foot as close as possible to the stool. The back also has a small recess to facilitate placement close to the toilet.

## Using FootStool

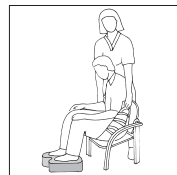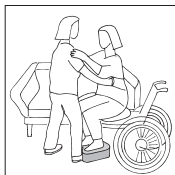FootStool complements other assistive devices such as EasyBelt/FlexiBelt, EasyGlide and SitWalk to facilitate transfers between two locations or for repositioning, for example, farther back in a chair. Since the knees are raised, the thighs are also lifted somewhat, which reduces friction and makes the transfer easier.

## Material

Polyethylene

## Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

**Note:** Do not subject the product to hard pressure or impact. This may leave indentations in the material.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Symbols

---

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)