XXL-Rehab Badetoiletstol med hjul
XXL-Rehab Shower Commode

DK

UK

MAX 325 KG

Kan med fordel anvendes til:

- Bariatriske brugere.
- Bariatriske afdelinger.
- Institutioner.
- Hjemmet.
- Hospitaler.

Ideal use for:

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DK

UK

Instructions for XXL-Rehab Shower Commode

⚠ CAUTION ⚠ CAUTION ⚠

- This product should not be used without proper instruction from a healthcare professional.
- Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.
- XXL-Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- If components are damaged or missing, contact XXL-Rehab immediately. Do NOT use substitute parts.
- XXL-Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from XXL-Rehab.

⚠ To ensure proper use of the XXL-Rehab Shower Commode, read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with ⚠. Also remember that in order to prevent the risk of injury to the user, the shower commode may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

USE

The XXL-Rehab Shower Commode is designed to help an individual weighing up to 325 kg during transfers to and from the bathroom, in the shower, and during lavatory use.

⚠ Using the product for individuals weighing more than 325 kg may pose a risk of tipping or damaging the shower commode.

⚠ Always use the XXL-Rehab Shower Commode on level, hard flooring with a maximum slope of 3 degrees, as otherwise there is a risk of tipping.

BEFORE USE

The armrests must be properly installed in the sockets on the side of the shower commode.

The width of the footrests may be gradually adjusted. The lock bolts beneath the foot plates must always be properly mounted in the footrest sockets and must not be secured to the footrests outside the sockets.

⚠ To prevent the risk of the user falling off the side of the shower commode, never use it without first properly installing the armrests.

DURING USE

⚠ Never stand on the foot plates or use them as a platform during transfers, as this will pose a high risk of tipping.

Pinch hazards

- Installing the armrests while the user is seated on the shower commode may present a pinch hazard between the armrests and the side of the commode.
- There is a risk of pinching a foot between the foot plates if the lock bolts are not properly mounted in the footrest sockets.
- There is a risk of pinching fingers and other parts of the body between the shower commode's seat frame and a toilet bowl or bedpan. Therefore, be particularly careful when placing the shower commode over the toilet or inserting a bedpan beneath the seat.

⚠ The shower commode is not designed to be driven over obstacles such as doorsteps, etc. To prevent the risk of tipping, only drive the shower commode on a level, hard surface.

⚠ Both wheels must be locked with the wheel locks when the user is seated on the shower commode, except when it is used for transport.

⚠ If the user requires assistance during transfers to the shower commode, the aides must be particularly aware of the risk of performing unsafe lifts or movements when placing the user on the shower commode. This applies whether the transfer is effected by means of a lift, slide board or support device.

MAINTENANCE AND CLEANING

The white frame components and the seat and armrests may be cleaned with water and a household detergent.

To thoroughly clean the seat: unscrew the four screws under the seat, clean and make sure to fasten the seat properly again.

Regularly check bolts and lock nuts, and retighten if required.

⚠ Failure to inspect the bolts as directed may cause functional failures.

GARANTI

The XXL-Rehab Shower Commode is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by XXL-Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.


XXL-Rehab Shower Commode is manufactured and tested in accordance with the international DS/EN 12182:2000 standard.

07/2014/JJP

MANUFACTURER

XXL-Rehab
 Fuglebækvej 1D
 DK-2770 Kastrup
 Denmark
 Tlf.: +45 70 25 25 22
 Fax: +45 70 25 25 44
 www.xxl-rehab.com

Maximum weight capacity is	325 kg
Total length	117 cm
Total outer width with standard armrests	68 / 78 cm
Total outer width with butterfly armrests	87 / 97 cm
Seat width with standard armrests	61 / 71 cm
Seat width with butterfly armrests	75 / 85 cm
Seat depth	56 cm
Seat tilt	No
Seat height	55 cm
Armrest width	6 cm
Width toilet opening	Diameter 24 cm, 35 cm long
Distance from floor to seat underside	44.5 cm
Pushbar from floor	100 cm
Seat to backrest	45 cm
Seat to footrest	40-42-44-46-48 cm
Height seat to armrest	25 cm
Weight	27 / 30 kg
Folding armrests	No
Swing-away armrests	Yes
Swing-away footrests	Yes
Removable footrests	Yes
Height adjustable footrests	Yes
Headrest	No
Individual wheel locks	Yes, 2 locks
Wheels	Diameter 125 mm
Frame	Powder-lacquered chromium molybdenum steel
Seat/back/armrests	Polyurethane
Footrest coating	PU foam
ISO - classification	ISO 09 12 03

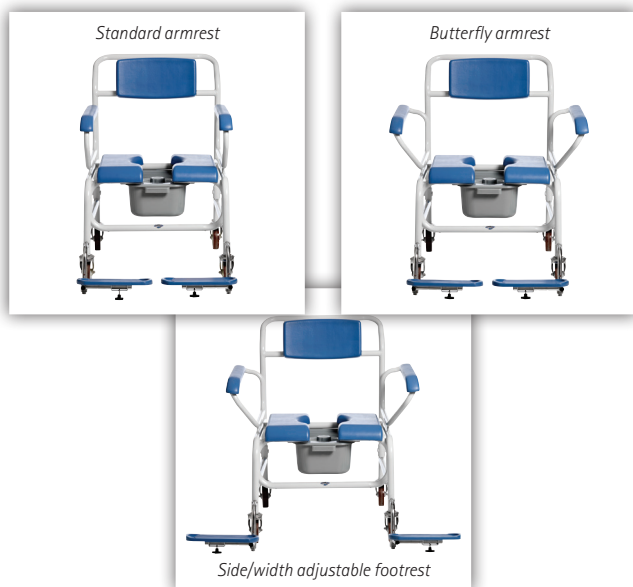Strongly engineered and functionally designed exclusively for bariatric users!

XXL-Rehab Minimaxx

XXL-Rehab Minimaxx
Push Motor

XXL-Rehab Manual
Wheelchair - The Eclipse

XXL-Rehab Manual
Wheelchair - The Eclipse Tilt

XXL-Rehab
HD Electric Wheelchair

XXL-Rehab Wheelchair Cushions

XXL-Rehab Shower Bench

XXL-Rehab
Bedside commode

XXL-Rehab
Shower Commode

XXL-Rehab
Shower Commode Wheeler

XXL-Rehab
Shower Commode Rise N Recline

XXL-Rehab
Shower Commode Tilt

XXL-Rehab Barton Transferstol

XXL-Rehab Patient/Dinner chair

XXL-Rehab Rollator King

XXL-Rehab Rollator Support

XXL-Rehab Stand Tall

XXL-Rehab Bariatric walking frame


XXL-Rehab Canes

XXL-Rehab Multifunktional Scale

XXL-Rehab Lift and Slings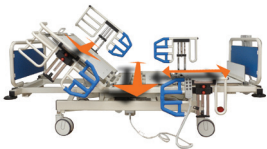

XXL-Rehab 400 bariatric hospital bed


XXL-Rehab Home Care Bed

XXL-Rehab Floor raiser

XXL-Rehab
Comfort Foam Mattress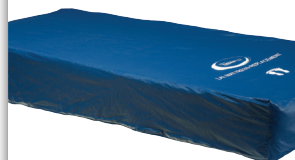

XXL-Rehab
Low Air Loss mattress

XXL-Rehab
Alternating mattress

XXL-Rehab Bariatric suit