
Dyna-Form® Pressure Redistributing Cushion Range

Instructions for Use - English
Bruksanvisning - Svenska
Brugsvejledning – Dansk
Käyttöohje – Suomi
Handleiding – Nederlands
Manuel d'utilisation – Français
Manual de usuario – Español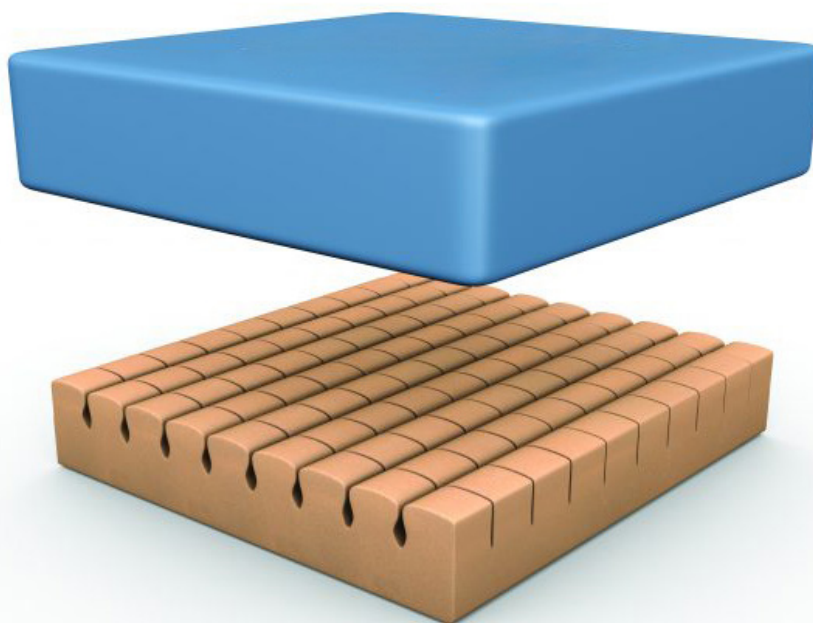

REF

INTE	CUS01
INTL	CUS02
INTM	CUS03
INTH	CUS04
INTV	CUS05
	CUS06
	CUS07
	CUS08
	CUS09

Table of Contents

English	03
Svenska	08
Dansk	13
Suomalain	18
Nederlands	23
Français.....	28
Español	33

Intended Use of the Product

The Dyna-Form pressure redistributing cushion range is a pressure redistributing foam cushion intended to be used in the prevention or treatment of pressure ulcers. The cushion is suitable for persons who are at high risk of pressure ulcers, including those with category 1 or category 2 pressure ulcers, alongside an individualised care plan.

Indications for Use

The Dyna-Form pressure redistributing cushion range is intended to be used by all kinds of users, including lay persons, in all healthcare environments including professional healthcare facilities and home healthcare.

Contraindications for Use

There are no known contraindications for use.

Safety Information

Visual inspection

- Inspect the packaging for any damage.
- Check that the correct product has been delivered.
- Check to ensure that the mattress is free from damage.

Always Read the User Manual

- Always read the user manual before use.
- Keep the user manual where it is accessible to users of the product.
- Always ensure that you are referring to the latest version of the user manual.
- The most recent version of the user manual is available to download from the DHG website: www.dhg-healthcare.com
- It is strongly prohibited to modify the original product.

Warnings and Cautions

Warnings

- Improper use of this product may cause injury or damage. If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional or provider before attempting to use this cushion.
- Do not use this cushion without first completely reading and understanding these instructions.
- Do not use a defective product.
- Which cushion is suitable for your individual use should be decided by your clinician who will also determine the most appropriate pressure ulcer prevention or treatment plan and the relevant nursing procedures.
- The usage of additional layers between the cushion and the user, such as sheepskins or other similar items, may reduce the effectiveness of the pressure redistributing properties of the cushion or indeed lead to an increase of body temperature, which may cause skin maceration and is therefore not advisable. Please seek clinical advice should you wish to use any additional covering other than the cover which has been supplied with the cushion by the manufacturer.
- Ensure the cushion is correctly sized to the user.
- Ensure the cushion is placed the correct way up and in the correct rotation, as indicated on the cushion cover.
- Do not use the cushion without the cover that has been supplied by the manufacturer.
- Inspect the inner and outer surfaces of the cover, foam, and their zip fastenings regularly for signs of damage. If the cover is stained, soiled, or torn, the foam should also be examined for contamination.

Cautions

- Prior to using the cushion, ensure that the surface it is being placed on has no rough edges that may damage the cushion.
- Ensure that the zip is closed and the zip flap is completely covering the zip.
- Avoid contact with sharp objects that may damage the cushion.
- It is recommended that a minimum of an annual audit of all cushions should be adhered to.
- Cushions can become damaged when stored incorrectly. Cushions should be stored off the floor and in a clean, dry environment.
- Do not store your cushion near heating devices such as electrical fires or radiators.

Symbols

	Read the user manual
	Medical device
	This product conforms to the requirements of Medical Device Regulation (EU) 2017/745
	This product conforms to the requirements of UK MDR 2002 for medical devices MDD/93/42 EEC
	General warning symbol
	Wash at 80°C
	Tumble dry on low
	Do not dry clean
	Do not bleach
	Do not iron
	No smoking
	Maximum user weight
	Do not use sharp instruments
	Do not use phenol

Using Your Cushion

Remove all packaging before use. Please dispose of all packaging in line with your relevant environmental policy.

Place the cushion on the wheelchair/chair, ensuring the correct orientation as indicated on the cover.

Take care when using mechanical items such as bridging board, PAT slides, or other patient transfer aids. Ensure that damage does not occur to the cushion due to sharp edges.

Patient Repositioning

Whilst your pressure redistributing cushion will provide redistribution properties and comfort, to prevent the build-up of pressure which may lead to tissue damage, and potentially pressure ulcer formation, it is important that the user either repositions themselves, or are repositioned, on a regular basis. This should be based on the clinical judgement of a qualified healthcare professional who will provide you with a suitable repositioning regime.

Maintenance and Inspection

Check cushions (foam and cover) for damage such as fluid ingress, stains, or rips, after the use of each user or on a minimum monthly basis. The individual performing the checks should be a suitably qualified and competent person.

To check the cushions:

1. Visually inspect the cover and zip.
2. Unzip the cover completely.
3. Check the inside of the cover for any staining.
4. Check the foam for any staining.
5. Always replace any damaged cover or stained foam, and dispose of the damaged product as per your local procedure.

It is recommended that an audit of all cushions is carried out on a minimum annual basis.

Cleaning and Maintenance

Cushion cleaning should take place before and after use, and between patients. The cushion can be used by more than one person, provided it is decontaminated as per below instructions, in between each user.

In the Case of Light Soiling

In the absence of gross contamination or unusual risk, the removal of dirt and spillages with clean, warm water and neutral detergent should be sufficient. However if required the cover can be wiped down with a solution of Sodium Hypochlorite diluted to 1,000ppm available chlorine. Ensure that the cover is rinsed with clean water using a single use non-abrasive cloth, and thoroughly dried. Ensure adequate ventilation.

In the Case of Heavy Soiling

In cases of gross contamination the cushion cover should be cleaned first with detergent and water then with a solution of Sodium Hypochlorite diluted to 10,000ppm available chlorine. Rinse well with clean water and a damp single use non-abrasive cloth. Ensure adequate ventilation.

Machine Laundering

If required, the cushion cover may be removed and machine-washed at a temperature of 80°C, for not less than 10 minutes.

Drying

To avoid shrinkage of the cover line dry in an indoor clean environment or tumble dry on a low heat setting not exceeding 40°C and for not longer than 10 minutes. Covers must be thoroughly dried before re-fitting to the cushion.

Re-Use, Storage and Disposal

Re-Use

The product is reusable. Before using the product again with a new user, it is recommended to clean the product using the above instructions.

Storage

It is advisable to store the cushion off the floor in a clean, dry environment. It is advisable to store the mattress flat. Do not store the cushion near

heating devices. Do not store the mattress where it might be prone to damage.

Disposal

All contaminated cushions must be disposed of as clinical waste in accordance with regional regulations and local legislation and guidelines.

Technical Information

Product	Warranty of Cushion (years)	Fire Testing	Foam Density (kg/m ³)	Nominal Hardness (N)	Minimum User Weight (kg)	Maximum User Weight (kg)	Weight of Product (kg)	Size*
CUS01 Dyna-Tek Pad	2	BS7176	35	125	0	127	1	Standard Size: 43cm x 43cm x 10cm
CUS02 Dyna-Tek Superior	2	BS7176	35 39	125 200	0	127	1.3	Standard Size: 43cm x 43cm x 10cm
CUS03 Dyna-Tek Owl	2	BS7176	40 39	250 200	0	152	1.3	Standard Size: 43cm x 43cm x 7.5cm
CUS04 Dyna-Flex Cushion	2	BS7176	50 39	200	0	222	1	Standard Size: 43cm x 43cm x 7.5cm
CUS05 Dyna-Tek Profile	1	BS7176	39	200	0	114	1	Standard Size: 43cm x 43cm x 5cm
CUS06 Dyna-Tek Posture Visco	2	BS7176	50 39	200	0	178	1.3	Standard Size: 43cm x 43cm x 7.5cm
CUS07 Dyna-Tek Gel	2	BS7176	34	100	0	159	3	Standard Size: 46cm x 46cm x 7.5cm
CUS08 Dyna-Tek Intelligent Air	2	BS7176	39	120	0	152	2	Standard Size: 46cm x 46cm x 7.5cm
CUS09 Transflo	2	BS7176	50	230	0	140 (leisure) 178 (standard) 203 (HD)	2.5	Standard Size: 43cm x 43cm x 9cm
INTE Integrity E 17S/28S	2	BS7176	36	125	0	107 177	1	Standard Size: 40cm x 40cm x 7.5cm

Product	Expected Life of Cushion	Fire Testing	Foam Density (kg/m ³)	Nominal Hardness (N)	Minimum User Weight (kg)	Maximum User Weight (kg)	Weight of Product (kg)	Size*
INTL Integrity L 17S	2	BS7176	41	150	0	107	1	Standard Size: 40cm x 40cm x 7.5cm
INTL Integrity L 28S	2	BS7176	50	230	0	177	1	Standard Size: 40cm x 40cm x 7.5cm
INTM Integrity M 17S	2	BS7176	60 50	135 140	0	107	1	Standard Size: 40cm x 40cm x 7.5cm
INTM Integrity M 28S	2	BS7176	60 50	135 230	0	177	1	Standard Size: 40cm x 40cm x 7.5cm
INTH Integrity H 17S	2	BS7176	60 50	135 230	0	107	1	Standard Size: 40cm x 40cm x 7.5cm
INTH Integrity H 28S	2	BS7176	60 50	135 230	0	177	1	Standard Size: 40cm x 40cm x 7.5cm
INTV Integrity V 17S	2	BS7176	60 39	135 200	0	107	3	Standard Size: 40cm x 40cm x 7.5cm
INTV Integrity V 28S	2	BS7176	60 39	135 200	0	177	2	Standard Size: 40cm x 40cm x 7.5cm

*Note: variable sizes are available upon request

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This page is left intentionally blank

This page is left intentionally blank


**PRESSURE ULCER PREVENTION
SAFE MOVING & HANDLING
SPECIALIST THERAPIES
BATHROOM SAFETY
RENTAL & SERVICE SOLUTIONS**

Direct Healthcare Group Ltd
Withey Court, Western Industrial Estate
Caerphilly, United Kingdom
CF83 1BF

T: +44 (0) 800 043 0881
F: +44 (0) 845 459 9832
E: info@directhealthcaregroup.com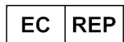

Direct Healthcare Group Sverige AB
Torshamnsgatan 35, SE-164
Kista, Sweden

T: +46 (0) 8 557 62 200

Australian Sponsor:

Direct Healthcare Group Pty Ltd.,
68 Howe Street, Osborne Park,
Western Australia 6017

T: +61 (0) 423 852 810