LiftSeat

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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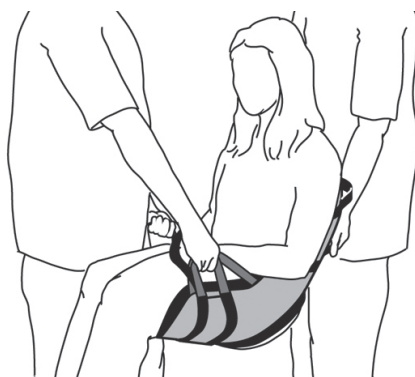
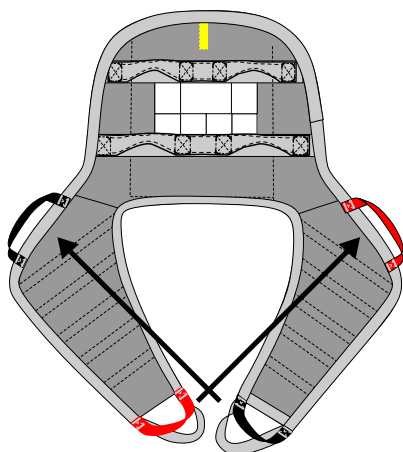
REF

4006
4007

SWL: 150kg/331lbs

Table of contents

English	4
Svenska	7
Norsk	10
Dansk	13
Suomi	16
Deutsch	19
Nederlands	22
Français	25
Italiano	28
Español	31
Size	33
Symbols	33

LiftSeat is a sling that is adapted for manual lifting. For the caregiver, it is easy to apply, and it gives the user a feeling of security during manual lifting in a seated position. It features leg sections, which means that the user does not sit on the sling. Instead, the sling provides support from the tailbone up over the head and under the legs. LiftSeat is designed for use in most common situations where manual lifting in a seated position is required, such as transfers from wheelchair to aircraft seat, etc.

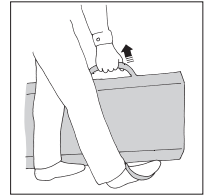
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Always read the user manual

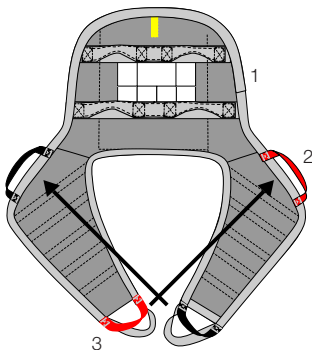
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.


The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Description of LiftSeat

1. LiftSeat has handles in two sections on the back support; choose the handle that enables the best working stance.
2. One of the handles on the upper part of the leg supports is red; after application, it should meet the red handle on the lower part of the opposite leg support. On the opposite side, these handles are black, and should meet each other after application.
3. On the lower part of the leg supports are two handles. For a smaller size, choose the innermost (red or black); for a larger size, choose the outermost (grey).

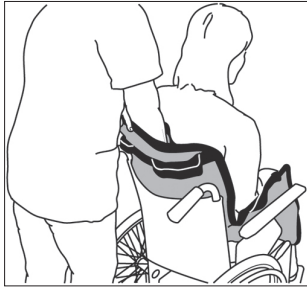
Application of LiftSeat when the user is sitting in a wheelchair/chair or in bed

Hold LiftSeat in one hand and grasp the lower edge of the sling with your other hand. This is easier if the user leans forward or can be helped to lean forward slightly. Guide the sling down behind the user's back, by sliding it along the back of the chair/wheelchair or along the bed, ideally, to the tailbone.

Take care to guide the sling down to the user's hips. Do this by pushing the sling into the space in the back corners of the wheelchair seat.

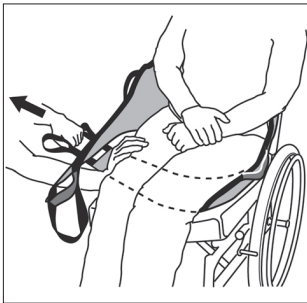Pull the leg supports out along the outside of the thighs. Sit facing the user and pull the lower edge of the leg support, so that the sling slides into place around the hips. Light pressure against the user's knees prevents him/her from sliding forward in the chair.

After checking to ensure that the leg supports are of equal length on both sides, place them under the user's thighs. Both leg supports should be under both of the user's legs.

Transfer or lifting

Choose handles: On the leg sections are two alternative handles. Choose the most appropriate handles according to the width of the user.

At the back, there are handles at two levels; choose the handle that enables the best working stance.

! Plan the transfer so as to minimize lifting.

Different positions for lifting

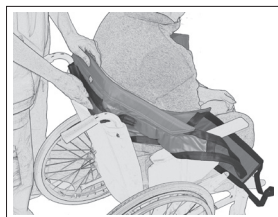
One caregiver on each side.

One caregiver in front and one caregiver behind.

Removal of the sling when the user is sitting


Fold the leg support and allow it to slide under itself when you pull it out at the outside of the thigh.

Then, pull the sling up behind the user's back.


For extra support behind the back, LiftSeat can be applied with the help of EasyGlide

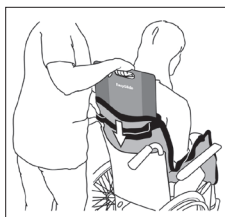Place EasyGlide in the pocket on LiftSeat.

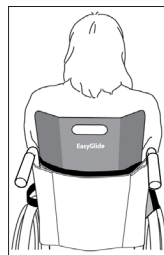Press LiftSeat with EasyGlide behind the user's back.

Leave EasyGlide behind the user's back for extra support during the transfer.

Material

Polyester

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Vida útil estimada

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Size

Art. no.	Productname	Size	Recommended user weight
4003	LiftSeat	XXS/XS	SWL: 15-45 kg/33-99 lbs
4006	LiftSeat	S/M	SWL: 35-90 kg/77-198 lbs
4007	Liftseat	L/XL	SWL: 80-120 kg/176-265 lbs

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

SystemRoMedic™

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.