LimbSling

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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REF

LimbSling
49200006

LimbSling Disposable
49292000

SWL LimbSling: 300kg/660lbs
SWL LimbSling Disposable: 150kg/330lbs

Table of contents

English	3
Svenska	5
Norsk	7
Dansk	9
Suomi	11
Deutsch	13
Nederlands	15
Français	17
Italiano	19
Symbols	21

LimbSling is designed to comfortably and conveniently assist lifting of a patient's leg or arm, for example, when dressing sores. The sling is applied by placing it under the limb and connecting the strap loops to the slingbar. SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

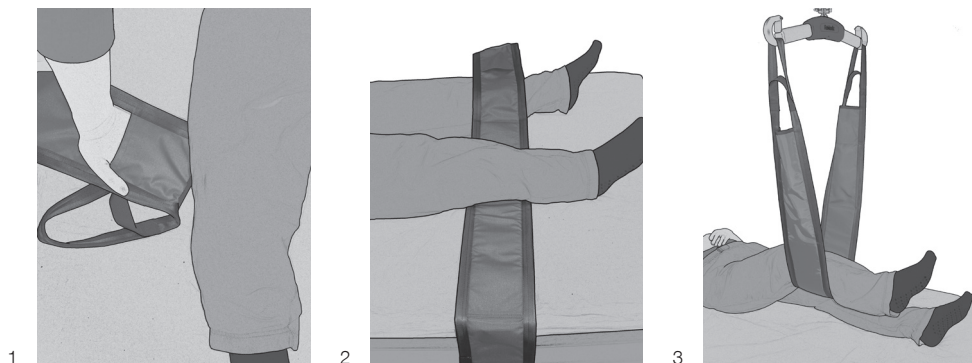
Always read the manual

Always read the manuals for all assistive devices used during a transfer.
Keep the manual where it is accessible to users of the product.
Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested. The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.
Contact your dealer if you have any questions.

Application of LimbSling under one leg

1. Apply LimbSling by placing your hand in the pocket at the edge of the sling and guiding it under the limb that is to be lifted.
2. Pull it out on the opposite side and connect the strap loops to the slingbar.
3. Start raising the lift, and then stop to ensure that everything is as it should be before lifting to the desired position.

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions. The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on www.directhealthcaregroup.com

Material:

Polyester: Durable material with low friction. Withstands high laundry temperatures.

Non-Woven: Polypropylene.

Care of the product:

Read the product label.

LimbSling

Do not use rinsing agent. To prolong product life, avoid tumble-drying.

LimbSling Disposable

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

Contact your local distributor if you have any questions about the product and its use. See www.directhealthcaregroup.com

for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

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Symbols

	<p>This product complies with the requirements of the Medical Device Regulation 2017/745</p>		<p>Visual Inspection</p>
	<p>Medical Device</p>		<p>Read the manual</p>
	<p>Caution</p>		<p>Product Code</p>
	<p>Manufacturer information</p>		<p>Batch Code</p>

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.