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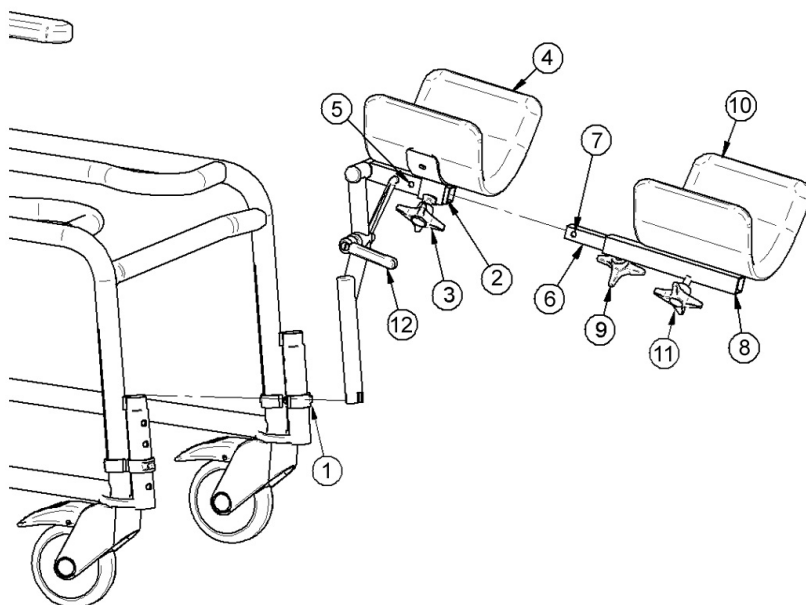
## **EN - Please read the instructions before using the product.**

***These instructions apply to the amputee support and leg support for the Combi chair.***

### **Intended use**

This product can be used with all Combi chairs.

The amputee support can be attached to the Combi chair to give support to a stump after an amputation. If it is necessary to support an entire leg, the leg support is mounted to the amputee support, to give support and relief for, as an example, a plaster cast.

### **Installation**

The amputee support can be installed on both the right and left sides of the chair

The amputee support is mounted on the holders for the foot support at the desired height. Make sure that the clips (1) are fully inserted.

*When moving to the opposite leg.*

Loosen thumbscrews (3) and remove the PUR support(4). Loosen handle (12), and swing the upper part (5) 180° over backwards. Tighten handle (12). Move the amputee support over to the opposite side by lifting up in the amputee support, and putting it on the opposite side, so the slots go down over the clip in the holder. Put on the PUR support, and tighten thumbscrews (3) in the desired position.

*To install the leg support.*

Remove the cap (2) for the amputee support. Loosen the thumbscrews (3), and pull the PVR support (4) out over the square tubing so that the hole (5) is visible. Put the middle tube (6) into the amputee support. Make sure that the pin (7) in the middle tube clicks correctly into the hole (5) of the amputee support.

Mount the leg support (8) on the middle tube and tighten the thumbscrews (9) at the desired length. The leg support (10) is fastened in the desired position with help of the thumbscrews (11).

### **Adjustment and use**

Lift the amputee support in its holder, and then swing out to one side.

To adjust the angle of the amputee support, loosen the handle (12). Adjust the amputee support to the desired position, and tighten the handle again.

The leg support can be placed in the desired position by loosening the thumbscrews and moving the tubes. After positioning make sure the thumbscrews are firmly tightened.

### **Maintenance**

The amputee support and the leg support can be cleaned with ordinary cleaning products that do not have polishing or etching properties. It is not recommended to use solvents or chlorine.

### **Warning!**

- Must only be used as described under the section "Intended use".
- When sitting down or standing up, the leg supports should be turned out to one side.
- Check on a regular basis that all handles and screws are properly tightened.
- If the amputee or leg support is taken off, it is important that it is correctly installed again.
- The amputee or leg support must only be used for the leg that it was installed for.
- If the Combi chair has an amputee or a leg support installed the user should not lean over forwards as the chair could tip.

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***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, Poland, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various power and manual wheelchairs, scooters, seating systems, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise- and recline chairs, and bathing and toileting products.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com)

Handicare AB  
Veddestav. 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)

Internet: [www.handicare.com](http://www.handicare.com)