EasyBelt Wipeable

SystemRoMedic™

CE

Manual - English

Manual - Svenska

Manual - Norsk

Manual - Danske

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Manual - Español


REF

6113

6114

6115

6116

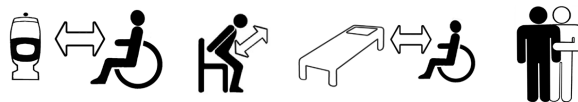SWL: 200 kg/440 lbs

Table of contents

English	3-4
Svenska	5-6
Norsk	7-8
Danske	9-10
Soumi	11-12
Deutsch	13-14
Netherlands	15-16
Français	17-18
Italiano	19-20
Español	21-22
<hr/>	
Product information	23
Accessories	23

EasyBelt, wipeable are used as support in different transfer situations and gait training. The belt is made in a material that is completely sealed, making it easy to wipe clean with disinfectant.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.

Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that velcro and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the manual


Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.


Functional testing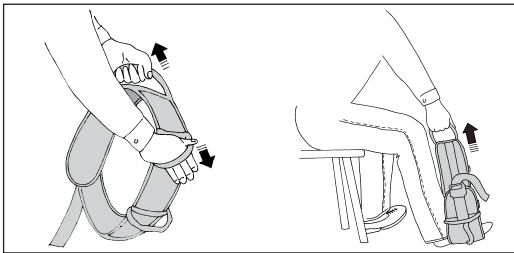

Pull hard on the handles in different directions.

Fasten the belt, place one foot in it, and then pull on one of the handles. Repeat the procedure with all handles.


Placement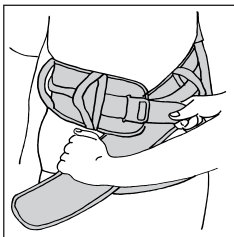

Apply the belt around the patient, below the waist.


Fasten the belt and tighten it.

Removing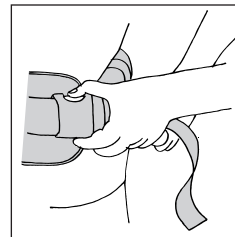

Release the clasp, first loosening any straps, and then remove the belt.


Using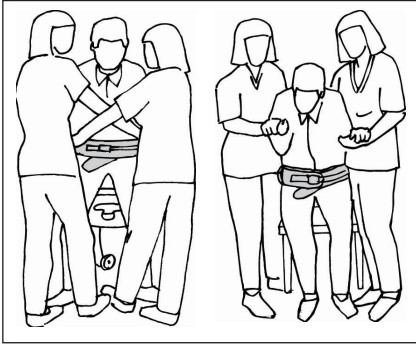

When standing up from sitting to standing the caregiver / caregivers can choose to work from the side or from the front of the patient. One advantage of working in front of the user is to give a feeling of a more secure and safe standing up.


In raising with two helpers, it is advisable to work with cross-grip front of the patient.

It is important to lean the user forward to get a natural pattern of movement, working in towards the body with the belt.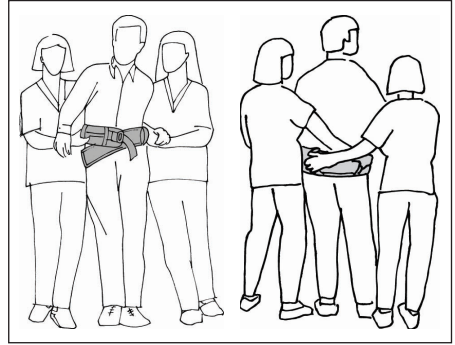

When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the patient by the hand at the front.

Cross-grips at the back provide support and enable a good working stance.

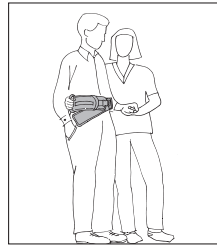Caregivers can work in pairs or alone, depending on the user's needs, since EasyBelt have several vertical and horizontal handles.

Material:

EasyBelt, wipeable: Nylon + TPU

Care of the product:

Read the product label.

Clean with a pressure washer and disinfectant.

Use ethanol (alcohol) or similar disinfectant to clean the cover.

The concentration should not exceed one part ethanol per three parts water.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

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
Accessories

SupportStraps

SupportStraps can be used as a leg harness in combination with EasyBelt.

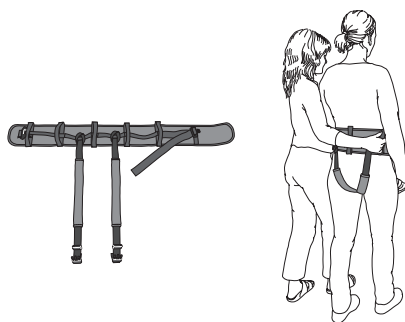SystemRoMedic™

For 25 years we have applied ourselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Our mission, to help people, has always been, and will continue to be, the driving force of innovation. We love easy transfers.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com

handicare

Handicare AB
Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se

Internet: www.handicare.com