Amputee support/Leg support Accessories for Combi chairs

Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 201060004 / 201061004

MAX 100 kg/220 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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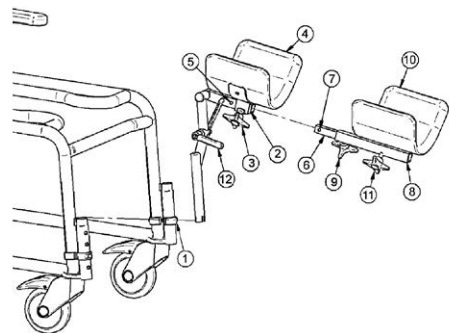
Linido

Instructions for use and user

EN - Please read the instructions before using the product. These instructions apply to the amputee support and leg support for the Combi chair. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.

Intended use

This product can be used with all Combi chairs. The amputee support can be attached to the Combi chair to give support to a stump after an amputation. If it is necessary to support an entire leg, the leg support is mounted to the amputee support, to give support and relief for, as an example, a plaster cast.

Installation

The amputee support can be installed on both the right and left sides of the chair. The amputee support is mounted on the holders for the foot support at the desired height. Make sure that the clips (1) are fully inserted.

When moving to the opposite leg.

Loosen thumbscrews (3) and remove the PUR support (4). Loosen handle (12), and swing the upper part (5) 180° over backwards. Tighten handle (12). Move the amputee support over to the opposite side by lifting up in the amputee support, and putting it on the opposite side, so the slots go down over the clip in the holder. Put on the PUR support, and tighten thumbscrews (3) in the desired position.

To install the leg support.

Remove the cap (2) for the amputee support. Loosen the thumbscrews (3), and pull the PVR support (4) out over the square tubing so that the hole (5) is visible. Put the middle tube (6) into the amputee support. Make sure that the pin (7) in the middle tube clicks correctly into the hole (5) of the amputee support. Mount the leg support (8) on the middle tube and tighten the thumbscrews (9) at the desired length. The leg support (10) is fastened in the desired position with help of the thumbscrews (11).

Adjustment and use

Lift the amputee support in its holder, and then swing out to one side. To adjust the angle of the amputee support, loosen the handle (12). Adjust the amputee support to the desired position, and tighten the handle again. The leg support can be placed in the desired position by loosening the thumbscrews and moving the tubes. After positioning make sure the thumbscrews are firmly tightened.

Maintenance

The amputee support and the leg support can be cleaned with ordinary cleaning products that do not have polishing or etching properties. It is not recommended to use solvents or chlorine.

Warnings

- When sitting down or standing up, the leg supports should be turned out to one side.
- Check on a regular basis that all handles and screws are properly tightened.
- If the amputee or leg support is taken off, it is important that it is correctly installed again.
- The amputee or leg support must only be used for the leg that it was installed for.
- If the Combi chair has an amputee or a leg support installed the user should not lean over forwards, as the chair could tip over.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.