Armrest lock

Accessories for Combi chairs

Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 201030310

MAX 100 kg/220 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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EN - Please read the instructions before using the product. These instructions apply to the armrest lock for the Combi chair. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.

Intended use

The armrest lock can be mounted as an accessory onto all Combi chairs, and insures that the armrests cannot inadvertently fold up, for example, in the Combi Manual Tilt Chair, when the chairs are tipped over backwards. The armrest lock can be used to advantage together with the front cross bar Combi chair.

Installation

Loosen and remove the two front screws on the armrest. Mount the armrest lock on the armrest using the supplied longer screws (1) and the nylon rings (2). Place the two rings on both sides between the U-profile of the armrest and the armrest lock. The bend in the armrest lock must point backwards, as shown in the sketch.

Adjustment and use

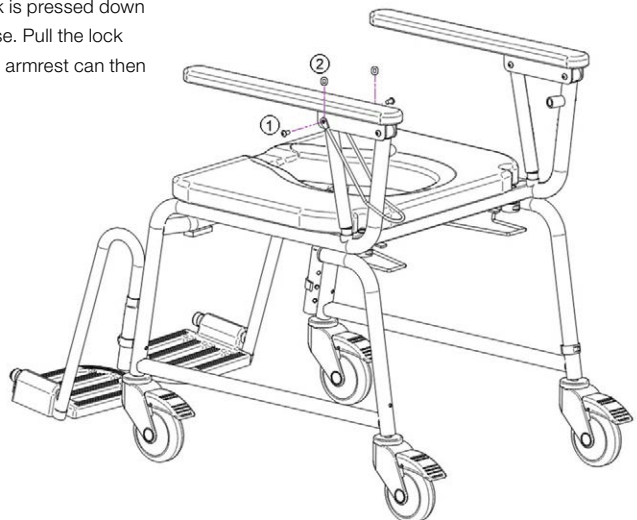
Press down the armrest lock vertically under the armrest to insure against inadvertently swinging up of the armrest. Be sure that the armrest lock is pressed down completely to the frame for optimal use. Pull the lock backwards, to lift up the armrest. The armrest can then be moved freely.

Maintenance

The armrest lock can be cleaned with ordinary cleaning products without polishing or etching properties. It is not recommended to use solvents or chlorine. Dry off with a clean cloth afterwards.

Warnings

- Should always be installed when the users are agitated or overly active.
- Check regularly that the screws are tightened.
- Make certain that the armrest lock is pressed completely down to the frame for optimal use.
- Be aware of the danger for being pinched when the armrest is being folded up or down.

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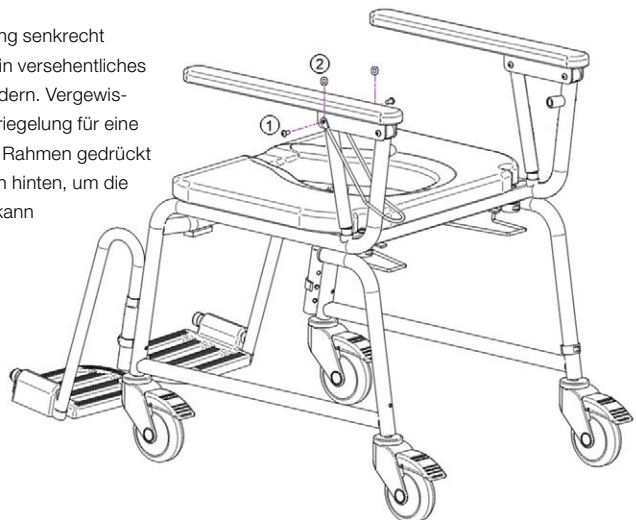
Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.