AutoSlide

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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460 x 460 mm/ 18.1 x 18.1 "

REF
8100

SWL: 160 kg/ 350 lbs

Table of contents

English	3-4
Svenska	5-6
Norsk	7-8
Dansk	9-10
Suomi	11-12
Deutsch	13-14
Nederlands	15-16
Français	17-18
Italiano	19-20
Español	21-22

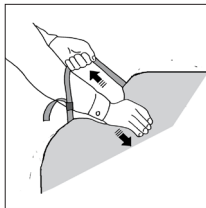
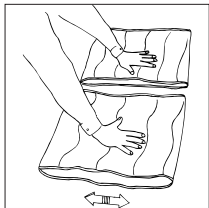
AutoSlide is used to aid transfers into and out of car seats. It consists of two sections with surfaces that slide against each other. AutoSlide has a tension strap for fastening at the back. There are handles at the sides for caregivers.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

Always read the user manual

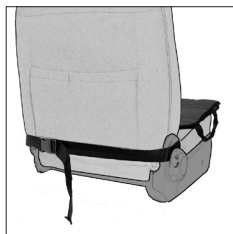
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.


The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement of AutoSlide


Place AutoSlide with the smaller section's rough surface facing the car seat cushion. Fasten the tension strap around the back support and tighten securely.

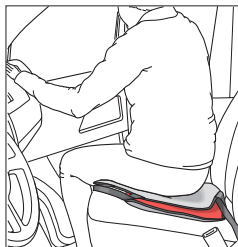Using AutoSlide

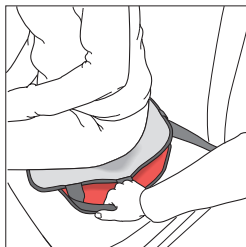The user can sit on the seat and then easily lift/turn the legs into the car while gliding all the way back onto the seat.

A caregiver can assist from the driver's side of the car by pulling on the handles on the sides of the AutoSlide (from the driver's side).

The user should always wear a seatbelt.

Material

Polyester

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Vida útil estimada

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SystemRoMedic™

Simple solutions for great results

SystemRoMedic™ is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group

Advancing Movement & Health

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com