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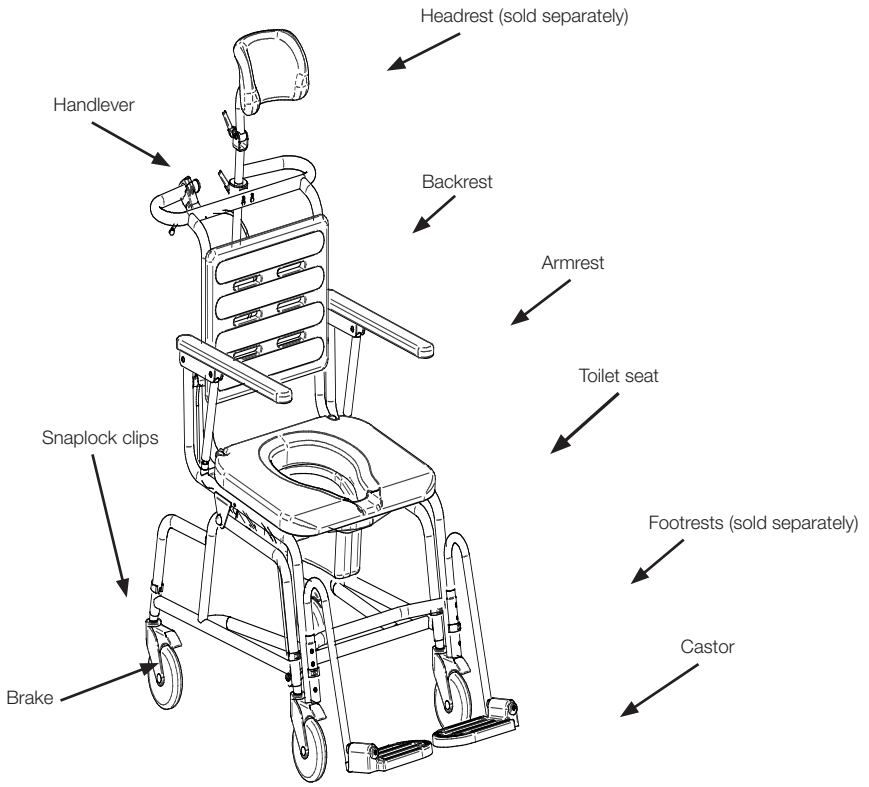
EN - Instructions for use

These instructions apply to product 201941314 Combi manual tilt chair.

Overview and Use:

The Combi manual tilt chair is a very versatile chair, suited for dependent individuals who require assistance with personal hygiene.

Manufactured entirely of rust-proof materials and components, the Combi manual tilt chair includes 4 locking, plastic castors. The chair can be used as a commode, a shower chair and a transport chair for indoor use. The Combi manual tilt chair features a backwards tilt of up to 25°, which facilitates perineal care and also transport of users who are unable to sit comfortably in an upright position. The 5° forward tilt is especially helpful during transfer to and from the chair, whether using manual methods or a mechanical lift.

The Combi manual tilt chair features fold up armrests, useful for both caregiver and user during assisted transfers. The polyurethane (PUR) seat, backrest, and armrests provide comfortable, anti-slip padding for the user.

Assembly

Frame Assembly - fig 1:

Remove the snaplocks and mount the backrest into the holes in the frame. Replace the snaplocks, ensuring they are inserted correctly into the steel tube with both pins correctly placed in the holes. It is possible, although not necessary, to detach the backrest when the Combi chair is used over a toilet.

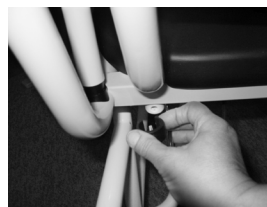
Fig 1

Assembling the castors - fig 2:

FRONT castors: Insert the inner-tube (castor) into the chair leg, aligning the larger hole of the castor tube with the hole in the leg tube. Insert the snaplock at the appropriate height with the black part facing to the front.

REAR castors: Insert the inner-tube (castor) into the chair leg, aligning the larger hole of the castor tube with the hole in the leg tube. Insert the snaplock at the appropriate height with the black part facing to the back.

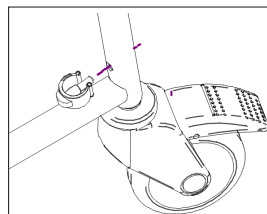
Fig 2

To ensure proper alignment of the castors, the snaplock pin will not fit into the tubes if the holes are mis-aligned.

IMPORTANT: Check that all four legs are the same height and that all of the snaplocks are inserted correctly into the frame before use.

Bedpan / Bucket rails:

The bed pan / bucket rails can be used in combination with a keyhole (fig.3) or round shaped bucket (fig.4).

Figure 3

Figure 4

Figure 5

To assemble - see figure 6:

The two end pieces (B, on the drawing) rest on the rear of the seat frame.

For keyhole shape bucket, insert the two front rods (A) into the inner holes (C).

For all other bucket shapes use the outer holes (D).

If no bucket or bed pan is being used, the rails may be detached so the chair fits easily over a toilet.

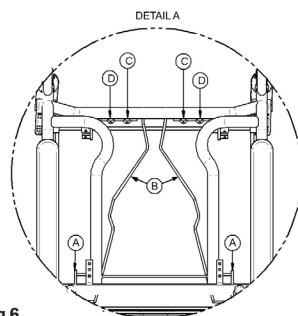
Fig 6

Individual Adjustment - see figure 7

Attachment of castors & frame height adjustment:

The pin on the snaplock clip narrows in width from the front towards the end. The hole in the tubing is bigger on one side and narrower on the other side. This safety feature means that the snaplock clip can only be inserted in one way into the tubing, ensuring the castors to be mounted correctly.

Front castor: insert snaplock clip into tube hole from rear of tubing so pin points frontwards.

Rear castor: insert snaplock clip into tube hole from front of tubing so pin points backwards.

Height Adjustment:

By re-positioning the snaplocks on the chair legs, the Combi chair may be adjusted to 6 different heights.

To adjust the height of the chair:

1. Remove snaplock.
2. Select one of the 6 positions.
3. Reinsert snaplock.

Repeat for all 4 legs.

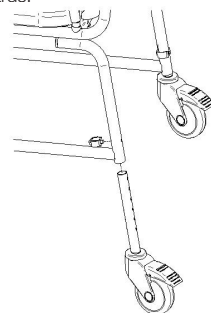
Fig 7

IMPORTANT: Check that all four legs are the same height and that all of the snaplocks are inserted correctly into the frame before use.

Tilting function:

The Combi manual tilt chair can be tilted 5° forward and 25 ° backwards.

IMPORTANT: Before using the tilting function remember to lock all four castors.

The tilting function is operated by means of a gas piston activated via a hand lever on the pushbar. While the backwards tilt may be operated with the chair empty, the forward tilt requires a counterweight to facilitate the movement for the caregiver.

Stand behind the chair and operate the hand lever. A small amount of downward force may be required to tilt the chair back depending on the weight of the user.

Maintenance / Cleaning

To ensure longevity of the chair and safety for the user, the following should be included in the daily inspection:

- Inspect and clean the castors of dust and dirt.
- Inspect and clean the castors brakes of dust and dirt.
- Check that all screws and nuts are properly tightened.
- Check that the snaplock clips are securely fastened.
- Regularly check that the cable to the gas piston has the correct tension.

After each use, or according to facility protocol, the Combi manual tilt chair may be cleaned/disinfected with non-abrasive, non-corrosive solutions. The PUR seat and backrest can be detached for cleaning, as can the bucket / bedpan rails (NOTE: the rear section of the bucket / bedpan holder must be lifted upwards before removing the two rods from the holes). The entire Combi Manual Tilt chair may also be washed / autoclaved at 85 degrees Celsius for 3 minutes. Avoid harsh solvents and chlorine. If alcohol is used to clean the black polyurethane, discolouration may occur. Ensure all parts are re-attached correctly prior to use.

Warning:

- Max. load Combi manual tilt: 150 kg / 330 lb.
- The tilt mechanism should ONLY be operated by the caregiver.
- Check the condition of the snaplocks frequently, and regularly check that the snaplocks are inserted correctly into the frame.
- Lock castors when chair is stationary and also during transfers to and from the chair.
- The Combi manual tilt chair is not designed to be used as seating for extended-periods.
- The Combi manual tilt chair should be pulled backwards over thresholds.
- For optimal caregiver ergonomics, the push handles should be at elbow height.
- Always use the Combi manual tilt chair with armrests in place (folded down).
- Armrests (and footrests, if present) should only be folded up during transfer to and from the chair.
- If the castors are detached, it is important to mount them again correctly (see "Assembling the Castors" above). Ensure that all 4 castors are adjusted to the same height.
- Keep fingers away from the armrest joints to avoid pinching during raising and lowering.
- Use the chair only on a flat, even surface.
- Insert or remove the bed pan/bucket when the chair is not in use.
- Do not store the chair in direct sunlight as the black surface may become hot and uncomfortable for the user.
- To avoid injury, do not insert fingers or toes into the drainage holes in the backrest or footrests.
- Regularly check that the cable to the gas piston has the correct tension.

Disposal:

The plastic parts should be sorted as "incinerated" and stainless steel parts as "metal".


Technical Specifications

Total height:	37-46" (950-1180 mm)
Total length excl. footrests:	34.25" (870 mm)
Total length incl. footrests:	43" (1090 mm)
Total width:	21,2" (535 mm)
Backrest:	16 x 12" (410 x 310 mm)
Seat height interval:	20.2"-23.6" (520-620 mm), adjustable
Seat width:	20" (510 mm)
Seat depth	17.1" (435 mm)
Seat hole width:	8.2" (210 mm)
Seat hole depth	12.2" (310 mm)
Tilting angle:	25° backwards, 5° forwards

Armrest, length	19" (480 mm)
Armrest, width	2" (50 mm)
Width between armrests:	17" (430 mm)
Castors	Ø 5" (125 mm), incl. brakes
Max. user weight:	330 lbs (150 kg)
Total weight:	24 kg / 53 lbs

Materials:

Frame:	Stainless steel
Surface coating frame:	Powder coating
Seat, Backrest, Armrests:	Polyurethane (PUR)
Snaplock Clips:	Chrome pin/Polypropylene

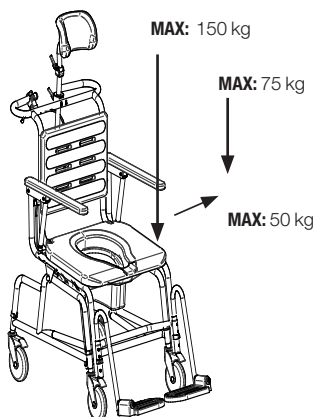List of configurations and accessories

Chairs

201941314 Combi manual tilt

To always be ordered with:

200810004 Footrests 1 pair. For all Combi chair models and versions. Recommended for use with all models and versions of Combi chairs.

200360004 Headrest For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.

200384004 Heel and calf strap For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.

200382004 Hip belt For all Combi chair models and versions.

Additional Accessories:

201131314 Soft backrest For Combi tilt, manual and Combi tilt, powered.

201040314 Front cross bar For all Combi chair models and versions with standard width. Recommended for use with the Armrest lock accessory.

201030310 Arm rest lock 1 pair. For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.

201070314 Lateral trunk support, right For all Combi chair models and versions.

201072314 Lateral trunk support, left For all Combi chair models and versions.

201060004 Amputee support For all Combi chair models and versions.

201061004 Leg support For Amputee support and all Combi chair models and versions.

853100604 Transport seat cover For all Combi chair models and versions.

853100034 Bucket with lid Keyhole shape, white plastic. For all Combi chair models and versions.

853100033 Bucket with lid Round, grey plastic. For all Combi chair models and versions.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health[®]

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com