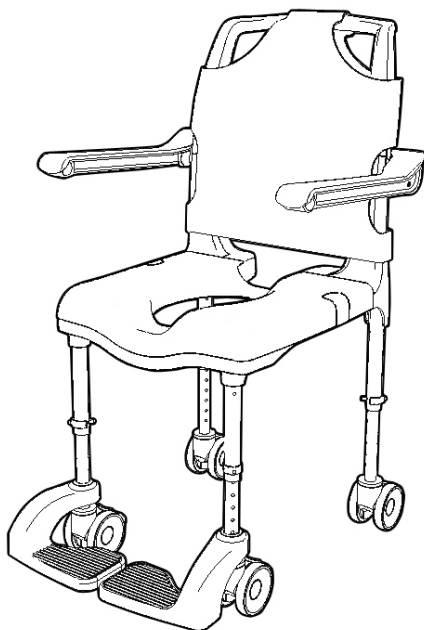
Coral

commode/shower chair, mobile

Manual - English
Manual - Svenska
Manual - Norsk
Manual - Danske
Käyttöohje - Suomi
Gebrauchsanweisung - Deutsch
Handleiding - Nederlands
Manual - Français
Manual - Italiano
Manual - Español

Art.no.:

3002020050

SWL, frame: 150 kg/330 lbs

SWL, arm supports: 150 kg/330 lbs (75 kg/165 lbs each)

Table of contents

Visual inspection/before use	3
Read the manual	4
<hr/>	
English	5-12
Svenska	13-20
Norsk	21-28
Danske	29-36
Soumi	37-44
Deutsch	45-52
Nederlands	53-60
Français	61-68
Italiano	69-76
Español	77-84
<hr/>	
Dimensions	86
Accessories	87

**EN Visual inspection**

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

Table of contents

Using the product	6-11
- Important information	7
- Assembly	8-9
- Disassembly	10-11
With Coral, you are the designer!	12

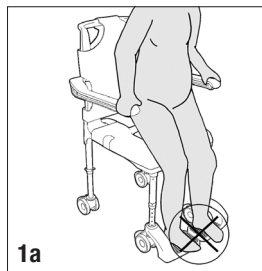Using the product


1a

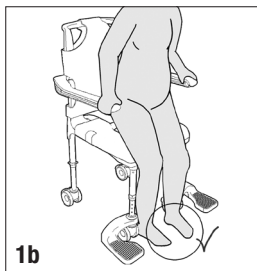
1b

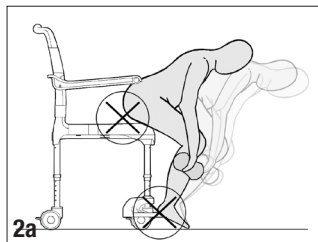- 1a. To prevent the chair from tilting or tipping over do not stand on the footrest!
- 1b. Swing the footrests outwards before standing up (See drawing 6).


2a

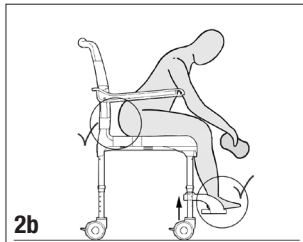
2b

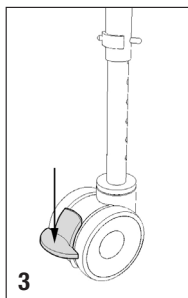- 2a, 2b. Use the full depth of the seat. Make sure that the feet are properly supported before leaning forward!


3

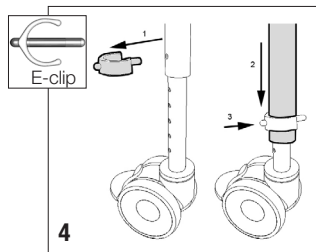3. Push down the brake on each castor to prevent the chair from moving.


4

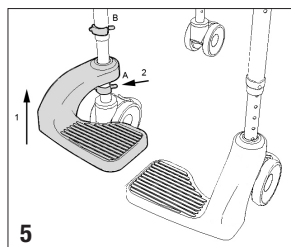4. Adjust the height of the chair by repositioning the E-clips without a user in the chair.
1. Remove the E-clips.
 2. Lower or lift the legs to desired height.
 3. Snap the E-clips in place.
 4. Assure that all legs have the same height.

5

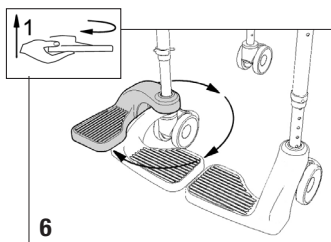5. Adjust the height of the footrest by repositioning the E-clips.
1. Remove the E-clips.
 2. Lower or lift the footrests to the desired height.
 3. Snap the E-clips in place.

6

6. Footrests can be swung outwards. To rotate the footrest first lift the footrest off its E-clip (See figure on footrest).

Important information

Purpose

Coral mobile chair is a very versatile chair. It can be used as both a commode, a shower chair, and a chair for indoor transportation. Coral mobile chair has fold up armrests, a useful feature for both caregiver and user in situations of assistance and transfers.

The mobile chair is particularly suitable for immobile persons, who are not able to walk to the toilet and bathroom by themselves and therefore need assistance with toileting and personal hygiene. The chair is intended for short-term use only.

Maintenance / Cleaning

To ensure optimal operation, the following should be regularly reviewed:

- Inspect and clean the castors of dust and dirt.
- Inspect and clean the castor brakes of dust and dirt.
- Check if the E-clips (see drawing 4 and 5, page 6) are securely fastened.

The chair is to be wiped clean after use. Clean Coral mobile chair with non-abrasive, non-corrosive domestic cleaning agents. Solvents and chlorine should not be applied. The chair surfaces can be disinfected with all disinfectants that are approved by Health Authorities.

The toilet rails can be detached for cleaning purposes.

Make sure to reattach all removed parts correctly.

Reuse

Follow the above procedure. Defective or worn parts must be replaced.

Caution

Max. load Coral mobile chairs: 150 kg / 330 lbs.

Max. load per armrest: 75 kg / 165 lbs.

Check the condition of the E-clips frequently.

Regularly check that the E-clips are inserted correctly into the tubing.

Regularly check that all parts are in place and secured.

It is advisable to lock the castors outwards when Coral mobile chair is not being used for transportation.

If the castors are detached, it is important to mount them again correctly. Make sure all 4 legs have the same height.

Avoid having your fingers/hands between the armrest rear and the backrest when the armrest are to be folded down, as it might cause a risk of squeezing fingers/hands.

The chair is only to be used on an even surface.

The bed pan/bucket is only to be inserted or removed when the user is not in the chair.

Do not place the chair in direct sunlight, as it might cause a hot surface on the seat or at the backrest.

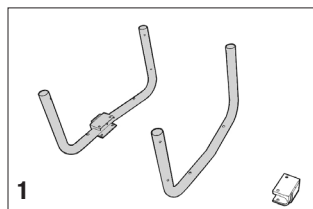
Disposal

The plastic parts should be sorted as incinerated and stainless steel parts as metal.


Assembly

Coral commode/shower chair, mobile


3002020050

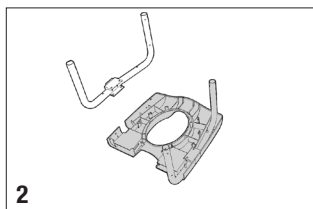
1. Slide the cover on the straight part of the chair leg (2x).

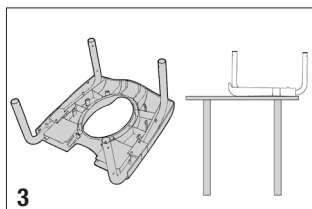
2. Position the chair leg with cover (2x) on the toilet seat. Make sure that all pins of the toilet seat fully protrude the chair leg and the cover.

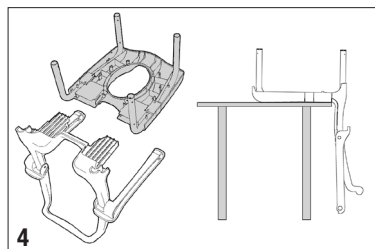
3. Make sure that the chair leg and cover are well fitted in the toilet seat. Lay this part on a table for easier assembly of the back support.

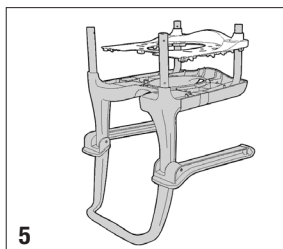
4. Slide the frame of the backrest in the rear pockets of the toilet seat.

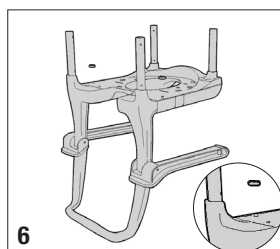
5. Slide the base plate over the front chair legs. Make sure that the rear pins of the toilet seat fully protrude the base plate. Push the base plate onto the toilet seat until all clickers snap into place.

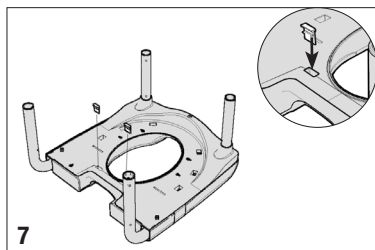
6. Mount the safety lock (2x) on the protruding pin. Make sure that both safety locks are screwed on tightly.

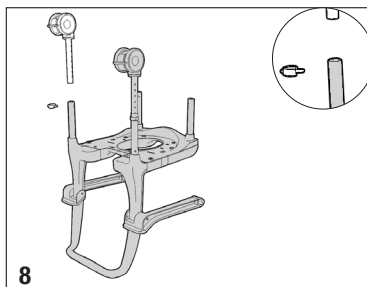
7. Mount the safety key (2x) between the clickers at the back of the stool. Push downwards until it snaps in place.

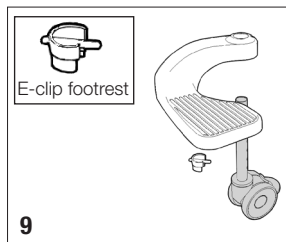
8. Slide the castor tube into the rear leg (2x). Adjust to the correct height and click the E-clip through the chair leg and castor tube. Make sure the castors are mounted at the same height.

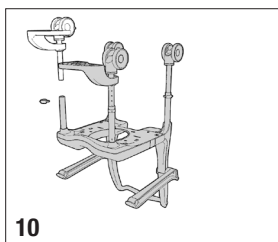
9. Click the E-clip for the footrest at the desired height in the castor tube. Make sure that the pin turned with the right side up, see picture. Slide the footrest on the castor tube and let it rest onto the pin of the E-clip.

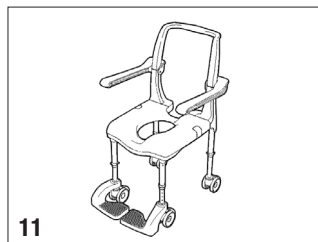
10. Turn the castor with footrest upside down. Slide the castor tube into the front leg. Make sure the open side of the E-clip for the footrest is facing inwards. Click the E-clip through the chair leg and castor tube at the same height as the rear castors.

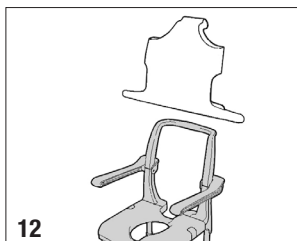
11. Turn the chair upright. Make sure that all parts are in place and secured.

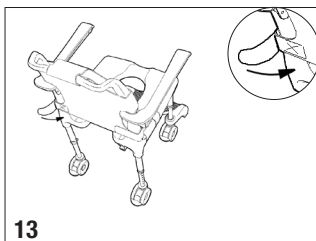
12. Place the backrest cover over the frame of the backrest and pull it firmly into place.

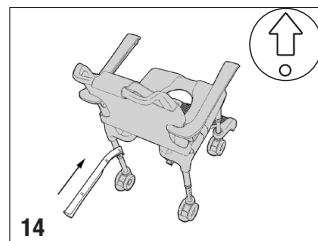13. Tighten the straps of the cover around the frame onto the velcro. To obtain the desired tension adjust the straps.

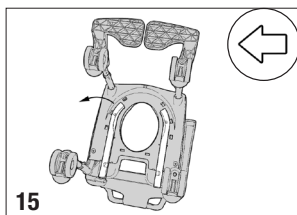14. Place the rear pin of the toilet rail (2x) into the rear keyhole in the base plate. Push forward until it snaps in place.

15. Rotate the toilet rail until the front pins align with the front keyholes. Push outward until it snaps in place.

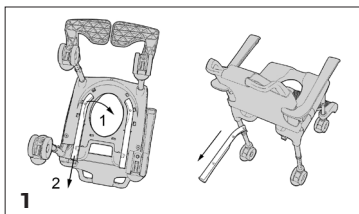
16. Finally, check the whole chair once again to make sure that all parts are in place and secured.


Disassembly

Coral commode/shower chair, mobile


3002020050

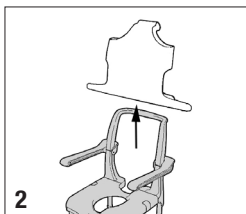
1. Remove the toilet rail (2x) by:
1. Pulling the rail inward until the upper part unlocks.
 2. Pushing the rail backward (direction: back of the chair) until the rear pin unlocks.

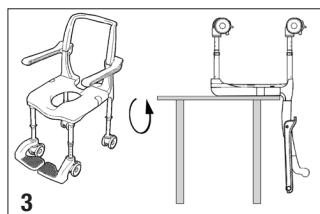
2. Detach the straps and pull the fabric of the backrest off the frame.

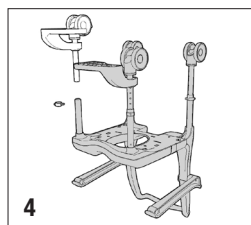
3. Turn the chair upside down and lay the chair with the seat on top of a table.

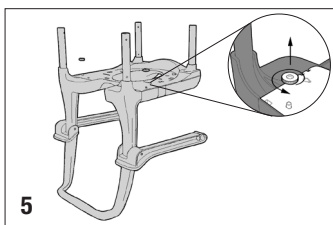
4. Remove the clips from the chair legs and remove the castors (4x) and the footrests (2x).

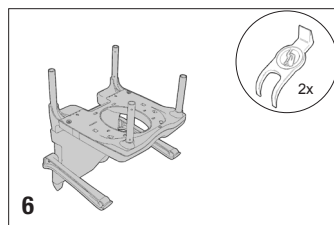
5. Remove the safety lock from the pin by unscrewing (2x).

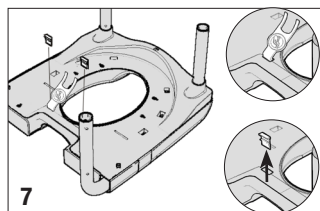6. Use the two tools for disassembly to unlock the base plate (see pictures 7 to 11). Only use the included tools.

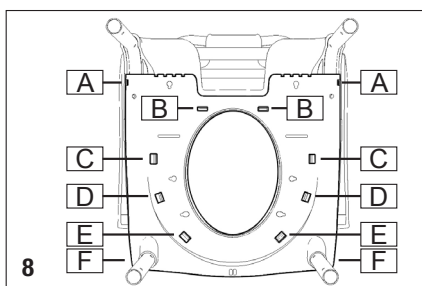7. Remove the safety keys at the backside of the base plate. Place the wedge end in the open space of the pocket. Remove the safety key by rotating the wedge upwards (2x).


8. Unlock the clickers in alphabetical order (see pictures 8 to 11).


9. Pos A (2x):

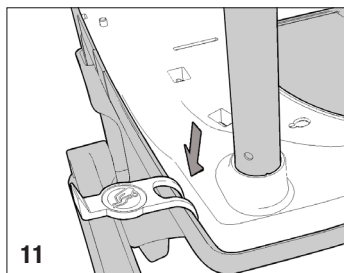- Fig 9A: Position the fork of the tool around the pocket. Make sure the tip of the fork is between the seat and the base plate.
- Fig 9B: Gently push/slide the tool downward between both parts until the clicker unlocks.


11. Pos F (2x):

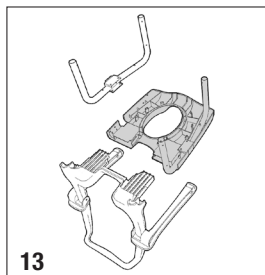- Position the tool with the tip of the fork between the seat and the base plate (at approximately 5cm/2" from the front corner of the seat).
- Press the tool gently between the seat and the base plate to unlock the clicker.


13. Pull the frame of the backrest and the chair legs off the seat.

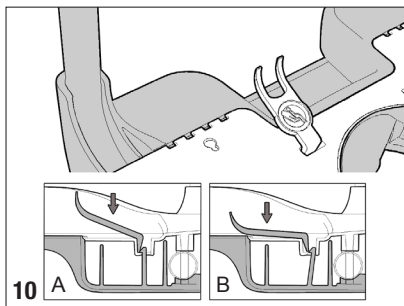The armrests can not be detached and replaced.

10. Pos B-E (2x);

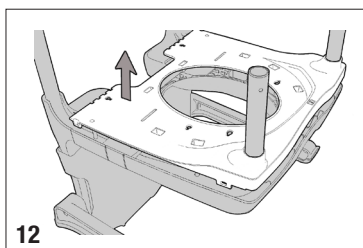- Fig 10A: Place the wedge end in the pocket by sliding along the back of the pocket. Tip of the wedge end touches the tip of the clicker. Tool and base plate are angled.
- Fig 10B: Press down the tool toward the base plate, while lifting the base plate, until the clicker unlocks.

12. Pull the base plate upward over the chair legs. Keep the base plate horizontal while pulling.

With Coral, you are the designer!

With one stable and height-adjustable frame combined with optional extras and accessories, Handicare's Coral concept offers numerous possibilities to create individually adapted showering and toileting solutions. All combinations offer safe and flexible user-friendliness, high-quality stainless materials and a contemporary design.

When life changes, so does your Coral

The mobile Coral commode/shower chair can be equipped with a bedpan or bucket and a soft toilet seat. The toilet seat can also be replaced with a shower seat. For even greater comfort, or to accommodate for changing needs and requirements, the chair can also be fitted with a calf support and a safety belt. Finally, with the 24" self-propelling wheel adapter set, the mobile Coral commode/shower chair can be conveniently transformed into a self-propelled version providing the user with increased independence in showering and toileting situations.

Recycling – an important part of the Coral concept

Naturally, an existing Coral configuration can also be adapted to a "smaller" solution or to fit another user. All dismantled parts and accessories can be reconditioned and re-used in new configurations for other users.

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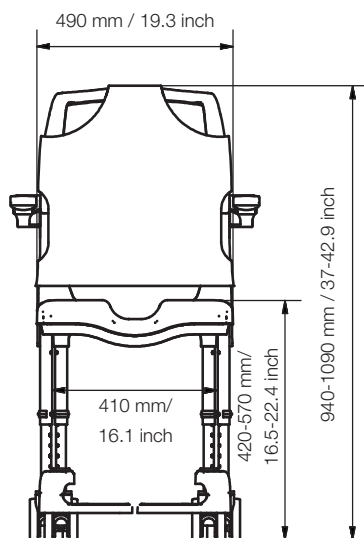
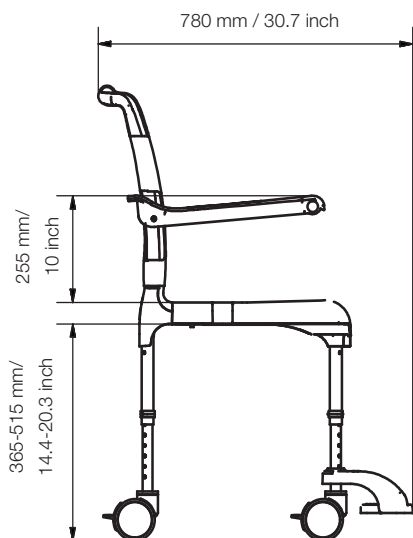
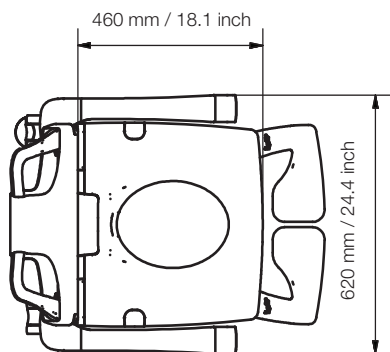
Accessories - for increased comfort and/or usability

3002920802	Bedpan, with lid, stainless steel
3002920803	Bucket, with lid, stainless steel
3002920400	Soft toilet seat, PUR, oval opening
3002920401	Soft toilet seat, PUR, keyhole opening
3002920500	Transportation seat, PUR
3002921100	Armrest lock, prevents folding function, 1 pair
3002920900	Front support, for Backrest frame with folding armrests 3002820200
3002820900*	Castors, twin castors, Ø 125 mm, incl legs, 4 pcs
3002810300	Folding footrests, plastic, with tubes, incl fixings, 1 set
3002921400	Headrest, for Backrest frame with folding armrests 3002820200
3002920200	Calf support, textile, with Velcro
3002920100	Amputee support left, for left underleg, with adjustable knee angle (a piece), incl fixings
3002920101	Amputee support right, for right underleg, with adjustable knee angle (a piece), incl fixings
3002921300	Safety belt, for hip and chest
3002800200	Shower seat, plastic, incl covers

Accessories - for creating more advanced configurations

3002921500*	24" self-propelling wheels adapter set, complete adapter set
3002921700	Anti tip set, for self-propelling chairs

Accessories for creating simpler configurations

3002820700	Legs with end caps, height adjustable, incl height adjustment clips, 4 pcs
3002800300	Back support, plastic
3002800500	Arm supports, plastic, 1 pair
3002920402	Soft toilet seat, PUR, oval opening, with cut-outs for arm supports
3002920403	Soft toilet seat, PUR, keyhole opening, with cut-outs for arm supports
3002920501	Transportation seat, PUR, with cut-outs for arm supports
3002820102	Cover for back of seat, if backrest/back support is removed, 1 pce

*Castors and self-propelling wheels can only be added to a Coral stool/chair fitted with a Backrest frame with folding armrests (3002820200).

For more information, please visit www.handicare.com or contact your local distributor.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve convenience for those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, Poland, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various power and manual wheelchairs, scooters, seating systems, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise- and recline chairs, and bathing and toileting products.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com

Handicare AB
Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se

Internet: www.handicare.com