EasyRoll Sliding sheets

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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REF

411051P	4110521	41803PS	411203PS	4111B	45901S	51901FAH	519031R
4106	4110531	41901PS	41902PSB	4112B	45902S	51902FAH	519031RAH
4404IB	4112011	41902PS	4115031	4113B	45903S	51903FAH	5110531RAH
418011	4112021	41904TV	419031D	4111LB	451051S	511051FAH	5112031RAH
418021	4112031	41903PS	418031R	4112LB	451052S	511052FAH	
418031	41701PS	411051PS	419031R	4113LB	451053S	511053FAH	
419011	41702PS	411052PS	4110531R	42901PS	451201S	511201FAH	
419021	41703PS	411053PS		42902PS	451202S	511202FAH	
419031	41801PS	411201PS			451203S	511203FAH	
4110511	41802PS	411202PS					

SWL: 200 kg/440 lbs

Table of contents

Product information	3
English	7
Svenska	9
Norsk	11
Dansk	13
Suomi	15
Deutsch	17
Nederlands	19
Français	21
Italiano	23
Español	25

**Product information/ Produktinformation/ Produktinformasjon/ Produktinformation/
Tuotetiedot/ Produktinformation/ Productinformatie/ Information produit/
Informazioni sul prodotto/ Información del Producto**

Art. no.	Product/Produkt/ Tuote/ Produktname/Product/ Produit/ Prodotto/ Producto	Size/ Storlek/ Størrelse/ Koko/ Größe/ Maat/ Taille/ Dimensione/ Tamaño
EasyRoll - sliding sheet/ glidlakan/ glidelaken/ glidelagen/ liukuvan telttakankaan/ piirtää arkki/ gleitlaken/ glijlaken/ feuille de glissement/ telone scorrevole/ sábana de deslizamiento		
(Can be combined with EasyRoll 2way drawsheet or EasyRoll 4way sliding drawsheet)		
411051P	EasyRoll Sliding sheet	100cm long (for 105cm bed)
4106	EasyRoll Sliding sheet	190cm long (for 90cm bed)
4404IB	EasyRoll Sliding sheet	100cm long (for 90cm bed)
418011	EasyRoll Sliding sheet	100cm long (for 80cm bed)
418021	EasyRoll Sliding sheet	140cm long (for 80cm bed)
418031	EasyRoll Sliding sheet	200cm long (for 80cm bed)
419011	EasyRoll Sliding sheet	100cm long (for 90cm bed)
419021	EasyRoll Sliding sheet	140cm long (for 90cm bed)
419031	EasyRoll Sliding sheet	200cm long (for 90cm bed)
4110511	EasyRoll Sliding sheet	100cm long (for 105cm bed)
4110521	EasyRoll Sliding sheet	140cm long (for 105cm bed)
4110531	EasyRoll Sliding sheet	200cm long (for 105cm bed)
4112011	EasyRoll Sliding sheet	100cm long (for 120cm bed)
4112021	EasyRoll Sliding sheet	140cm long (for 120cm bed)
4112031	EasyRoll Sliding sheet	200cm long (for 120cm bed)
41701PS	EasyRoll Sliding sheet	100cm long (for 70cm bed)
41702PS	EasyRoll Sliding sheet	140cm long (for 70cm bed)
41703PS	EasyRoll Sliding sheet	200cm long (for 70cm bed)
41801PS	EasyRoll Sliding sheet	100cm long (for 80cm bed)
41802PS	EasyRoll Sliding sheet	140cm long (for 80cm bed)
41803PS	EasyRoll Sliding sheet	200cm long (for 80cm bed)
41901PS	EasyRoll Sliding sheet	100cm long (for 90cm bed)
41902PS	EasyRoll Sliding sheet	140cm long (for 90cm bed)
41904TV	EasyRoll Sliding sheet	160cm long (for 90cm bed)
41903PS	EasyRoll Sliding sheet	200cm long (for 90cm bed)
411051PS	EasyRoll Sliding sheet	100cm long (for 105cm bed)
411052PS	EasyRoll Sliding sheet	140cm long (for 105cm bed)
411053PS	EasyRoll Sliding sheet	200cm long (for 105cm bed)
411201PS	EasyRoll Sliding sheet	100cm long (for 120cm bed)
411202PS	EasyRoll Sliding sheet	140cm long (for 120cm bed)
411203PS	EasyRoll Sliding sheet	200cm long (for 120cm bed)
41902PSB	EasyRoll Sliding sheet	140x200cm (for 90cm bed)
419031D	EasyRoll Sliding sheet	200cm long (for 90cm bed)
418031R	EasyRoll Sliding sheet	200cm long (for 80cm bed)
419031R	EasyRoll Sliding sheet	200cm long (for 90cm bed)
4110531R	EasyRoll Sliding sheet	200cm long (for 105cm bed)
519031R	EasyRoll Sliding sheet	200cm long (for 90cm bed)
45901S	EasyRoll Sliding sheet Anticubitus	100cm long (for 90cm bed)
45902S	EasyRoll Sliding sheet Anticubitus	140cm long (for 90cm bed)
45903S	EasyRoll Sliding sheet Anticubitus	200cm long (for 90cm bed)
451051S	EasyRoll Sliding sheet Anticubitus	100cm long (for 105cm bed)
451052S	EasyRoll Sliding sheet Anticubitus	140cm long (for 105cm bed)

451053S	EasyRoll Sliding sheet Anticubitus	200cm long (for 105cm bed)
451201S	EasyRoll Sliding sheet Anticubitus	100cm long (for 120cm bed)
451202S	EasyRoll Sliding sheet Anticubitus	140cm long (for 120cm bed)
451203S	EasyRoll Sliding sheet Anticubitus	200cm long (for 120cm bed)
51901FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 90cm bed)
51902FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 90cm bed)
51903FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 90cm bed)
511051FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 105cm bed)
511052FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 105cm bed)
511053FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 105cm bed)
511201FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 120cm bed)
511202FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 120cm bed)
511203FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 120cm bed)
519031RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 90cm bed)
5110531RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 105cm bed)
5112031RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 120cm bed)
42901PS	EasyRoll Sliding sheet w Barrier	100cm long (for 90cm bed)
42902PS	EasyRoll Sliding sheet w Barrier	140cm long (for 90cm bed)

EasyRoll 2Way - drawsheet/ draglakan/ stikklagenet/ træktagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera

(To be combined with EasyRoll sliding sheet)

2101	EasyRoll 2way Drawsheet	100x180cm
2101S	EasyRoll 2way Drawsheet	140x200cm

EasyRoll 4Way - drawsheet/ draglakan/ stikklagenet/ træktagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera

(To be combined with EasyRoll sliding sheet)

4011S1	EasyRoll 4way Sliding Drawsheet	150cm
4011S	EasyRoll 4way Sliding Drawsheet	140x200cm
4011L	EasyRoll 4way Sliding Drawsheet	200x200cm
4011SSL	EasyRoll 4way Sliding Drawsheet	150x200cm
4111T	EasyRoll 4way Sliding Drawsheet	140cm long (for 90cm bed)
4112T	EasyRoll 4way Sliding Drawsheet	140cm long (for 105cm bed)
4113T	EasyRoll 4way Sliding Drawsheet	140cm long (for 120cm bed)
4111LT	EasyRoll 4way Sliding Drawsheet	200cm long (for 90cm bed)
4112LT	EasyRoll 4way Sliding Drawsheet	200cm long (for 105cm bed)
4113LT	EasyRoll 4way Sliding Drawsheet	200cm long (for 120cm bed)
4111B	EasyRoll 4way Sliding sheet	140x200cm
4112B	EasyRoll 4way Sliding sheet	140x215cm
4113B	EasyRoll 4way Sliding sheet	140x230cm
4111LB	EasyRoll 4way Sliding sheet	200x200cm
4112LB	EasyRoll 4way Sliding sheet	200x215cm
4113LB	EasyRoll 4way Sliding sheet	200x230cm

EasyRoll - pillowcase/ örngott/ putetrek/ hovedpudebetræk/ tynniina kahvat/ Kissenbezug/ kussensloop/ Taie d'oreiller/ federa/ unda de almohada

4303	EasyRoll Sliding Pillow case	50x60cm
4304	EasyRoll Pillow case w Barrier	50x60cm

EasyRoll 2way Inco, EasyRoll 4way Inco, EasyRoll Chair pad, EasyRoll Underpad and EasyRoll Mattress Protector

10991	EasyRoll Inco 2way Sliding sheet	185x90cm
10992	EasyRoll Inco 4way Sliding sheet	120cm
10992-5	EasyRoll Inco 4way Sliding sheet	140cm
1298k	EasyRoll Chair pad	45x45cm
1299k	EasyRoll Chair pad	45x45cm
14545GR	EasyRoll Chair pad	45x45cm
16060GR	EasyRoll Chair pad	45x45cm
16090GR	EasyRoll Chair pad	45x45cm
16075V	EasyRoll Chair pad	45x45cm
17590V	EasyRoll Chair pad	45x45cm
80011	EasyRoll Underpad	85x90x180 cm
80011B	EasyRoll Underpad	80x180cm
80011P	EasyRoll Underpad	80x180cm
80011PL	EasyRoll Underpad	90x210cm
80011PI	EasyRoll Underpad	80x175cm
80017	EasyRoll Underpad	85x90 cm
80017P	EasyRoll Underpad	85x90 cm
80018	EasyRoll Underpad	150x90 cm
80018P	EasyRoll Underpad	150x90 cm
80019	EasyRoll Underpad	120x90x180cm
80011PS	EasyRoll Underpad w Barrier	70x190cm
80010P	EasyRoll Underpad w Sliding sheet	80x85x180 cm
90.200J01	EasyRoll Mattress protector	90x200cm
105.200J01	EasyRoll Mattress protector	105x200cm
120.200J01	EasyRoll Mattress protector	120x200cm

Product combinations, Produktkombinationer, Produkt kombinasjon, Produkt kombination, Tuoteyhdistelmä, Produktkombination, Productcombinatie, Combinaison de produit, Combinazione di prodotti, Combinación del producto

Art. no.	Combination	Art. no.	Combination
4111	4111 + 4111T	41401S	41401S + 4011S
4112	4112 + 4112T	41402S	41402S + 4011S
4113	4113 + 4113T	41403S	41403S + 4011S
4111L	4111L + 4111LT	41401LS	41401LS + 4011L
4112L	4112L + 4112LT	41402LS	41402LS + 4011L
4113L	4113L + 4113LT	41403LS	41403LS + 4011L
41402SET	41402SET + 4011S	10990	10990 + 10992
41403SET	41403SET + 4011L	10990-5	10990-5 + 10992-5

EasyRoll Sliding sheets

SystemRoMedic™

English

EasyRoll is a slide sheet that facilitates positioning and turning in bed. It is bedded in the bed like a normal sheet. EasyRoll with elastic spring is bedded in the bed like a normal sheet and is fastened around the mattress with the elastic spring. EasyRoll can be used in combination with drawsheet, EasyRoll2Way, or sliding drawsheet, EasyRoll4Way. EasyRoll is available in several sizes and models.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.


Always read the user manual

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product.


Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.


Placement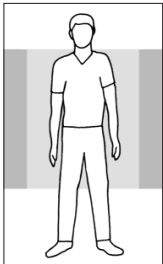

100x180
100x190
100x205
100x220
100x200
100x215
100x230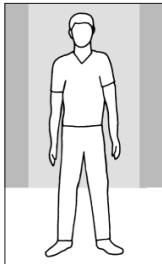

140x180
140x190
140x205
140x220
140x200
140x215
140x230
140x210
150x200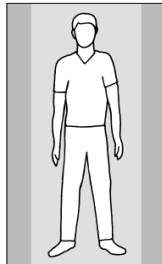


200x180
200x190
200x205
200x220
200x200
200x215
200x230
200x210

Place EasyRoll with the smooth surface in the middle of the bed. It is bedded in the bed like a normal sheet.

Ensure that the mattress cover's rough surfaces or evenly distributed on both sides of the bed. These rough surfaces prevent the user from sliding out of bed.


Using EasyRoll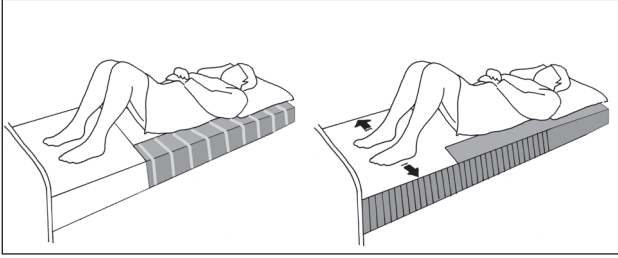

EasyRoll can be bedded in cases where the user is able to turn himself/herself, but finds it easier with less friction.

The smooth surface of the sheet reaches over the foot of the bed, which makes it easier for the user to shift his/her feet when sitting up at the edge of the bed.

Accessories

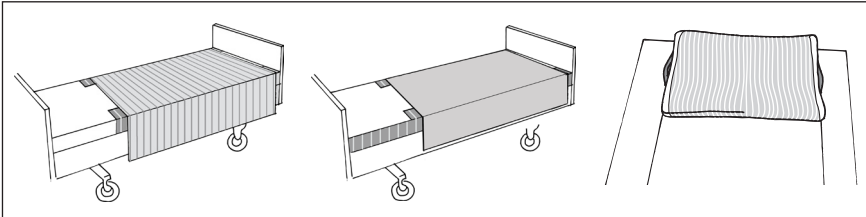With EasyRoll2Way, the user can turn himself/herself without sliding down in the bed when the head end is raised. If the user needs some assistance or turning is heavy, EasyRoll2Way can function as a draw sheet for the caregiver.

With EasyRoll4Way, the user can be helped to a position higher up in bed or closer to one side.

EasyRoll Pillow case with an anti-friction surface on the underside. The anti-friction surface facilitates positioning of the pillow.

Material

Duchesse, Cotton, Polyester

Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Simple solutions for great results

SystemRoMedic™ is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group

Advancing Movement & Health

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com