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EN - Please read the instructions before using the product.

These instructions apply to the Footrests Combi chair.

Purpose

The footrests can be mounted on all models of Combi chairs, giving support to the user's feet.

Assembly

Select a hole at the desired height and insert the snaplock clips (2) securely into the tubing. Slot the footrest tubing into the frame holder, letting it rest securely onto the snaplock pin. Lift the tubing slightly to swing out at 90°.

Adjustment

The footrest height can be adjusted by changing the position of the snaplock pins in the frame tubing. There are 4 positions with 25 mm / 0.9 " increments.

Maintenance

Clean with regular household cleaning agents with non-corrosive or non-abrasive properties. Do not use solvents or chlorine.

Caution!

- Use only as described under "Purpose".
- Do not stand on the footplates when the footrests are mounted onto the chair.
- Footrests should always be used when transporting the user in the chair.
- Footrests should always be used with tilt chairs.
- The footrests must be swung out at 90°, or the footplates flipped up, when the user is dismounting the chair.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com