Footrests

Accessories for Combi chairs

Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 200810004

MAX 50 kg/110 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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Linido

Instructions for use and user

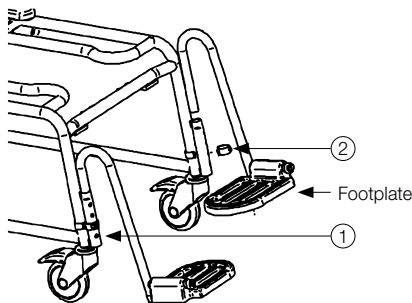
EN - Please read the instructions before using the product. These instructions apply to the footrests for Combi Chairs. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.

Intended use

The footrests can be mounted on all models of Combi chairs, giving support to the user's feet.

Installation

Select a hole at the desired height (1) and insert the snaplock clips (2) securely into the tubing. Slot the footrest tubing into the frame holder, letting it rest securely onto the snaplock pin. Lift the tubing slightly to swing out at 90°.

Adjustment and use

The footrest height can be adjusted by changing the position of the snaplock pins in the frame tubing. There are 4 positions with 25 mm / 0.9" increments.

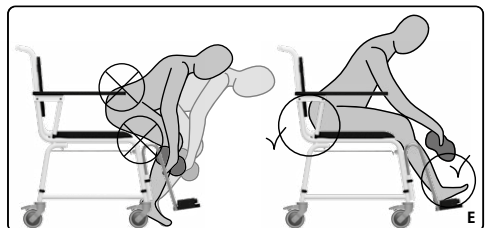
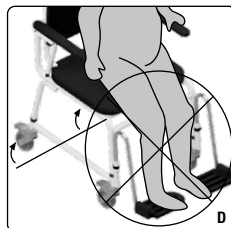
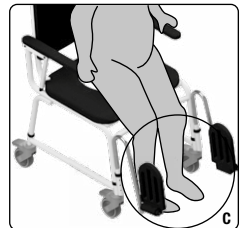
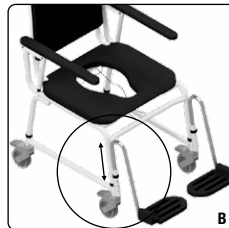
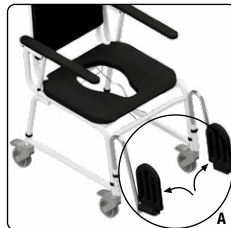
Maintenance

Clean with regular household cleaning agents with non-corrosive or non-abrasive properties. Do not use solvents or chlorine.

Warnings

- Do not stand on the footplates when the footrests are mounted onto the chair.
- Footrests should always be used when transporting the user in the chair.
- Footrests should always be used with tilt chairs.
- The footrests must be swung out at 90°, or the footplates flipped up, when the user is dismantling the chair.

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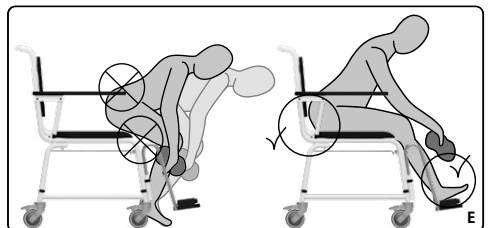
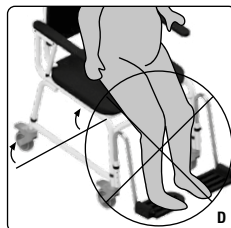
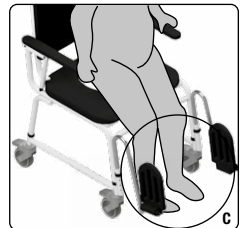
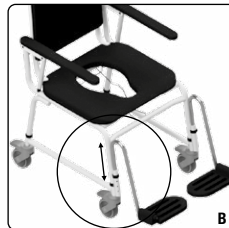
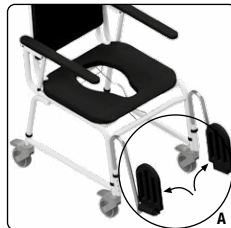
Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.