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EN - Please read the instructions before using the product.

These instructions apply to the front cross bar for all Combi chairs of standard width.

Intended use

The front cross bar fits all Combi chairs of standard width.

The front cross bar is mounted onto the Combi chair with the purpose of increasing security for its users, who might have reduced balance, and a tendency to fall over forwards in the chair. It should be used in combination with the armrest lock combi chair.

Installation

- Remove the armrest on both sides by loosening the two screws (1) and taking the armrests off.
- Attach the armrest with cross bar on the chair. Make sure the screws are tight.
- On the user's righthand side, attach the locking bracket (2) in the rear hole between the armrest and the U-profile.

Adjustment and use

The front cross bar can be opened in two ways:

- On the user's left side, pull down the positioning bolt (3) and push the front cross bar out in front of the user. Swing it all the way around in a position parallel with the armrest on the Combi chair, and lock it in this position.
- On the user's right side pull the positioning bolt (4) out and lift the front cross bar up in front of the user. If you do both of these, then the front cross bar can be removed from the armrests.

Maintenance

The front cross bar can be cleaned with ordinary cleaning products without polishing or etching properties. It is not recommended to use solvents or chlorine. Dry off with a clean cloth afterwards.

Warning!

- Must only be used as described under the section "Intended use".
- Make sure that the positioning bolts go into place when the front cross bar is in use.
- Be certain that the user or the helper does not have their fingers down on the side of the armrest when the front restraint is swung out to one side and locked in that position.
- Always be aware of the user's hands when the front cross bar is opened or closed, as there can be danger of being pinched between the front cross bar and the armrest.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health[®]

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com