Headrest

Accessory for Combi chairs

Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 200360004

MAX 30 kg/66 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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Linido

Instructions for use
and user

EN - Please read the instructions before using the product. These instructions apply to the headrest standard Combi chairs. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.

Purpose

The Headrest can be used on all Combi chair models. The contoured padding lends support to the user's head and neck. The headrest is angle and height adjustable.

Assembly

Before the headrest can be assembled the black covers in the backrest need to be removed. Screw the cylindrical metal bracket onto the backrest push bar using the 2 screws (3) and metal bushes (4) provided. Slot the tubing into the bracket and tighten the hand lever.

Adjustment

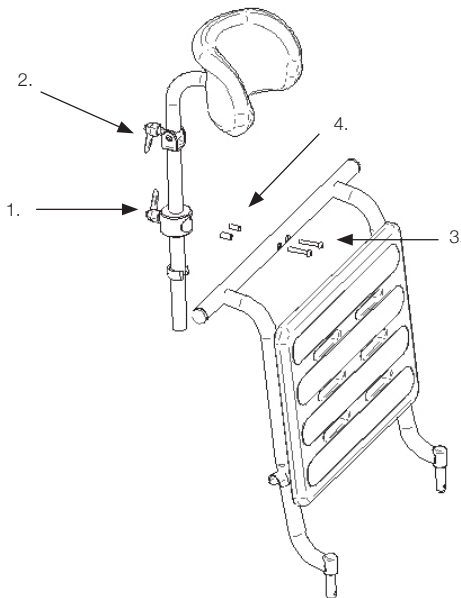
The Headrest can be height adjusted by loosening and tightening the lower hand lever (1). Angle adjustment can be achieved by loosening and tightening the upper hand lever (2).

Maintenance

Clean with regular cleaning agents without corrosive or abrasive properties. We advise against the use of chlorine and solvents.

Warnings

- Use only as described under "Purpose"
- Make sure to tighten the hand levers after adjustment.
- A headrest should always be used on a tilt chair.
- The end stop/clip at the headrest may not be removed. This end stop ensures the safety and the maximum height of the headrest.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.