Lateral trunk support

Right/left, accessory for Combi chairs

Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 201070314 / 201072314

MAX 75 kg/165 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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Lateral trunk support

Right/left, accessory for Combi chairs

Instructions for use
and user

Linido

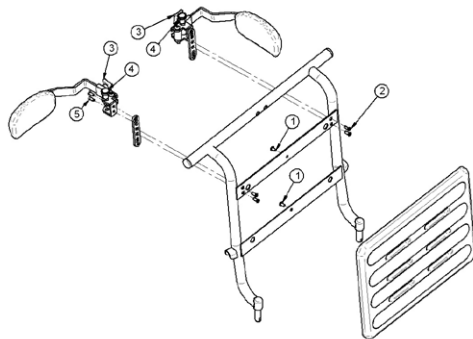
EN - Please read the instructions before using the product. *The instructions apply to lateral trunk supports for the right or left side. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.*

Intended use

The lateral trunk support can be installed on all Combi chairs. It is well suited to people who have a reduced sense of balance and can fall over to the side. The body support works by providing support to the upper body of the user.

Installation

The body support can be supplied for the right or left side. First remove the PUR backrest by loosening the two screws (1). The body support is installed on the back of the Combi chair at the desired height, using the two screws supplied (2). See the drawing. Reinstall the PUR backrest with the two screws (1).

Adjustment and use

When the trunk supports are installed on the Combi chair, they can be adjusted in width by loosening the thumbscrews (3) in the "arm", and adjusting them to the desired position. The trunk supports can be moved away from the user by pulling up on the positioning bolts(4), and then swinging the supports out to the side. When the user is back in the chair, the supports can be swung in again so that they support the user's upper body. The trunk support can be adjusted in height by loosening the screws (5), and positioning the "arm" with the PUR support at the desired height. Then the screws (5) are tightened again.

Maintenance

The trunk support can be cleaned with ordinary cleaning products that do not have polishing or etching properties. It is not recommended to use solvents or chlorine. Please refer to the user manual of the main product for more detailed cleaning instructions.

Warnings

- When the user is moved out of the chair, the trunk supports should be swung out to the side.
- Be certain that the positioning bolts go into the notches, when the supports are swung in to support the user.
- Check on a regular basis that the screws are tight.
- Be careful not to have hands or fingers on the pipe support of the back rest where the trunk support is installed, so as to prevent the risk of being pinched.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.