QuickLink

SystemRoMedic™

Art.no.
70200101

Max. 300 kg /660 lbs

Manual xxxx - Ver. 1, 2020-12-16

Direct Healthcare Group

Advancing Movement & Health®

INTELLIGENT PRESSURE CARE

SPECIALIST SEATING

RENTAL & SERVICE SOLUTIONS

SAFE MOVING & HANDLING

BATHROOM SAFETY SOLUTIONS

Direct Healthcare Group is quality and environmentocertified in accordance with ISO 9001, ISO 13485 and ISO 14001.

Direct Healthcare Group

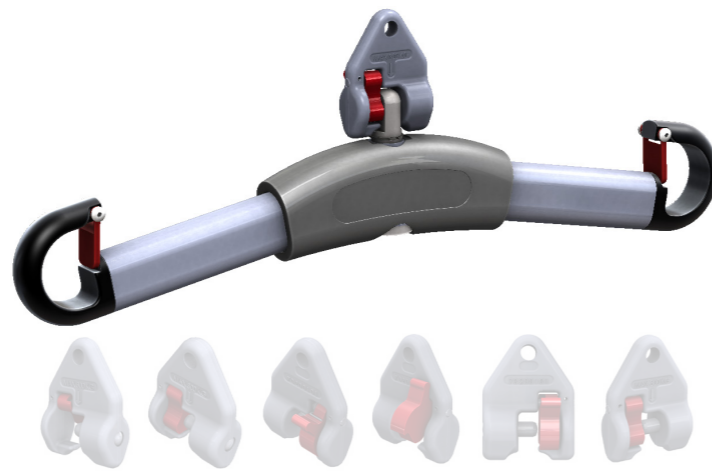
Advancing Movement & Health®

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Manual - English (EN)
Manual - Svenska (SE)
Manual - Norsk (NO)
Manual - Dansk (DK)
Käyttöohje - Suomi (FI)
Gebrauchsanweisung - Deutsch (DE)
Handleiding - Nederlands (NL)
Manual - Français (FR)
Manual - Italiano (IT)
Manual - Español (ES)

Assembly / Monterig / Norwegian heading / Danish heading / Finnish heading

EN Use appropriate tools depending on the type of lift. Tighten by hand. For Floor Lifts: Assemble the QuickLink on the link sticking out of the lift arm. Use a new lock nut if the existing one is too worn out. (Nylon-insert DIN985 M8 FZB).

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Assembly German / Dutch heading / French heading / Italian heading / Spanish heading

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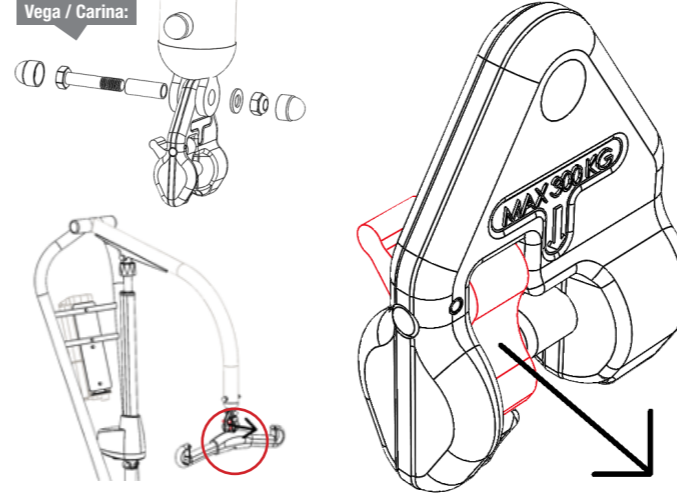
EN The QuickLink can be used with several sling bar types: The standard sling bars, the Stretcherbar, (70200006), and the 4-point sling bar (RFL X4, 70200017). Use hand grips like those described in this manual and ensure proper connection before starting lifting a patient.

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Vega / Carina:

EN Use Two 13mm spanners. The QuickLink needs to be assembled with its opening facing away from the mast. See arrow in image to the left.

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QuickLink

EN QuickLink– Quickconnect system - is an accessory to SystemRoMedic ceiling and mobile lifts. It allows the caregiver to change sling bar or add other accessories quickly and easy without the need for additional tools. The safe working load, SWL, (max patient weight) for the QuickLink is 300kg. The QuickLink is marked with the safe working load, the article number, and a batch number for manufacturer's traceability.

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EN Attaching a sling bar

- 1: Hold the QuickLink with one hand. Tilt and position the sling bar as in the image.
- 2: Push the sling bar onto the indentation in the red latch, until the latch is completely open.
- 3: Slide the sling bar sideways onto the load-bearing pin of the QuickLink.
- 4: Let the sling bar hang down freely and ensure that the latch is completely closed as in the image.

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EN Detaching a sling bar

- 1: Tilt and position the sling bar as in the image.
- 2: Open the latch by squeezing it with your thumb and index and middle finger. Keep it open.
- 3: Slide the sling bar off the load-bearing pin towards the open latch.
- 4: Remove the sling bar from the QuickLink, let go of the latch.

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Technical Information

Article name	QuickLink
Article number	70200101
Maximum user weight	300kg / 660lbs
Material	Forged in highquality steel
Weight	Approx 420g
Tested according to / Standards	Tested according to ISO10535:2006
Expected lifetime	Minimum 20,000 lifts (approx. 5 lift/day for 10 years)
Dimensions	60 x 82 x 30 mm
Height affecting lift height	50 mm
Compatibility, lifts	Vega, Carina, Eva, EvaDrive, RiseAtlas, RiseBasic
Compatibility slingbars	Sling bars S, M, L (article number 70200001, 70200002, 70200003), 4-point slingbar (70200017), StretcherBar (70200006)
Compatibility accessories	Charder scale, 70100002 (Older Charders with adapter components)

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FI

Warnings:

- The caregiver must be able to read and understand this Manual/IFU of the product.
- For optimal function and safety, the product should be inspected regularly, see the Maintenance section.
- The product must be properly fitted and tested in relation to the user’s needs and functional ability.
- The product must only be used with SystemRoMedic lifts, sling bars, slings and accessories.

Precautions:

- Check that the product is correctly mounted/assembled before its first use.
- Verify that the product hangs vertically and can move freely.
- Check that the sling bar is correctly attached to the QuickLink and the sling correctly to the sling bar before proceeding with a lift.

Contraindications:

- The product may not be used by patients above its maximum load
- The product must not be lowered into water or used in the shower/wet rooms.
- The product must not be left or stored in a damp or humid environment.
- Do not lift more weight than the limit specified on any of the components (lift, sling, sling bar, QuickLink, etc)

DE

Warnings:

- The caregiver must be able to read and understand this Manual/IFU of the product.
- For optimal function and safety, the product should be inspected regularly, see the Maintenance section.
- The product must be properly fitted and tested in relation to the user’s needs and functional ability.
- The product must only be used with SystemRoMedic lifts, sling bars, slings and accessories.

Precautions:

- Check that the product is correctly mounted/assembled before its first use.
- Verify that the product hangs vertically and can move freely.
- Check that the sling bar is correctly attached to the QuickLink and the sling correctly to the sling bar before proceeding with a lift.

Contraindications:

- The product may not be used by patients above its maximum load
- The product must not be lowered into water or used in the shower/wet rooms.
- The product must not be left or stored in a damp or humid environment.
- Do not lift more weight than the limit specified on any of the components (lift, sling, sling bar, QuickLink, etc)

NL

Warnings:

- The caregiver must be able to read and understand this Manual/IFU of the product.
- For optimal function and safety, the product should be inspected regularly, see the Maintenance section.
- The product must be properly fitted and tested in relation to the user’s needs and functional ability.
- The product must only be used with SystemRoMedic lifts, sling bars, slings and accessories.

Precautions:

- Check that the product is correctly mounted/assembled before its first use.
- Verify that the product hangs vertically and can move freely.
- Check that the sling bar is correctly attached to the QuickLink and the sling correctly to the sling bar before proceeding with a lift.

Contraindications:

- The product may not be used by patients above its maximum load
- The product must not be lowered into water or used in the shower/wet rooms.
- The product must not be left or stored in a damp or humid environment.
- Do not lift more weight than the limit specified on any of the components (lift, sling, sling bar, QuickLink, etc)

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IT
<p>Warnings:</p> <ul style="list-style-type: none">The caregiver must be able to read and understand this Manual/IFU of the product. For optimal function and safety, the product should be inspected regularly, see the Maintenance section. The product must be properly fitted and tested in relation to the user’s needs and functional ability. The product must only be used with SystemRoMedic lifts, sling bars, slings and accessories.
<p>Precautions:</p> <ul style="list-style-type: none">Check that the product is correctly mounted/assembled before its first use. Verify that the product hangs vertically and can move freely. Check that the sling bar is correctly attached to the QuickLink and the sling correctly to the sling bar before proceeding with a lift.
<p>Contraindications:</p> <ul style="list-style-type: none">The product may not be used by patients above its maximum load The product must not be lowered into water or used in the shower/wet rooms. The product must not be left or stored in a damp or humid environment. Do not lift more weight than the limit specified on any of the components (lift, sling, sling bar, QuickLink, etc)

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SystemRoMedic

EN SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.
Transfer, between two locations.
Positioning, within the same location.
Support, for mobility.
Lifting, both manual and mechanical.

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FI SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.
Transfer, between two locations.
Positioning, within the same location.
Support, for mobility.
Lifting, both manual and mechanical.

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IT SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.
Transfer, between two locations.
Positioning, within the same location.
Support, for mobility.
Lifting, both manual and mechanical.

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EN
<p>Maintenance</p> <p>The QuickLink must undergo thorough inspection at least once per year as part of the service and inspection of the lift itself. Inspection must be performed by authorized personnel. The QuickLink must not be discarded as general waste, but as metal scrap.</p>
<p>Always read the manual</p> <p>Keep the manual where it is easily accessible for users of the product.</p>

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