RollerSlide

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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
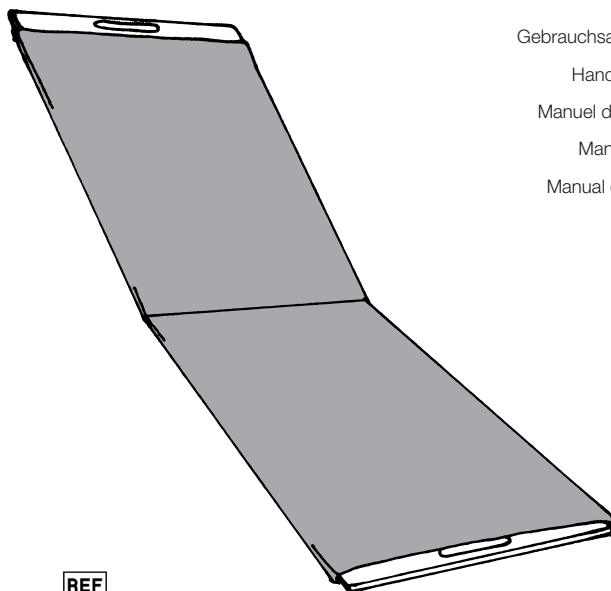
REF

9110 9155
9120 9158
9145 9170
9148 9175

Accessories:

REF

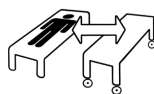9136 9138
9156 9168
9172
9177

SWL: 120-300 kg/ 265-661 lbs

**Direct Healthcare
Group**

Advancing Movement & Health[®]

Table of contents

English	3
Svenska	5
Norsk	7
Danske	9
Suomi	11
Deutsch	13
Netherlands	15
Français	17
Italiano	19
Español	21
<hr/>	
Size and Accessories	23
Symbols	23

Differences in height can pose a big problem. And sometimes it is difficult to get close enough to ensure smooth transfers for both users and caregivers. RollerSlide is used for supine transfers of inert users between beds and an x-ray or operating tables, shower trolleys, ambulance stretchers or similar. RollerSlide bridges gaps and moves with the user during the transfer.

RollerSlide is a padded sliding board with a sliding cover made from ULF (Ultra Low Friction), Direct Healthcare Group's own sophisticated sliding material. The sliding cover slides round the board during transfer, reducing the friction under the user and thereby facilitating a sliding transfer without unnecessary lifting for the caregivers. RollerSlide also ensures very gentle transfers for users who are injured or very sensitive to pain.

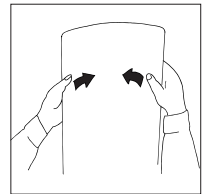
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Stability test

Carefully bend the assistive device to check strength and durability. The mat should be both elastic and rigid. Discard the EasyGlide when its strength and durability are diminished.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

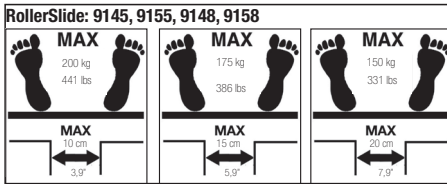
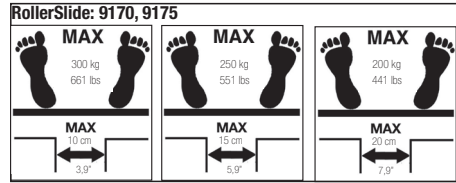
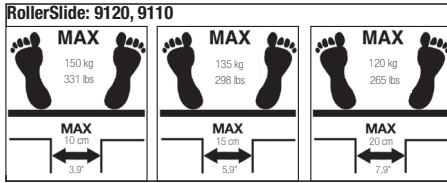
Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.


The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Important information

RollerSlide must always be removed before x-raying, but RollerSlide Light can be left under the user during x-raying. Do not leave the user unattended during a transfer situation.


Placement/removal of RollerSlide

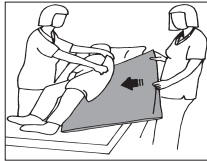
Lean the user to one side and guide the RollerSlide in under 1/3 of the body. To remove, lean the user to one side and pull out the RollerSlide.

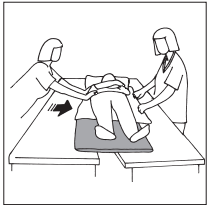Using RollerSlide

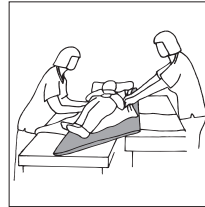Caregivers should work on either side of the user, and push and pull while standing in a walking stance. Distance between underlying surfaces must not exceed 20 cm.

Roller Slide can also be used where there is a minor difference in height between the underlying surfaces.

Material

Polyamid, Nylon

Care of the product

RollerSlide board: Clean with disinfectant.

The sliding cover can be machine laundered. Do not use fabric softener. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.