Shower bench soft seat

Linido

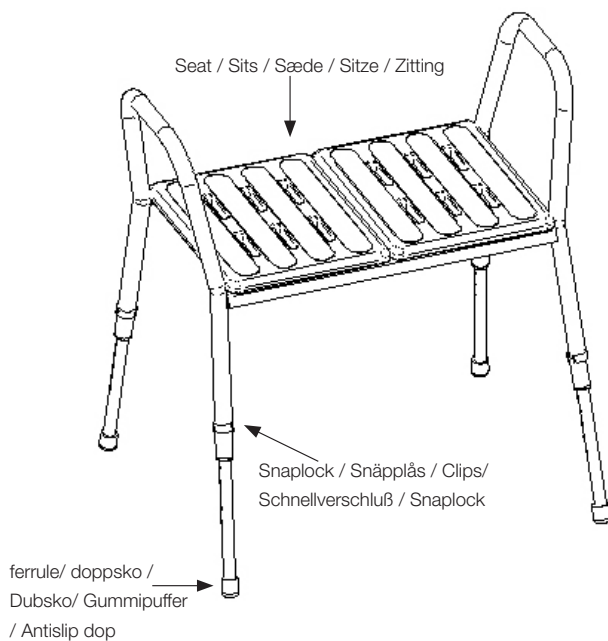
Manual - English

Manual - Svenska

Manual - Danske

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Art.no.: 200550313

SWL: 300 kg/661 lbs

**Direct Healthcare
Group**

Advancing Movement & Health®

**EN Visual inspection**

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EN - Please read the instructions before you start using this product

Please keep the user instructions for future reference.

Purpose

The shower bench is for use in a bathroom environment, eg., in a showering situation or for placement in front of a washbasin. The shower bench has extra high side handles.

Assembling – see drawing, page 10

The loose legs are mounted in the frame and secured with the snaplock clips. Make sure the clips are properly inserted in the leg holes. The seat is pressed onto the seat frame, with the 4 teats securely inserted in the 4 holes.

Check the correct position of the seat before use.

Adjustment – see drawing, page 10

The shower bench can be height adjusted in 8 different positions by the use of snaplock clip. Remove the snaplock clip on the legs, choose the new height position and replace the snaplock clip. Before use, make sure that all 4 legs have the same height and that the snaplock clips are properly inserted in the leg tubing. The rear legs can be height adjusted to slope the seat forwards, however, not more than max. 2 holes difference to the front legs.

Maintenance

We recommend that the shower bench be wiped after use. Clean with regular domestic cleaning agents without abrasive or corrosive properties. We advise against using acidic or chlorine based cleaning agents on the seat as this can cause discolouration to the polyurethane material. Polyurethane can tolerate temperatures up to 85 °C / 3 minutes.

Caution!

- Max weight load. 300 kg.
- Should be placed against a wall for optimal safety.
- Use the product only as instructed in the section "Purpose".
- Do not stand on the shower bench.
- Do not lift the shower bench when the user is seated on it.
- Do not push or pull the shower bench across the floor when the user is seated on it.
- For optimal safety, regularly check the condition of the rubber ferrules and replace if necessary.
- For optimal safety, regularly check the condition and correct placement of the snaplock clips.
- Check that the snaplock clips are correctly inserted in the tubing before using the shower bench.
- Height adjustment of the rear legs should be max. 2 holes difference to the front legs.

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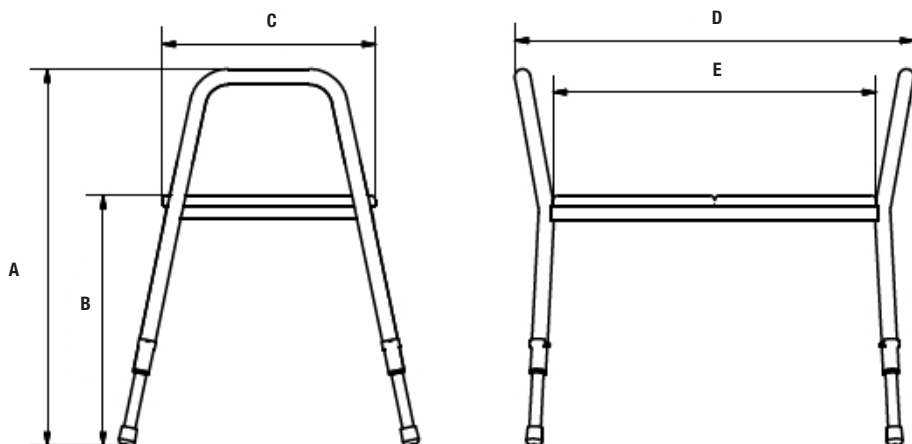
Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com