Toilet raiser with lid

Lindo

User manual - English

Bruksanvisning - Svenska

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REF
10735

SWL: 130 kg/285 lbs

Direct Healthcare
Group

Advancing Movement & Health®

Table of contents

English	3
Svenska	4
Deutsch	5
Nederlands	6
Technical information	7

Toilet raiser with lid, raises toilet seat height by 100 mm/4"

Safety information

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use

Ensure that the product is correctly mounted/installed. Be sure that adjustment knobs on sides of seat are tight.

Always read the user manual

Always read the user manuals for all assistive devices used. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Installation instructions

Remove toilet seat and lid from toilet. Turn both locking clamps counter clockwise until they can no longer turn. This will open the locking devices. Place the toilet raiser on the toilet. Be sure the two locking brackets are on both sides of the toilet and that the raiser is flush with the toilet. Turn the locking brackets clockwise until tight. Hand tighten only. Do not use tools as threads on the adjustment bracket may strip.

Removal instructions

Rotate knobs counter clockwise until seat is loose. Lift front of toilet raiser and pull forward to remove.

Using the product

The toilet raiser increases the height of a toilet by 100 mm/4". User's weight should be centered over toilet seat.

Warning

- Do not load the seat surface on one side only.
- Under no circumstance stand on the seat.

Material

Plastic

Care of the product

Clean regularly with normal household detergent. Do not use aggressive cleaners.

Medical Device Class I

The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.directhealthcare-group.com for a complete list of distributors.

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Verwachte levensduur product

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Technical information

Material	Plastic
Total depth	412 mm, 16.2"
Total height	155 mm, 6.1"
Raiser	100 mm, 4"
Hygiene cutouts	In front and back
Toilet opening, width x length	200 mm x 270 mm, 7.8" x 10.6"
Weight	1.5 kg, 3.3 lbs
Max. user weight	130 kg/285 lbs

Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com