Instructions for use – English

Bruksanvisning – Svenska

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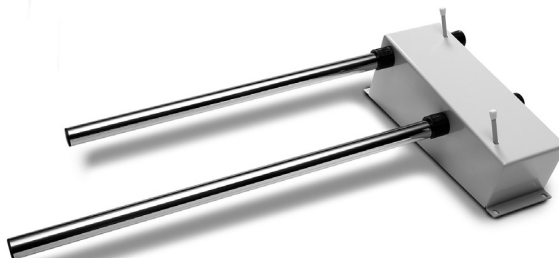
REF

1085

Table of contents

| | |
|-------------------------|-----------|
| English | 3 |
| Svenska | 4 |
| Norsk | 5 |
| Dansk | 6 |
| Suomi | 7 |
| Deutsch | 8 |
| Nederlands | 9 |
| Français | 10 |
| Italiano | 11 |
| Español | 12 |
| Symbols | 13 |

WallRack is a wall-mounted hanger that is used mainly for storing disposable covers for EasySlide and RollerSlide, but can also be used for storing the products themselves.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the user manual

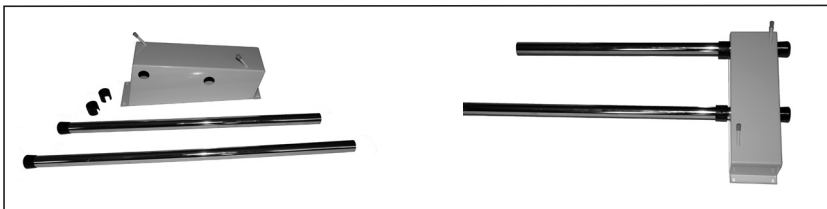
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Wallrack

WallRack can be mounted either right-hung or left-hung on the wall.

Easily disassembled for remounting hangers on either the left or right side of the mid-section.

Care of the product

Clean with dishwashing liquid or disinfectant.

Wipe with a cloth.

Expected lifetime

Up to 10 years of normal use

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Symbols

| | | | |
|--|---|---|-------------------|
|  | This product complies with the requirements of the Medical Device Regulation 2017/745 |  | Visual Inspection |
|  | Medical Device |  | Read the manual |
|  | Caution |  | Product Code |
|  | Manufacturer information |  | Batch Code |

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

**Direct Healthcare
Group**

Advancing Movement & Health®

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.