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English

Slingbar RFL X4 is a four-point sling crossbar that is intended for use with Handicare-approved mobile and overhead lifts. It is designed for use with correctly fitted slings for lifting patients. Slingbar RFL X4 is designed to provide more space in the sling, for example, for obese and/or pain-sensitive patients. Slingbar RFL X4 is made of steel.

SWL: 300 kg/661 lbs

Functional inspection

Visual inspection

Inspect the sling bar regularly. Check to ensure that material is free from damage.

Before use:

Ensure that the product is correctly assembled.
Check sling bar connection and safety latch function.

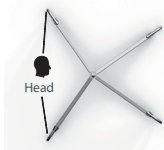
Always read the manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

The lift may only be used by persons who have received instruction in the operation of the lift.

Important information

- Lifting accessories must be properly trial fitted and tested in relation to the patient's needs and functional ability.
- Do not leave the patient unattended during a transfer situation.
- NOTE!** The lift sling's back loops are connected to the wider section of the slingbar (see image).

Safe working load

Different products on the same lift system (lift unit, slingbar, sling, scales and other lifting accessories) may have different allowable safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Cleaning

Clean the lift with warm water or rubbing alcohol and ensure that the castor wheels are free of dirt and hair. Do not use cleaning agents containing phenol or chlorine, as this may damage the material.

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