English
Svenska
Norske
Danske
Deutsch
Français

Ready for life

Manual - ReadyBelt

Manual nr: 00802 Ver. 3 110211

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

English

Svenska

ReadyBelt is a belt that is designed to provide the user with support and help during mobility training and different transfer situations. For the caregiver, ReadyBelt offers an ideal means of supporting the user during mobility training or visits to the toilet.

Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Pull hard on the handles in different directions (1).

Fasten the belt, place one foot in it, and then pull on one of the handles (2). Repeat the procedure with all handles.

Always read the instructions

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

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
Placement of ReadyBelt

1. Apply the belt around the patient, below the waist but above the hips.

2. Fasten the belt and tighten it.

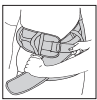English

Removing ReadyBelt

Release the clasp, first loosening any straps, and then remove the belt.

Using ReadyBelt

Caregivers can work in pairs or alone, depending on the patient's needs, since ReadyBelt has several vertical and horizontal handles. Cross-grips at the back provide support and enable a good working stance (1 and 2).

When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the patient by the hand at the front (3).

ReadyBelt provides good support during transfers between wheelchair and bed (4).

Material:

Polypropylene

Care of the product:

Read the product label.  Do not use rinsing agent. To prolong product life, avoid tumble-drying.

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