

# Mover Aqua

## Quick Guide

This is a quick guide regarding how a transfer is carried out with Mover Aqua.

The steps below shows a transfer from one chair to another, but the same approach applies regardless of whether the transfer is to/from a wheelchair, bed or toilet.



1. Fold up the two half-seats.



2. Manoeuvre up to the care recipient.



3. Lock the brakes.



4. Place the Mover Aqua so the care recipient can easily place his or her feet on the step and his other knees/shins against the shinsupport. Let the care recipient take hold of the crossbars/handles however he or she wishes.



5. Fold down the two half-seats and ask the care recipient to sit down. With the stable platform and comfortable seat cushions, and thanks to constant eye contact with the carer, the care recipient always feels comfortable and confident during the transfer with Mover Aqua.



6. Release the brakes and move the care recipient. When the care recipient is in position, lock the brakes and ask he or she to stand up. Remove the care recipients clothing if necessary and then fold up the half-seats. Ask the care recipients to carefully sit down on the chair/wheel chair/ bed/toilet.

## Safety instructions

- This product is only intended for moving care recipients. No other use is permitted.
- This product has been constructed and tested to safely move/transfer care recipients with a body weight of up to 180 kg.
- Always carry out recommended daily maintenance before using the product.
- Always move the product using the handlebars.
- Before using the product, a clinical assessment must be carried out as to whether the care recipient can be moved using this product and, if so, whether the move/transfer should be carried out by one or two carers.
- Do not move a loaded product over obstacles that the castors cannot easily get over (thick carpet edges, thresholds, etc.).
- Mover Aqua can also be used in a shower. Repair any paint damage as soon as possible to avoid a shorter lifespan due to corrosion.
- Do not allow the Mover Aqua to stand in direct sunlight since dark surfaces can become extremely hot.

## Cleaning

The product must be regularly cleaned/disinfected – after every use if necessary or at least once a day. Clean the product by wiping with a damp cloth moistened with warm water and disinfectant detergent. Dry thoroughly to effectively disinfect all surfaces. Do not use chlorine, phenols, pentanols or other strong detergents that could damage the surface/dry out the cushions or cause premature cracking.

