

Sheets for all occasions

System**RoMedic**[™]




- Sliding sheets
- Draw sheets
- Sliding draw sheets
- Incontinence protection
- Mattress covers etc

**Direct Healthcare
Group**


Advancing Movement & Health[®]

Sheet guide


Choose the correct sliding zone on the sheet depending on user function.

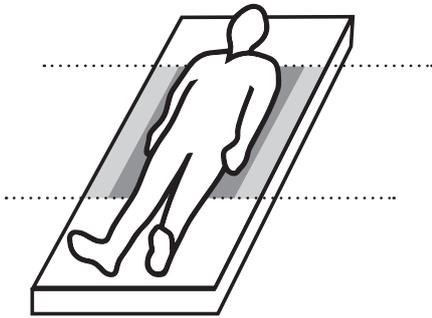

Sliding zone Hip: For more active/lighter users who only need the sliding function at hip level.

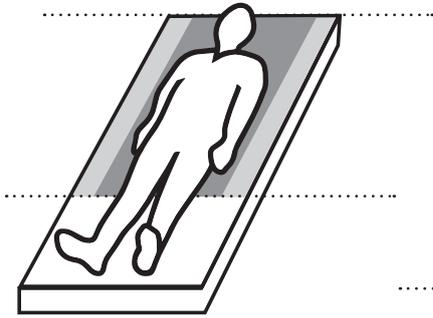


Sliding zone Head-Hip: For less active/heavier users who need the sliding function from hip level and up.

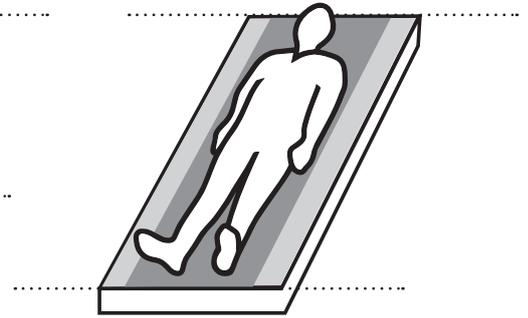

Sliding zone Head-Feet: For users who are passive/in need of more intensive care where the sliding function is required under the entire body.

Choose the correct sliding sheet based on bed width.

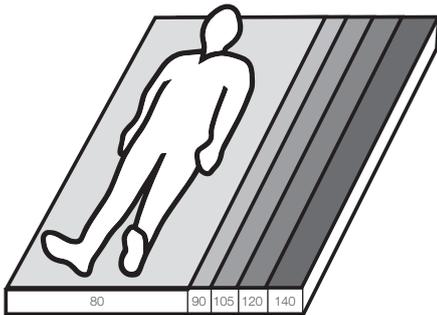

80 cm 31.5"	90 cm 35.4"	105 cm 41.3"	120 cm 47.2"	140 cm 55.1"
----------------	----------------	-----------------	-----------------	-----------------

Bed width

The mattress dimensions are important for optimising the sliding surface of the sheet and stimulating the activity of the user. For our draw sheets and sliding draw sheets, there is no recommended mattress width.

Find the correct function.



Incontinence protection

Incontinence protection is a washable incontinence sheet placed on top of an ordinary sheet and/or other bedding. Incontinence protection is available with or without a sliding surface on the bottom. The material is highly absorbent and distributes liquid over a larger surface.



Handle

Some sliding draw sheets are also equipped with handles that can be connected to lifts. This facilitates easier positioning, takes care of lower body hygiene and facilitates harness attachment.



Elastic


Sliding sheets with this symbol have elastic that is pulled over the mattress. Other sheets have side sections that are tucked in under the mattress.

Care instructions.


Choose the right product bearing in mind the care instructions.




Select the correct combinations.

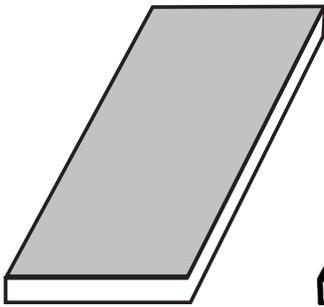


Mattress cover with barrier layer. Placed on the top of the mattress.

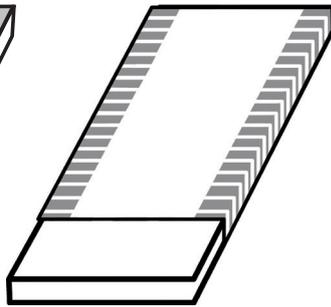

Sliding sheet. Satin for the centre section and non-slip material for the side sections. Slide in the direction of the pattern.

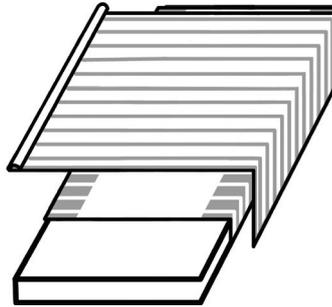

Sliding sheet + draw sheet. Facilitates turning and reduces the heat from the satin of the sliding sheet. Slide in the direction of the pattern.

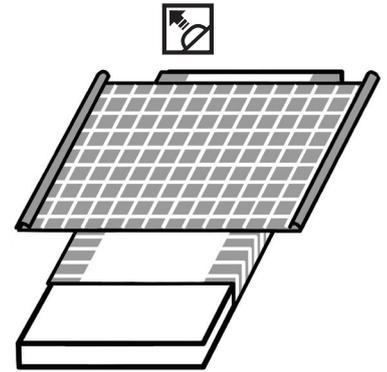


Sliding sheet + sliding draw sheet. Both the sheets have sliding surfaces on both the bottom and the top. Slide in the direction of the pattern.


How to use the combinations.



For patients able to turn over on their own, often a sliding sheet that improves the slide in the zones where this is needed is sufficient.

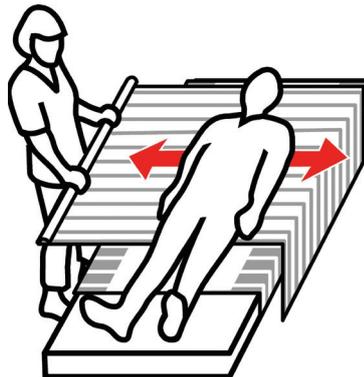

A combination of a sliding sheet and a draw sheet is used when the user is less active, heavier/requires more intensive care.

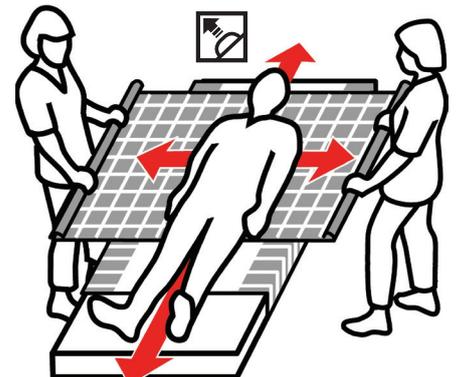

The combination of a sliding sheet and a sliding draw sheet is used when the user is passive, heavy/ in need of more intensive care. This combination is often used with a lift, during hygiene work and when attaching a harness.

Find the right sheet.

Example from item list:

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
1661	WendyLett	120/47.2	120/47.2	120/47.2	95/37.4	Grey/white striped	Sliding sheet	Possible info

Item number (1661) | Product name (WendyLett) | The sheet is suitable for this bed width (120/47.2) | Dimensions (length + width) (120/47.2) | Zone/sliding surface (95/37.4) | Product colour (Grey/white striped) | Product function (Sliding sheet) | Other info (Possible info) | Special properties (Icons) | Sheet + sliding zone on the mattress (Art. No. 1661) | Repetition of Item number (1661)


WendyLett portfolio




WendyLett Sliding Sheet
page 5



WendyLett2Way Draw Sheet
page 6

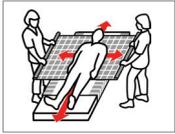

WendyLett4Way Sliding Draw Sheet
page 7

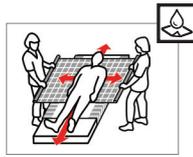


WendyLett inco range WendyLett 4Way, inco Sliding Draw Sheet
page 7



WendyLean pillow case
page 8

EasyRoll portfolio

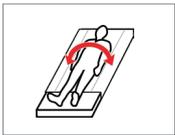

EasyRoll Sliding Sheet
page 9-11



EasyRoll Sliding Sheet, inco
page 11



EasyRoll 2way Draw Sheet
page 12



EasyRoll Inco 2way Sliding Sheet
page 12




EasyRoll 4way Sliding Drawsheet
page 13



EasyRoll 4way Sliding Drawsheet, inco
page 13



EasyRoll Pillow Case
page 15

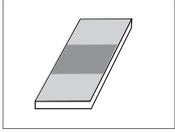

EasyRoll underpads and mattress protector
page 16-18



WendyLett Sliding Sheet

Our WendyLett products are our patented high-quality systems designed to facilitate repositioning higher up in bed, turning and minor repositioning in bed.

Together, Direct Healthcare Group's three WendyLett sheets constitute a well thought out system of sliding and draw sheets that complement each other and contribute to making day-to-day life easier for both users, healthcare professionals and private carers.

WendyLett Sliding Sheet

WendyLett is a satin sliding sheet. Sliding takes place sideways, in the direction of the stripes. WendyLett is for users who have trouble turning over in bed. WendyLett has a smooth centre section, which means very little energy is required to turn over. It is very easy for the user to slide sideways, but they are stopped by a non-slip material on the side sections so that they do not slide out of the bed. The material also prevents the

user from sliding down the bed. When not in use, the WendyLett can be tucked in around the bed like a normal sheet, with the smooth surface in the centre of the bed. For users who need more help, WendyLett can be combined with the WendyLett2Way draw sheet or the WendyLett4Way sliding draw sheet.

WendyLett is available in several different versions and sizes.



Product overview WendyLett Sliding Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Sheet	Sheet length	Sliding zone	
1609	WendyLett	80-90/31.5-35.4	100/39.4	200/78.7	60/23.6	White	Sliding sheet		Art. No. 1609			
1614	WendyLett	80-90/31.5-35.4	140/55.1	200/78.7	60/23.6	White	Sliding sheet		Art. No. 1614			
1629	WendyLett	80-90/31.5-35.4	100/39.4	200/78.7	60/23.6	Grey/white striped	Sliding sheet		Art. No. 1629			
1634	WendyLett	80-90/31.5-35.4	140/55.1	200/78.7	60/23.6	Grey/white striped	Sliding sheet		Art. No. 1634			
1641	WendyLett	90/36	200/78.7	90/36	70/27.6	Grey/white striped	Sliding sheet		Art. No. 1641			
1639	WendyLett	105/41.3	200/78.7	200/78.7	70/27.6	Grey/white striped	Sliding sheet		Art. No. 1639			
1651	WendyLett	105/41.3	200/78.7	105/41.3	80/31.5	Grey/white striped	Sliding sheet		Art. No. 1651			
1667	WendyLett	120/47.2	100/39.4	220/86.6	100/39.4	Grey/white striped	Sliding sheet		Art. No. 1667			
1668	WendyLett	120/47.2	140/55.1	220/86.6	100/39.4	Grey/white striped	Sliding sheet		Art. No. 1668			
1669	WendyLett	120/47.2	200/78.7	220/86.6	100/39.4	Grey/white striped	Sliding sheet		Art. No. 1669			
1661	WendyLett	120/47.2	200/78.7	120/47.2	95/37.4	Grey/white striped	Sliding sheet		Art. No. 1661			
1671	WendyLett	140/55.1	200/78.7	140/55.1	110/43.3	Grey/white striped	Sliding sheet		Art. No. 1671			
1681	WendyLett	-	200/78.7	365/143.7	70/27.6	Grey/white striped	Sliding sheet	Extra wide to allow for optional placing of sliding surface	Art. No. 1681			

Material: Polyester, cotton, satin

Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.



CE Class 1 medical device



WendyLett2Way Draw Sheet

WendyLett2Way is a draw sheet made from a material adapted for WendyLett sliding sheets. Sliding takes place sideways, in the direction of the stripes, which means that the user will not slide down in the bed.

WendyLett2Way is placed on top of the WendyLett sliding sheet. The combination makes it easy for the care assistant to turn the user sideways. WendyLett2Way is available in several versions and sizes.



Product overview WendyLett2Way Draw Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
1620	WendyLett2Way	-	90/35.4	180/70.9	-	White	Draw sheet	Art. No. 1620
1631	WendyLett2Way	-	100/39.4	200/78.7	-	Grey/white striped	Draw sheet	Art. No. 1631
1636	WendyLett2Way	-	140/55.1	200/78.7	-	Grey/white striped	Draw sheet	Art. No. 1636

Material: Polyester, cotton, satin

Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.



CE Class 1 medical device



Our WendyLett products



- Thin, high-quality fabric that breathes
- Suitable for users at risk of developing pressure ulcers
- Clear pattern shows slide direction



WendyLett4Way Sliding Draw Sheet

WendyLett4Way is a sliding draw sheet that can be combined with a WendyLett sliding sheet to facilitate sliding both sideways and up/down in the bed, as the grid pattern indicates. WendyLett4Way is placed on top of the WendyLett sliding sheet. The combination allows positioning in four directions and thereby facilitates

turning in bed, repositioning higher up in bed and minor repositioning while lying in bed, even with large, heavy or completely passive users. WendyLett4Way is available in many versions and sizes.

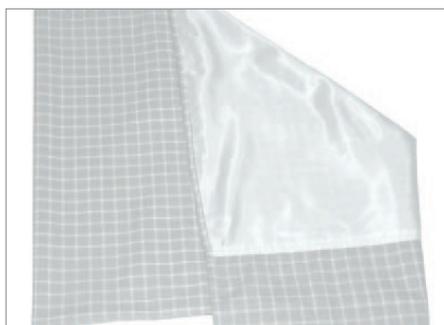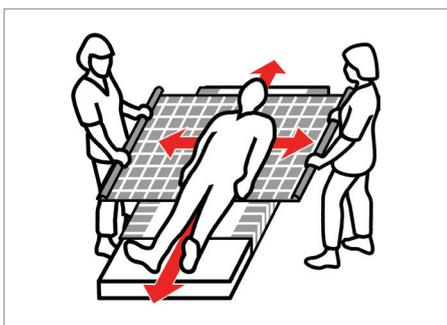

Product overview WendyLett4Way Sliding Draw Sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
1657	WendyLett4Way	-	140/55.1	190/74.8	-	Grey/white checked	Sliding draw sheet	2 x 7 handles	
1637	WendyLett4Way	-	140/55.1	200/78.7	-	Grey/white checked	Sliding draw sheet		
1647	WendyLett4Way	-	200/78.7	200/78.7	-	Grey/white checked	Sliding draw sheet		
1652	WendyLett4Way	-	140/55.1	260/102.4	-	Grey/white checked	Sliding draw sheet	Wider mattress	

Material: Polyester, cotton, satin

Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.



CE Class 1 medical device



Product overview WendyLett4Way Inco Sliding Draw Sheet


Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
1648	WendyLett4Way, inco	-	75/29.5	200/78.7	-	Grey/white checked	Sliding draw sheet		
1649	WendyLett4Way, inco	-	140/55.1	120/47.2	-	Grey	Sliding draw sheet	2 x 7 handles	

Material: Polyester, cotton, satin

Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.




CE Class 1 medical device

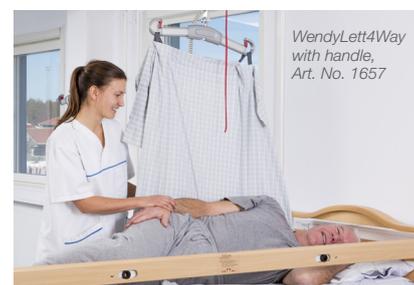

WendyLett4Way with handle, Art. No. 1657

WendyLean Pillow Case

The patented, self-retracting handles are not the only features of this pillowcase, but they are very handy when you need to help a user move higher up in bed or into a seated position at the edge of the bed. The underside of the pillowcase is made of smooth satin which makes it very easy both to position the pillow and to pull the pillow in under the shoulders for raising the upper body of the user. The stripes indicate in which direction sliding is facilitated.

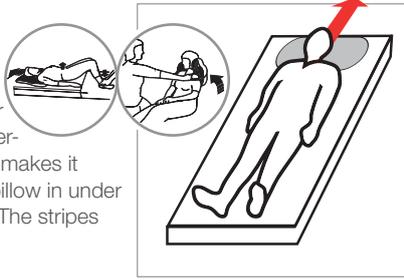

- Smooth satin with self-retracting handles

Product overview WendyLean Pillow Case

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
1680	WendyLean	-	50/19.7	60/23.6	-	Grey/white striped	Pillow case	2 handles	

Material: Top: Polyester/cotton. Rear: Cotton/polyester. Underside: Satin
Care instructions: Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.



EasyRoll Sliding Sheet White

The economic alternative to our WendyLett system. The difference is that EasyRoll is unlined and consists of a sliding surface laminated on a sheet. The EasyRoll system is available in numerous versions and sizes with various functions adapted to ensure you can find the best solution for your needs.

- Marker seams for easier placement of the sheet



Product overview EasyRoll Sliding Sheet / White

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
41801PS	EasyRoll Sliding Sheet	80/31.5	100/39.4	180/70.9	60/23.6	White	Sliding sheet	Art. No. 41801PS 
41802PS	EasyRoll Sliding Sheet	80/31.5	140/55.1	180/70.9	60/23.6	White	Sliding sheet	Art. No. 41802PS 
41803PS	EasyRoll Sliding Sheet	80/31.5	200/78.7	180/70.9	60/23.6	White	Sliding sheet	Art. No. 41803PS 
41901PS	EasyRoll Sliding Sheet	90/35.4	100/39.4	200/78.7	60/23.6	White	Sliding sheet	Art. No. 41901PS 
41902PS	EasyRoll Sliding Sheet	90/35.4	140/55.1	200/78.7	60/23.6	White	Sliding sheet	Art. No. 41902PS 
41903PS	EasyRoll Sliding Sheet	90/35.4	200/78.7	200/78.7	60/23.6	White	Sliding sheet	Art. No. 41903PS 
411051PS	EasyRoll Sliding Sheet	105/41.3	100/39.4	215/84.6	75/29.5	White	Sliding sheet	Art. No. 411051PS 
411052PS	EasyRoll Sliding Sheet	105/41.3	140/55.1	215/84.6	75/29.5	White	Sliding sheet	Art. No. 411052PS 
411053PS	EasyRoll Sliding Sheet	105/41.3	200/78.7	215/84.6	75/29.5	White	Sliding sheet	Art. No. 411053PS 
411201PS	EasyRoll Sliding Sheet	120/47.2	100/39.4	230/90.6	90/35.4	White	Sliding sheet	Art. No. 411201PS 
411202PS	EasyRoll Sliding Sheet	120/47.2	140/55.1	230/90.6	90/35.4	White	Sliding sheet	Art. No. 411202PS 
411203PS	EasyRoll Sliding Sheet	120/47.2	200/78.7	220/86.6	90/35.4	White	Sliding sheet	Art. No. 411203PS 

Material: Side sections: Cotton/polyester. Sliding surfaces: Duchesse (polyester)

Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.





CE Class 1 medical device



EasyRoll Sliding Sheet White/blue striped

The economic alternative to our WendyLett system. The difference is that EasyRoll is unlined and consists of a sliding surface laminated on a sheet. The EasyRoll system is available in numerous versions and sizes with various functions adapted to ensure you can find the best solution for your needs.

- Marker seams for easier placement of the sheet

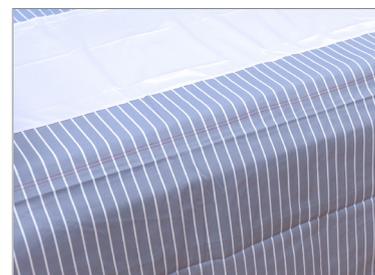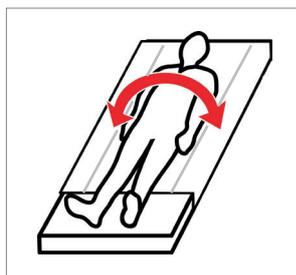

Product overview EasyRoll Sliding Sheet / White/blue striped

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Sheet	Sheet length	Sliding zone	
418011	EasyRoll Sliding Sheet	80/31.5	100/39.4	180/70.9	50/19.7	White/blue striped	Sliding sheet		Art. No. 418011			
418021	EasyRoll Sliding Sheet	80/31.5	140/55.1	180/70.9	50/19.7	White/blue striped	Sliding sheet		Art. No. 418021			
418031R	EasyRoll Sliding Sheet	80/31.5	200/78.7	180/70.9	60/23.6	White/blue striped	Sliding sheet		Art. No. 418031R			
419011	EasyRoll Sliding Sheet	90/35.4	100/39.4	190/74.8	60/23.6	White/blue striped	Sliding sheet		Art. No. 419011			
419021	EasyRoll Sliding Sheet	90/35.4	140/55.1	190/74.8	60/23.6	White/blue striped	Sliding sheet		Art. No. 419021			
419031	EasyRoll Sliding Sheet	90/35.4	200/78.7	190/74.8	60/23.6	White/blue striped	Sliding sheet		Art. No. 419031			
419031R	EasyRoll Sliding Sheet	90/35.4	200/78.7	90/35.4	70/27.6	White/blue striped	Sliding sheet		Art. No. 419031R			
4110511	EasyRoll Sliding Sheet	105/41.3	100/39.4	205/80.7	75/29.5	White/blue striped	Sliding sheet		Art. No. 4110511			
4110521	EasyRoll Sliding Sheet	105/41.3	140/55.1	205/80.7	75/29.5	White/blue striped	Sliding sheet		Art. No. 4110521			
4110531	EasyRoll Sliding Sheet	105/41.3	200/78.7	205/80.7	75/29.5	White/blue striped	Sliding sheet		Art. No. 4110531			
4110531R	EasyRoll Sliding Sheet	105/41.3	200/78.7	105/41.3	85/33.5	White/blue striped	Sliding sheet		Art. No. 4110531R			
4112011	EasyRoll Sliding Sheet	120/47.2	100/39.4	220/86.6	90/35.4	White/blue striped	Sliding sheet		Art. No. 4112011			
4112021	EasyRoll Sliding Sheet	120/47.2	140/55.1	220/86.6	90/35.4	White/blue striped	Sliding sheet		Art. No. 4112021			
4112031	EasyRoll Sliding Sheet	120/47.2	200/78.7	220/86.6	90/35.4	White/blue striped	Sliding sheet		Art. No. 4112031			

Material: Side sections: Cotton/polyester. Sliding surfaces: Duchesse (polyester)

Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.




CE Class 1 medical device



EasyRoll Sliding sheet with Anti-slip edge

EasyRoll Sliding Sheet with Anti-slip edge has braking surfaces at the sides of the sliding surface reinforced with silicone to prevent the user from slipping out of the bed. The sliding sheet is slipped onto the mattress and the elastic secures the sliding sheet around the mattress. Suitable for those with reduced mobility and/or severe pain since it gives good support when turning if you are lying directly on the sheet. Alternatively, use EasyRoll in combination with a draw sheet or a sliding draw sheet as a technical aid for healthcare professionals when turning/repositioning care recipients in bed.

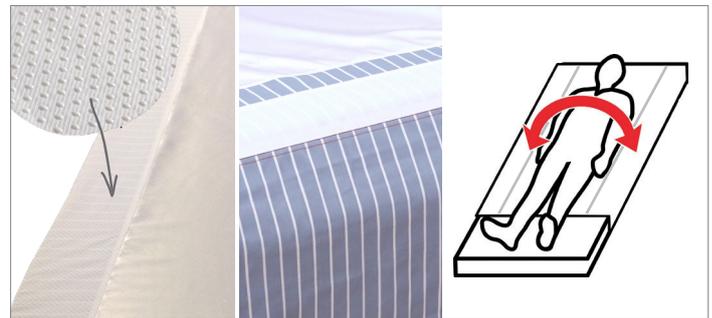


51902FAH

51901FAH, 51902FAH

Product overview EasyRoll Sliding Sheet with Anti-slip edge


Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
51901FAH	EasyRoll Sliding Sheet with Anti-slip edge	90/35.4	100/39.4	180/70.9	50/19.7	White/blue striped	Sliding sheet	Silicone braking surface Art. No. 51901FAH
51902FAH	EasyRoll Sliding Sheet with Anti-slip edge	90/35.4	140/55.1	180/70.9	50/19.7	White/blue striped	Sliding sheet	Silicone braking surface Art. No. 51902FAH
519031RAH	EasyRoll Sliding Sheet with Anti-slip edge	90/35.4	200/78.7	90/35.4	70/27.6	Blue/white	Sliding sheet	Silicone braking surface Art. No. 519031RAH

Material: Side sections: Cotton/polyester. Sliding surfaces: Duchesse (polyester)
Care instructions: Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.

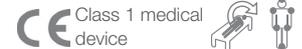

EasyRoll Sliding Sheet with Barrier

EasyRoll Sliding Sheet with Barrier is bedded like a regular sheet directly on the bed, its bed sheets are embedded under the mattress for fixation. The slip sheet with liquid barrier is used when there is a risk of leakage. The barrier layer is fluid tight and prevents fluid from spreading to underlying bedding / mattress. Red marker stripes make it easier to check the sliding sheet is placed correctly.

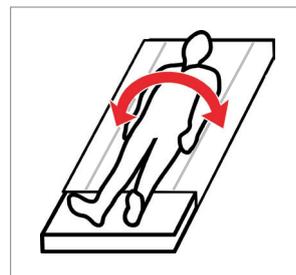

Product overview EasyRoll Sliding Sheet with Barrier


Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
42901PS	EasyRoll Sliding Sheet w Barrier	90/35.4	100/39.4	200/78.7	60/23.6	White/blue	Sliding Sheet w barrier	Slide direction → Sheet Sliding zone Art. No. 42901PS
42902PS	EasyRoll Sliding Sheet w Barrier	90/35.4	140/55.1	200/78.7	60/23.6	White blue	Sliding Sheet w barrier	Art. No. 42902PS

Material: Polyester. Barrier: PU. Absorbent: Polyester.
Care instructions: Wash before first use. Do not dry clean. Do not use bleach. Do not use fabric softener. Avoid tumble drying to ensure optimum service life of the material.



EasyRoll 2way Drawsheet

EasyRoll 2way Drawsheet is used in combination with EasyRoll Sliding Sheet. The draw sheet is placed directly on top of the sliding sheet, and the user lies on the draw sheet. The combination of the sliding sheet and the draw sheet is a very good aid for turning a care recipient who is in bed. By gripping the draw sheet and pulling it, the user can be moved into the desired position very easily.


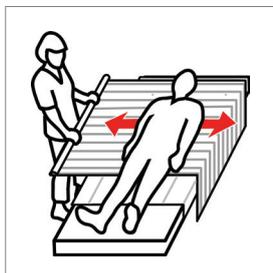


Product overview EasyRoll 2way Drawsheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
								Art. No. 2101S
Material: Duchesse, Cotton/Polyester		Care instructions: Wash before first use. Dry at max. 70°C.						

EasyRoll Inco 2way Sliding sheet

EasyRoll Inco 2way Sliding sheet is an absorbent incontinence pad with a barrier layer that prevents leakage. In addition, the underside of the product has a low-friction coating (sliding fabric) that allows easy sliding in bed. This means it facilitates the turning of the patient for caregivers. The EasyRoll Inco 2way Sliding sheet consist of three handles on each side for best grip in lateral movement of the patient.

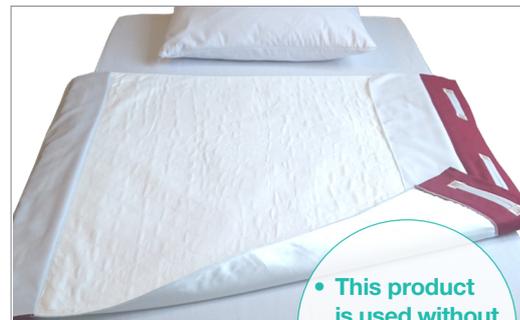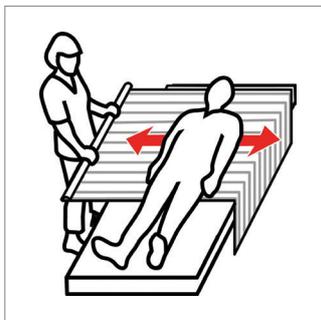


• This product is used without a sliding sheet underneath

Product overview EasyRoll Inco 2way Sliding sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
10991	EasyRoll Inco 2way	-	90/35.4	185/72.8	-	White/red	Sliding sheet	2x3 handles Art. No. 10991
Material: Duchesse. Barrier: PU. Absorbent: Polyesterrayon		Care instructions: Wash before first use. Dry at max. 70°C.						

EasyRoll 4way Sliding Drawsheet

This sliding draw sheet is placed directly on top of a sliding sheet, with the sliding surfaces against each other. EasyRoll Sliding Draw Sheet is a technical aid for healthcare professionals for easy turning/repositioning of a user who is in bed. When turning the user, the sliding draw sheet is released from the mattress. The healthcare professional grips the sliding draw sheet by the user's hip and shoulder and makes sure that they fill their hands with fabric. Then, keeping their wrists straight, the healthcare professional transfers their weight to lean backwards. If there are two care assistants, the second can help by pressing down on the shoulder and hip on the opposite side and pushing. The user will then turn over. After turning, the sliding draw sheet is tucked in on both sides of the mattress.

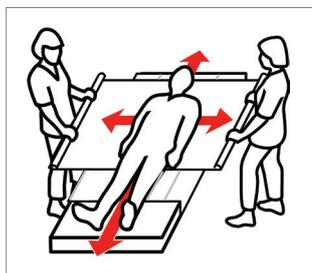

Product overview EasyRoll 4way Sliding Drawsheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
4011S	EasyRoll 4way Sliding Drawsheet	-	140/55.1	210/82.7	-	White/blue	Sliding draw sheet		Art. No. 4011S
4011L	EasyRoll 4way Sliding Drawsheet	-	200/78.7	200/78.7	-	White/blue	Sliding draw sheet		Art. No. 4011L
4011S1	EasyRoll 4way Sliding Drawsheet	-	150/59.1	200/78.7	-	White/blue striped	Sliding draw sheet		Art. No. 4011S1
4111T	EasyRoll 4way Sliding Draw Sheet	-	140/55.1	115/45.3	-	White/blue	Sliding draw sheet (w. underlying bed tabs)	2 x 9 handles	Art. No. 4111T
4111LT	EasyRoll 4way Sliding Draw Sheet	-	200/78.7	115/45.3	-	White/blue	Sliding draw sheet (w. underlying bed tabs)	2 x 13 handles	Art. No. 4111LT

Material: Duchesse, Cotton/Polyester

Care instructions: Do not use fabric softener. Wash before first use. Dry at max. 70°C.



Product overview EasyRoll 4way Sliding Drawsheet, Inco

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
10992-5	EasyRoll 4way Sliding Draw Sheet, Inco	-	140/55.1	200/78.7	-	White/red	Sliding draw sheet	2 x 5 handles	Art. No. 10992-5

Material: Duchesse. Barrier: PU. Absorbent: Polyesterrayon

Care instructions: Do not use fabric softener. Wash before first use. Dry at max. 70°C.





EasyRoll Pillow Case

EasyRoll pillow case (4303) with sliding surface has low friction sideways, which increases mobility in the neck and shoulders. It also gives lower friction on the back of the head and ears. EasyRoll pillow case w Barrier (4304) has a barrier layer on the inside to protect it from dirt and moisture, and it also functions as a mite barrier. The cover is reinforced by the pillow case being sealed with a zip. This pillow case has no sliding function.



Product overview EasyRoll Pillow Case

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Slide direction
4303	EasyRoll Sliding Pillow case	-	50/19.7	60/23.6	-	White	Pillow case	Sliding surface with low friction	Art. No. 4303
4304	EasyRoll Pillow case w Barrier	-	50/19.7	60/23.6	-	White	Pillow case	Barrier layer	Art. No. 4304

Material: 4303: Sliding surface: Duchesse (100% Polyester). Rear: Cotton/polyester. 4304: Barrier layer: Polyurethane. Area: Cotton jersey.

Care instructions: Do not use fabric softener. Wash before first use. Dry at max. 70°C.



CE Class 1 medical device



EasyRoll Underpad with bed tabs

The EasyRoll Underpad sheet cover is used as an absorber in the bed when the user is incontinent. The protection absorbs approx. 1–2 litres/m² and has a laminated barrier layer that prevents liquid seeping through to other bed linen. The sheet cover is strong and very suitable for use when turning/repositioning patients in bed. When moving/positioning users, grip the absorption part since the side sections are not suitable for pulling and repositioning. When used together with a sliding sheet, an excellent product combination of aids for incontinence and repositioning is achieved.



Product overview EasyRoll Underpad with bed tabs

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Barrier layer
80011B	EasyRoll Underpad	80–90/31.5–35.4	80/31.5	180/70.9	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80011B
80011P	EasyRoll Underpad	80–90/31.5–35.4	80/31.5	180/70.9	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80011P
80011-1	EasyRoll Underpad	80–90/31.5–35.4	85/33.5	180/70.9	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80011-1
80011PL	EasyRoll Underpad	80–90/31.5–35.4	90/35.4	210/82.7	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80011PL
80011PS	EasyRoll Underpad	90/35.4	70/27.6	190/74.8	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80011PS

Material:

80011B: Bed surface: CottonPolyester; Barrier: PU; Absorbent: Polyester.

80011P: Bed surface : Polyester; Barrier: PU; Absorbent: Polyester

80011-1: Bed surface: CottonPolyester; Barrier: PU; Absorbent: Polyester.

80011PL: Bed surface: Polyester; Barrier: PU; Absorbent: Polyester

80011PS: Bed surface: Duchesse, CottonPolyester; Barrier: PU; Absorbent: PolyesterRayon

Care instructions: Can be tumble dried at 70°C. Do not dry clean. Do not use bleach. Do not use fabric softener.




Class 1 medical device



EasyRoll Underpad with bed tabs /extra protection

The EasyRoll Underpad sheet cover is used as an absorber in the bed when the user is incontinent. This model have extra absorbing functionality. The sheet cover is strong and very suitable for use when turning/repositioning patients in bed. When used together with a sliding sheet, an excellent product combination of aids for incontinence and repositioning is achieved.

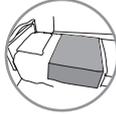

Product overview EasyRoll Underpad with bed tabs/extra protection


Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Barrier layer
80019	EasyRoll Underpad	90/35.4	120/47.2	180/70.9	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80019
80018	EasyRoll Underpad	90/35.4	150/59.1	180/70.9	-	White/Pale green	Sheet cover with bed tabs		Art. No. 80018

Material: Cotton/polyester bed surface. Polyester/rayon absorber. Polyurethane liquid barrier.
Care instructions: Can be tumble dried at 70°C. Do not dry clean. Do not use bleach. Do not use fabric softener.



EasyRoll Underpad without bed tabs

Sheet covers are used as absorbent protection in the bed for occupants who are incontinent. The protection absorbs approx. 1–2 litres/m² and has a laminated barrier layer that prevents liquid seeping through to other bed linen. The sheet cover is strong and very suitable for use when turning/repositioning patients in bed. When moving/positioning users, grip the absorption part since the side sections are not suitable for pulling and repositioning. When used together with a sliding sheet, an excellent product combination of aids for incontinence and repositioning is achieved. By virtue of the sheet cover's high absorption capability, the user experiences a better sense of dryness since the product rarely works at maximum absorption.

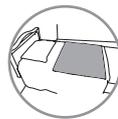

Product overview EasyRoll Underpad without bed tabs


Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other	Barrier layer
80017-1	EasyRoll Underpad	80–90/31.5–35.4	85/33.5	90/35.4	-	White/Pale green	Sheet cover without bed tabs		Art. No. 80017-1

Material: Polyester. Barrier: PU. Absorbent: Polyester.
Care instructions: Can be tumble dried at 70°C. Do not dry clean. Do not use bleach. Do not use fabric softener.



EasyRoll Underpad with Sliding sheet

The EasyRoll Underpad with Sliding sheet is used in the same way as the EasyRoll Underpad, but has ergonomic side sections and is used for protection in the bed for those with incontinence and as an aid when repositioning. The protection absorbs approx. 1–2 litres/m² and has a laminated barrier layer that prevents liquid seeping through to other bed linen. The Underpad is strong and very suitable for use when turning/repositioning patients in bed. The sliding surface on the bottom gives an excellent product combination of aids for incontinence and repositioning.

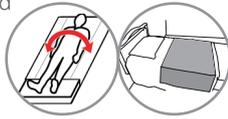

Product overview EasyRoll Underpad with Sliding sheet

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
80010P	EasyRoll Underpad w Sliding sheet	80–90/31.5–35.4	80/31.5	180/70.9	-	White	Sheet cover w. bed tabs, slide on the underside	Art. No. 80010P

Material: Polyester bed surface. Polyester/rayon absorber. Polyurethane liquid barrier. Duchesse sliding surface.
Care instructions: Can be tumble dried at 70°C. Do not dry clean. Do not use bleach. Do not use fabric softener.



EasyRoll Mattress protector

Mattress protectors protect your mattress from moisture in the event of e.g. incontinence, or from mites, or just act as general protection for the mattress/the mattress overlay. The mattress protector has a barrier layer on the bottom to prevent moisture from seeping through, and this also functions as a mite barrier.



Product overview EasyRoll Mattress protector

Art. No.	Product name	Fits mattress (cm/inch)	Length (cm/inch)	Width (cm/inch)	Sliding surface (cm/inch)	Colour	Functionality	Other
90.200J01	EasyRoll Mattress protector	90/35.4	200/78.7	90/35.4	-	White	Mattress cover	Art. No. 90.200J01
105.200J01	EasyRoll Mattress protector	105/41.3	200/78.7	105/41.3	-	White	Mattress cover	Art. No. 105.200J01
120.200J01	EasyRoll Mattress protector	120/47.2	200/78.7	120/47.2	-	White	Mattress cover	Art. No. 120.200J01

Material: Cotton jersey bed surface. Polyurethane liquid barrier.
Care instructions: Can be tumble dried at 70°C. Do not dry clean. Do not use bleach. Do not use fabric softener.





SystemRoMedic™

More than 30 years in the service of simplicity

System RoMedic's unique easy transfer concept is the market's widest and most complete range of clever, easy-to-use and safe assistive devices for all types of patient transfers and manual handling requirements.

The philosophy behind the concept is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, we offer improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Designed and produced by Direct Healthcare Group

For more than 30 years, Direct Healthcare Group has offered simple solutions for great results, and an easier everyday life for both users and personnel in the care sector.

Direct Healthcare Group offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

Direct Healthcare Group

Advancing Movement & Health®

Withey Court, Western Industrial
Estate Caerphilly, United Kingdom /
CF83 1BF
Tel: +44 (0) 800 043 0881
info@directhealthcaregroup.com
www.directhealthcaregroup.com