SlingBar RFL X4

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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REF

70200017

Maximum patient weight: 300 kg/ 660 lbs

Table of contents

English	3
Svenska	4
Norsk	5
Dansk	6
Suomi	7
Deutsch	8
Français	9
Español	10
Dimensions	11
Symbols	11

SlingBar RFL X4

SystemRoMedic™

Instructions for use - English

Slingbar RFL X4 is a four-point sling crossbar that is intended for use with DHG-approved mobile and ceiling lifts. It is designed for use with correctly fitted slings for lifting patients. Slingbar RFL X4 is designed to provide more space in the sling, for example, for obese and/or pain-sensitive patients. Slingbar RFL X4 is made of steel.

Functional inspection

Visual inspection

Inspect the sling bar regularly. Check to ensure that material is free from damage.

Before use:

Ensure that the product is correctly assembled.

Check sling bar connection and safety latch function.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

The lift may only be used by persons who have received instruction in the operation of the lift.

Assembly

Installation of Sling bar RFL X4 on mobile and overhead lift units:

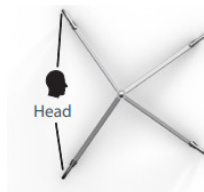
The four-point sling crossbar comes with a locking nut for installation on lift units. Install using existing screw fasteners.

NOTE! Locking nuts must not be reused.

Important information

- lifting accessories must be properly fitted and tested in relation to the patient's needs and functional ability.
- Do not leave the patient unattended during a transfer situation.
- **NOTE!** The lift sling's back loops are connected to the wider section of the slingbar (see image).

Maximum patient weight

Different products on the same lift system (lift unit, slingbar, sling, scales and other lifting accessories) may have different allowable safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Cleaning

Clean the slingbar with warm water or rubbing alcohol. Do not use cleaning agents containing phenol or chlorine, as this may damage the material.

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Dimensions

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.