StandingVest

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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REF

44503003

44503004

44503006


44503007

44503008

SWL: 300 kg/ 660 lbs

Table of contents

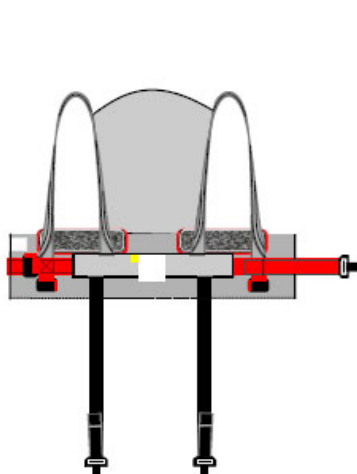English	3
Svenska	7
Norsk	11
Dansk	15
Deutsch	19
Nederlands	23
Français	27
Italiano	31
Symbols	34

StandingVest is specially designed for lifting with a standing shell. For the caregiver, it is easy to apply and it gives the patient a feeling of security. It is designed to provide upper-body support and is held in place by leg harness at the groin. StandingVest is designed to activate the patient safely and comfortably.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully.

First, apply the sling's back loops to the slingbar, and then apply the leg support loops.

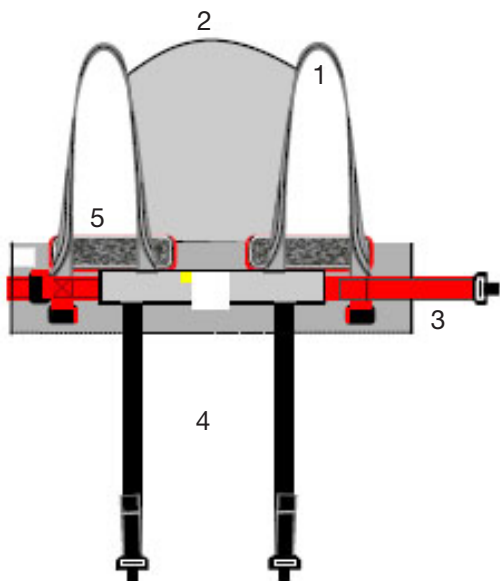
Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

Description of StandingVest


StandingVest consists of:

1. Lift loops (grey)
2. Back/head support
3. Waist belt (red)
4. Leg harness (black)
5. Padding for vest

The lift loops are connected to the slingbar. The waist belt and leg harness are equipped with a quick-release buckle for easy application. The leg harness is fitted around the thighs, from back to front. The lift vest is padded at arm height to prevent pressure and chafing.

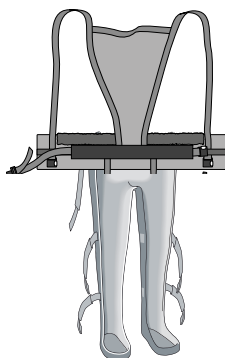Placement of the StandingVest when the patient is lying down in a standing shell


Place the RoMedic StandingVest under the standing shell, and then place the patient in the standing shell according to the standing shell instructions. Then put the patient's arms through the lift loops on the StandingVest. The mid-section of the StandingVest should align with the patient's spine and provide support for the head.

The waist belt is fastened and adjusted.

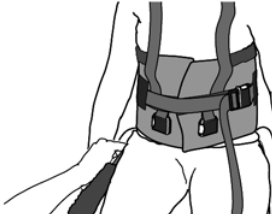Placement of the leg harness when the patient is lying down in a standing shell


When the vest is placed around the patient's upper body, the leg harness is applied from the side and under the thighs up to the groin.

The leg harness is fastened at the front.

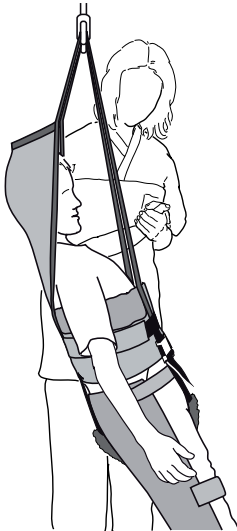Rising to a standing position

Prepare the patient by explaining the procedure.

Connect the arm loops to the slingbar and start to lift.

Never leave the patient unattended during a lifting situation. Always be attentive to the patient's signals. Stop raising if the patient is uncomfortable.

The standing shell should be secured according to the standing shell instructions while the lift stabilizes the patient.

When the patient is standing securely, the lift can be lowered and the StandingVest removed.

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions. The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on www.directhealthcaregroup.com

Material:

Polyester: Durable material with low friction. Withstands high laundry temperatures.

Care of the product:

Read the product label.

Do not use rinsing agent.
To prolong product life,
avoid tumble-drying.

Size information

Size		XS/Orange	S/Red	M/Yellow	L/Green	XL/Purple
Lift vests range	cm	82	96	110	122	130
	inch	32	38	43	48	51
Rec. user weight	lbs	33-88	44-110	88-132	110-176	154-220
	kg	15-40	20-50	40-60	50-80	70-100

Contact your local distributor if you have any questions about the product and its use.

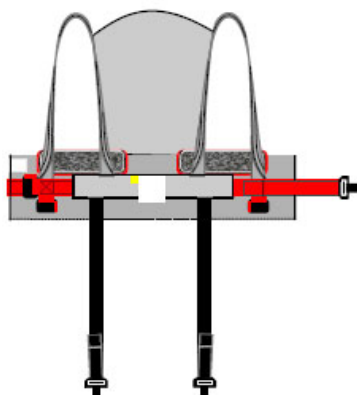
See www.directhealthcaregroup.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.