# StretcherSling

System**RoMedic**

---

Instructions for use – English

Bruksanvisning – Svenska

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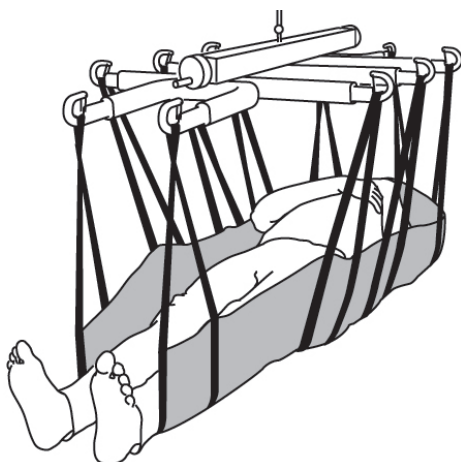
**REF**

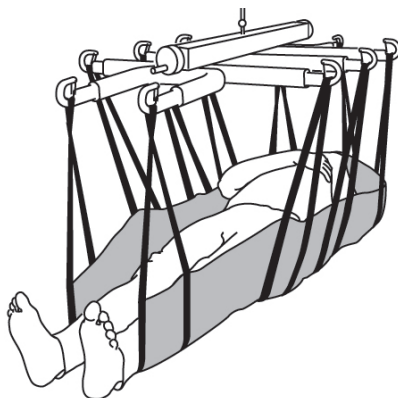
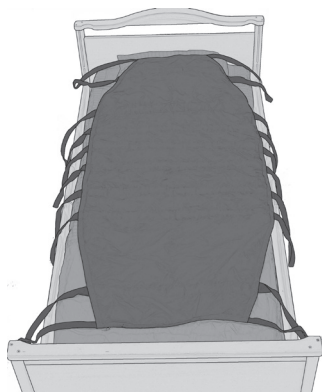
46502007

---

**SWL: 300 kg/ 660 lbs**

## Table of contents

English .....	3
Svenska .....	7
Norsk .....	11
Dansk .....	15
Deutsch .....	19
Nederlands .....	23
Français .....	27
Italiano .....	31
Polski .....	35
Symbols .....	34

StretcherSling is used in combination with a specially designed slingbar, StretcherBar, for horizontal lifting. It features ten suspension points, five on each side of the patient, which provide uniform support under the entire body. StretcherSling is designed to meet the need for transferring patients in a supine position.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

## Functional inspection

### Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

## Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

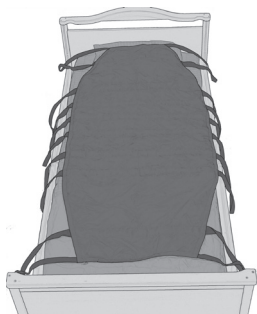
To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system.


Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

## Product description StretcherSling and StretcherBar

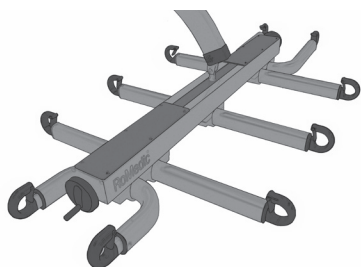
StretcherSling features a 180-cm-long fabric sheet with five lift straps that extend from one side to the other. This means that the weight of the patient is distributed over ten straps.


StretcherSling has been designed for use exclusively with the StretcherBar. No other combinations are recommended.

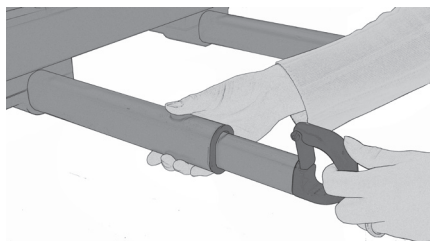StretcherBar is made of aluminium. The mid-section has a built-in balance regulator and five aluminium profiles with lift hooks on each end. See manual for StretcherBar.

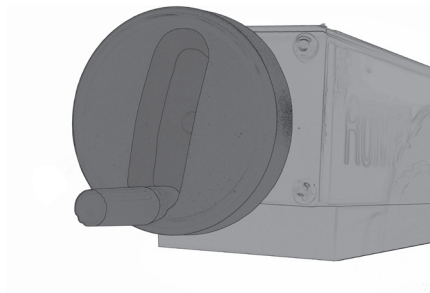
The lift hooks are equipped with safety latches and all lift arms can be adjusted according to the patient's width by lightly pulling them straight out before starting the lift. See manual for StretcherBar.


The mid-section of the StretcherBar contains a built-in balance regulator. This is regulated by rotating a dial on both sides to achieve the desired balance. See manual for StretcherBar.

## StretcherSling can only be applied and removed when the patient is lying down

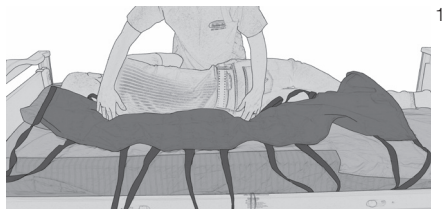
1

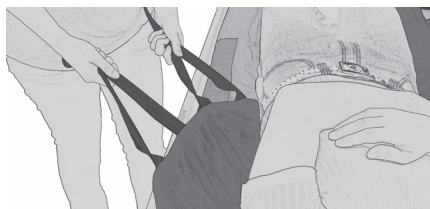1. Turn the patient towards you. Place StretcherSling so that the product label is facing the underlying surface and the sling's mid-section follows the patient's spine. The upper edge of StretcherSling should support the head.


2

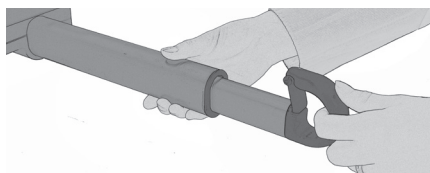2. Turn the patient back onto the sling and pull the sling out from the other side with the help of the straps.

3

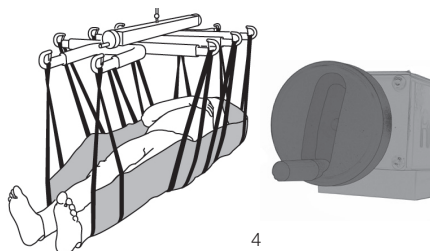3. If necessary: Adjust the width of the StretcherBar to fit the patient's width by pulling out the lift hooks. Then, hook the lift loops onto the bar.

4

5

4. Raise the lift until all loops are fully extended. Pause to ensure that everything is as it should be before raising the patient clear of the underlying surface.


5. If the patient is not balanced after being lifted from the underlying surface, this can be corrected turning the balance regulator.

## Removal of the StretcherSling

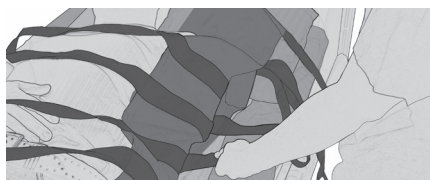1

1. Release all suspension hooks from the lift and move it out of the way. Turn the patient toward you and tuck StretcherSling in under itself against the underlying surface.

2

2. The fabric slides against itself when you pull out StretcherSling, after turning back the patient.

## Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions. The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## Alternative application of StretcherSling

If the situation does not allow the patient to be turned, begin by placing two ReadySheets under the patient (from the bottom up) and then draw StretcherSling between the smooth materials.

## Material:

Polyester: Durable material with low friction. Withstands high laundry temperatures.

## Care of the product:

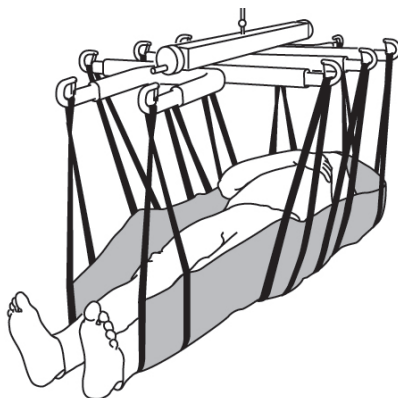
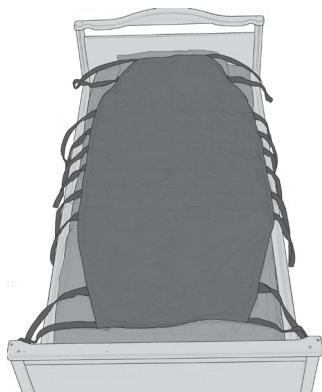
Read the product label.

Do not use rinsing agent.  
To prolong product life,  
avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)