

QuickMove™

SystemRoMedic™



Safeguards
and strengthens
the user's abilities

Small and
convenient

Safe and secure sit-to-stand and transfer

QuickMove is a safe and easy-to-use aid for sit-to-stand and transfer of users with impaired balance and reduced lower limb strength. QuickMove supports the user when rising, using his/her own strength and then transfers the user, either standing up or sitting down, from bed to chair or wheelchair or to and from the toilet. QuickMove is an excellent transfer aid which spares the backs of the carers and reduces the risk of work related injuries.

Direct Healthcare
Group

Advancing Movement & Health®

Safe and secure sit-to-stand and transfer

QuickMove supports and activates the user when rising up to a standing position and during transfers, either standing or sitting, from bed to chair, from chair to chair or to and from the toilet. QuickMove has been developed for users with impaired balance and strength in their legs but the user must have some strength in the arms and must be able to weightbear to actively participate in the transfer.



QuickMove has adjustable base width which makes it very easy to get close enough and in the right position for the user.



The adjustable buttock cushions are angled to make room for the user and carer.



The user places his/her feet on the stable anti-slip foot plate, and gets support from the adjustable lower leg support.



The user chooses a comfortable horizontal or vertical grip and rises using his/her own strength. When the user has risen using his/her own strength, the carer or the user folds down the cushions behind the buttocks, to provide a stable support.



The user then has the option to remain standing or to be seated during the transfer. The support of the cushions behind the buttocks and the knee-brace facilitates a very stable and safe transfer.



When the user is rising again, the carer can, with a push on the handle, angle the cushions to assist achieving a standing position.

Safeguards and strengthens the user's abilities

QuickMove provides a more secure sit-to-stand procedure for both user and carer, and, at the same time, it contributes to safeguarding and strengthening the user's abilities. QuickMove is suitable both for long-term and for occasional needs, for example in post-operative care when the balance is temporarily impaired.

Easy to use, everywhere

QuickMove is made of non-corrosive materials. It's also compact and convenient and very easy to manoeuvre which makes it very useful in most spaces and situations. QuickMove has a stable construction and can manage users weighing up to 170kg/26.5 stone



QuickMove

Product specification

Item No.	Product
400801334	QuickMove

Technical information

Material:	Powder coated steel
Length:	98cm/39.2"
Width:	60.5cm/24.2"
Height:	112cm/44.8"
Base width:	51–90cm/20.4"–36"
Castor size:	Ø 75mm/3"
Weight:	35kg/77lbs
Safe working load:	170kg/26.5 stone



Accessories for extra support and security

BackSupport is a new accessory which can be used as torso/side support. BackSupport provides extra stability and security for users during transfers with QuickMove.

Another alternative is SupportStraps which, together with EasyBelt, FlexiBelt or FlexiMove, can be used as torso/side support for QuickMove.

BackSupport



Item No.	Product
70200037	BackSupport, accessory for QuickMove



SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.

Transfer, between two locations. **Positioning**, within the same location. **Support**, for mobility. **Lifting**, both manual and mechanical.

Direct Healthcare Group

Advancing Movement & Health®

Withey Court, Western Industrial Estate
Caerphilly, United Kingdom CF83 1BF
T: +44 (0) 800 043 0881
F: +44 (0) 845 459 9832
info@directhealthcaregroup.com
www.directhealthcaregroup.com