TurnTableSoft

SystemRoMedic™

Instructions for use – English

Bruksanvisning – Svenska

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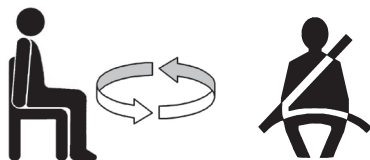
REF

5086
5087

Ø 450 mm/ 17.7"
Ø 400 mm/ 15.7"

SWL: 150 kg/ 330 lbs

Table of contents

Always read the manual	3
English	4
Svenska	5
Norsk	6
Dansk	7
Suomi	8
Deutsch	9
Nederlands	10
Français	11
Italiano	12
Español	13
Polski	14
Description	15
Symbols	15

EN Always read the user manual

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SystemRoMedic™

Instructions for use - English

TurnTableSoft is used mainly for transfers to and from car seats, but also chairs and beds, as well as to facilitate rotating/turning. TurnTableSoft has two round discs with an anti-friction material inside that allows the discs to glide freely against each other. TurnTableSoft is available in two sizes.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

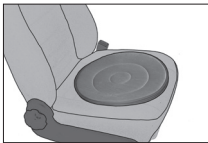
Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

NOTE!

TurnTableSoft cannot be used on leather or rubberized upholstery.


Placement and use TurnTableSoft

If used on a car seat, TurnTableSoft is placed on the middle of the seat.

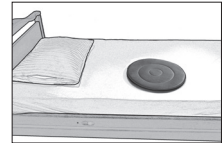When TurnTableSoft is used on a bed, it is placed near the edge of the bed at a level that allows the user's head to rest on the pillow when he/she is lying down.

The user sits at an angle on the car seat with legs outside of the car.

The user then rotates easily while lifting his/her legs into the car.

The user remains seated on the TurnTableSoft during travel. To get out of the car, the user simply rotates to the side while lifting the legs out of the car.

Always wear a seatbelt!

Material

Polyester, polyamide, cotton

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com