# TurningSling

System**RoMedic**

---

Instructions for use – English

Bruksanvisning – Svenska

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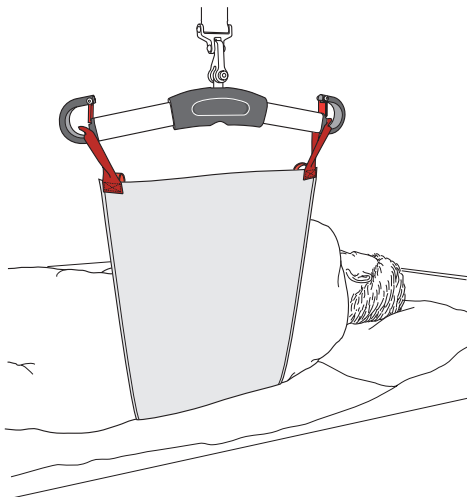
**REF**

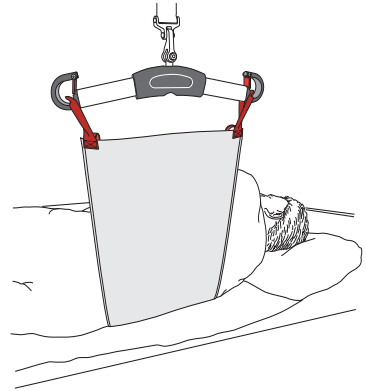
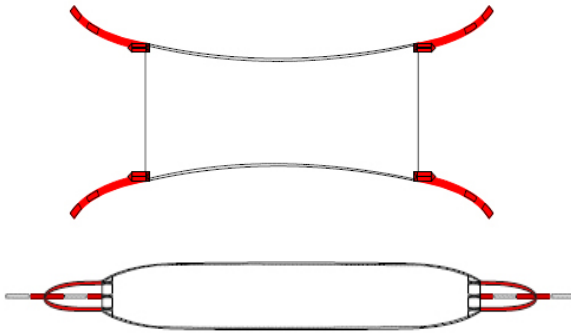
49592000

---

**SWL: 250 kg/ 550 lbs**

## Table of contents

English .....	3
Svenska .....	7
Norsk .....	11
Dansk .....	15
Deutsch .....	19
Nederlands .....	23
Français .....	27
Symbols .....	30

TurningSling is an assistive device for turning patients that is used in combination with a two-point slingbar mounted on an overhead or mobile lift. It is designed to provide support at the lower spine while allowing the seat to remain open, thereby facilitating hygiene and dressing. TurningSling has suspension points on each side. TurningSling is made of a durable disposable fabric, which means that it cannot be laundered and must be discarded when it becomes soiled or is no longer required by the patient.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

## Functional inspection

### Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

## Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system.

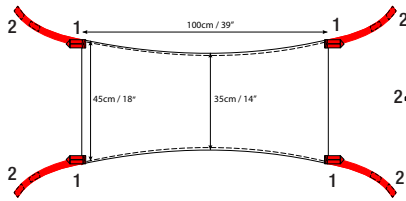
Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

## Description

### 1. Slingbar loops


### 2. Straps with Velcro strips

#### TurningSling

Art no: 49592000

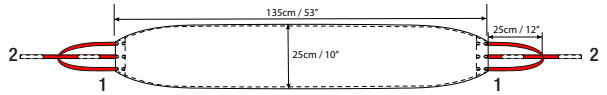TurningSling is made of a durable disposable fabric and has two slingbar loops and two straps with Velcro strips on each end.

#### TurningSling, one loop

Art no: 49592005

TurningSling is made of a durable disposable fabric and has one slingbar loop and one strap with Velcro strips on each end.

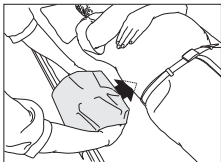
**Note!** Straps with Velcro strips must not be fastened to the slingbar!

**Note!** The product may only be used for turning patients.

**Note!** The bed side rail must be raised on the side towards which the patient is to be turned.

## Placement


Fold in one end of the TurningSling and place your hands in the pocket that is formed. Place the TurningSling under the patient's lower spine.

Press the fabric down towards the mattress and push it in under the patient as far as possible.

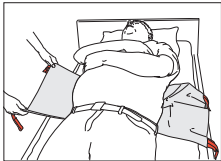
Go to the opposite side, place your hands under the patient, and then grasp the TurningSling and pull it out.

Pull until there is an equal amount of fabric on both sides of the patient.

## Placement with EasyGlide Oval

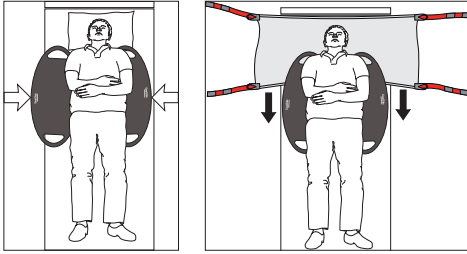
Place two EasyGlide Ovals under the patient.

Place the TurningSling between the sliding boards and the underlying surface. Then, pull the TurningSling down behind the patient's back.


Remove the sliding boards.

## Use

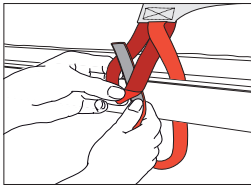
If the TurningSling is to be left under the patient after turning, one side of it must be secured to an appropriate part of the bed frame.

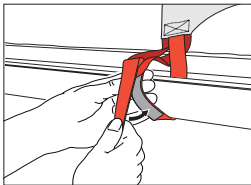
The loose strap with Velcro is secured to one side of the bed frame and the loop to the other side.

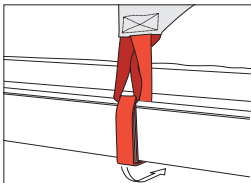Guide the strap with Velcro through the loop and fasten the Velcro.

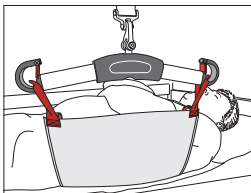Check to ensure that it is securely fastened by applying tension to the strap.


Prepare the patient for turning.

Raise the bed side rail on the side towards which the patient is to be turned.

Hook the loops to the slingbar. Begin turning, but pause and check to ensure that everything is as it should be. Continue turning until the patient is lying on one side.

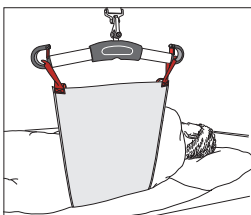**⚠ Note!** Straps with Velcro strips must not be fastened to the slingbar!

Allow the lift to hold the TurningSling in position to provide back support for the patient during the hygiene/dressing procedure.

**⚠ Note!** The product may only be used for turning patients.

## Removal

Pull one end of the TurningSling, so that it slides under the patient.

The TurningSling can also be left under the patient if it is to be used often. It can also be bedded under a draw sheet.

## Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions. The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## Material:

Non-Woven: Polypropylene, hydrophobic

## Care of the product, disposable:

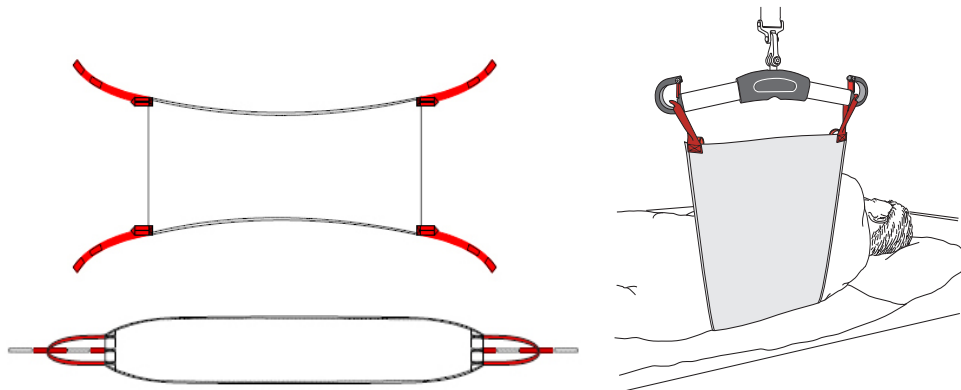
Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.