# WalkingVest

System**RoMedic**

---

Instructions for use – English

Bruksanvisning – Svenska

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Návod na použitie – Slovensky

## REF

44200002

44200003

44200004

44200006

44200007

44200008

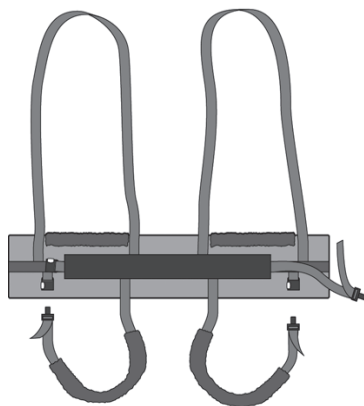
44200009

---

**SWL: 300 kg/ 660 lbs**

## Table of contents

English.....	3
Svenska.....	7
Norsk.....	11
Dansk.....	15
Suomi.....	19
Deutsch.....	23
Nederlands.....	27
Français.....	31
Italiano.....	35
Český.....	39
Español.....	43
Slovensky.....	47
Symbols.....	50

WalkingVest for gait training is easy for the caregiver to apply and it gives the patient a feeling of security. It is designed to provide support around the upper body and at the groin, via leg harness, during raising and gait training. WalkingVest is designed to activate the patient safely and comfortably.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

## Functional inspection

### Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

## Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort, trial fit the sling carefully.

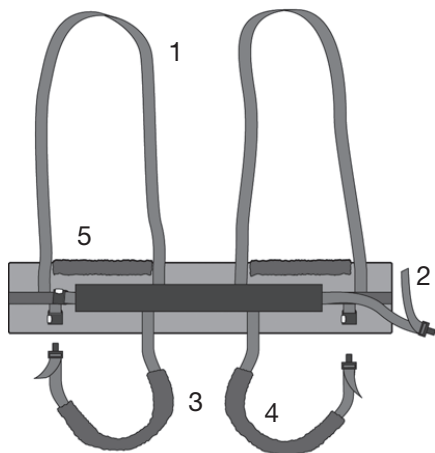
Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

## Description of the WalkingVest


WalkingVest consists of:

1. Lift loops (grey)
2. Waist belt (red)
3. Leg harness (black)
4. Padding for leg harness
5. Padding for vest

The lift loops are connected to the slingbar. The waist belt and leg harness are equipped with a quick-release buckle for easy application. The leg harness is fitted around the thighs, from back to front. The leg harness is padded to prevent pressure and chafing at the groin. The lift vest is also padded at arm height to prevent pressure and chafing.

## Application of the WalkingVest in a seated position

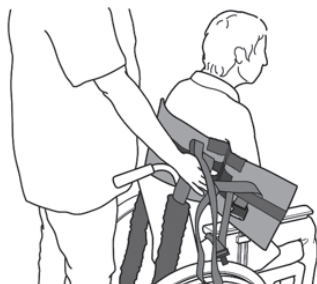
1. Place the WalkingVest around the upper body with the buckle on the front. The lower edge of the vest is positioned above the hip joints. The arms are held outside. The mid-section of the vest should follow the patient's spine.

2. The waist belt is connected and adjusted to the size of the patient.

## Application of the leg harness in a seated position

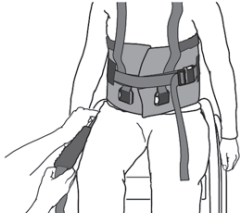1. When the vest is placed around the patient's upper body, the leg harness is applied from the side and under the thighs.

2. The leg harness is fastened at the front.

3. Adjust of the leg harness to fit the patient.

## Rising to a standing position

1. Prepare the patient by explaining the procedure. Never leave the patient unattended during a lifting situation. Always be attentive to the patient's signals. Stop raising if the patient is uncomfortable.

2. When the patient is standing, a walking support can be used.

## Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions. The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## Material:

Polyester: Durable material with low friction. Withstands high laundry temperatures.

## Care of the product:

Read the product label.

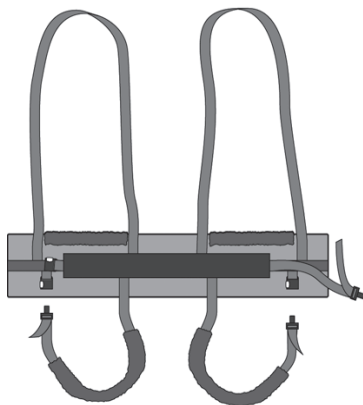
Do not use rinsing agent.  
To prolong product life,  
avoid tumble-drying.

## Size information

Size		XXS/Grey	XS/Orange	S/Red	M/Yellow	L/Green	XL/Purple	XXL/White
Sling measure	cm	60	70	82	96	110	122	142
	inch	24	27,5	32	38	43	48	56
Rec.	lbs	33-88	44-110	88-132	121-176	165-220	198-243	220-287
Patient weight	kg	15-40	25-50	40-60	55-80	75-100	90-110	100-130

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.