Moving Health Forward

EN [User Manual](#)

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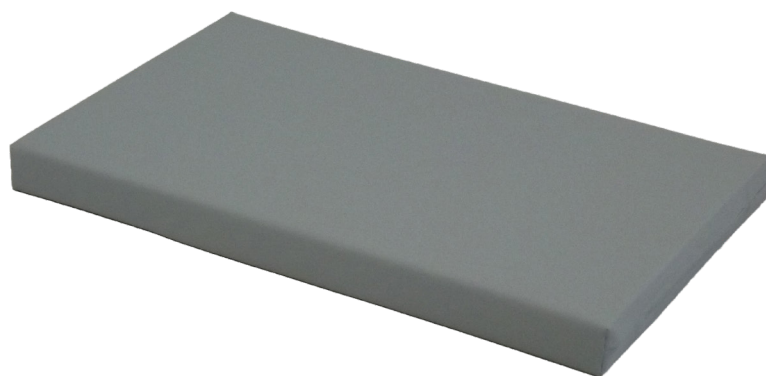
LEJRELET

Pad Low

EN

Application

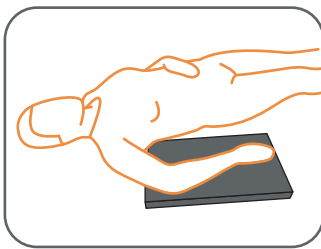
LEJRELET Pad Low is made from pressure relieving viscoelastic foam. This makes Pad Low ideal for pressure relief and support in several different positioning situations.

Pad Low can be used flat, bent, folded or coiled.


Pad Low can be used alone or in combination with other LEJRELET products.

Examples for use of LEJRELET Pad Low:


- Support of arm/hand
- Pressure relief of heels
- Pressure relief between knees/ankles
- Coiled, bent or folded i.e. as abduction wedge


Support of arm/hand

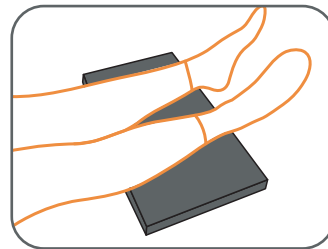Pressure relief of heel

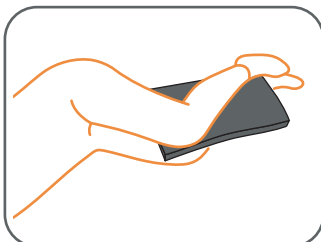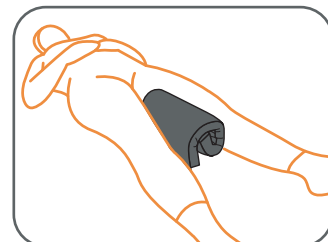Pressure relief between knees in lateral position

Coiled as abduction wedge

Technical Information

Filling:

LEJRELET Pad Low is made of viscoelastic foam preventing decubitus.

Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

Size: H 4 x W 30 x L 50

Item no.: 5000004

Weight: 350 g

Be aware that the patient is positioned such that airways are kept clear.

Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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This Page is Left Intentionally Blank

This Page is Left Intentionally Blank

Moving Health Forward

PRESSURE ULCER PREVENTION
SAFE MOVING & HANDLING
SPECIALIST THERAPIES
BATHROOM SAFETY
RENTAL & SERVICE SOLUTIONS

Direct Healthcare Group Ltd
Withey Court, Western Industrial Estate
Caerphilly, United Kingdom
CF83 1BF

T: +44 (0) 800 043 0881
F: +44 (0) 845 459 9832
E: info@directhealthcaregroup.com

EC **REP**

EU Authorised Representative, Importer, and Distributor:

Direct Healthcare Group Sverige AB,
Torshamnsgatan 35, SE-164 40, Kista
Sweden
T: +46 (0) 8 557 200

Australian Sponsor:

Direct Healthcare Group Pty Ltd.,
68 Howe Street, Osborne Park,
Western Australia 6017
T: +61 (0) 423 852 810

Issue 3
Date: June 2022

DHG-HEALTHCARE.COM