Moving Health Forward

EN [User Manual](#)

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
LEJRELET

Wedge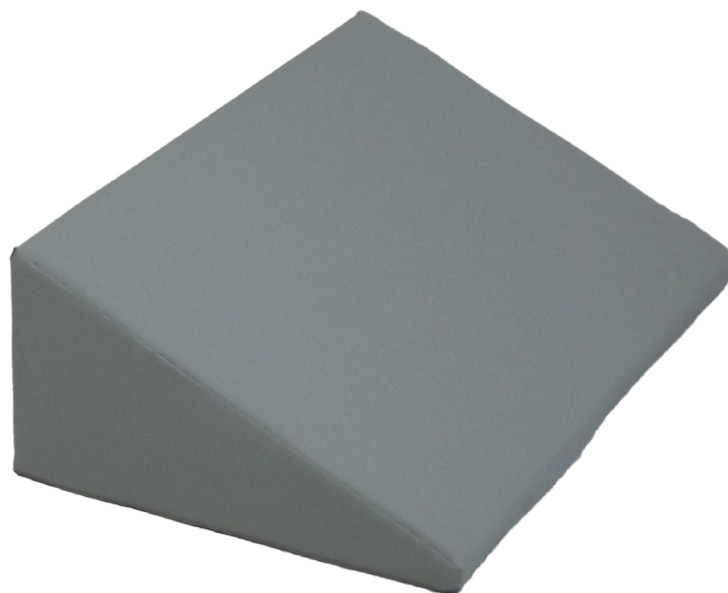

EN

Application


LEJRELET Wedge is a triangular cushion made of polyurethane foam and viscoelastic foam.

The pressure relieving foam offers a large support surface, good pressure relief and high comfort for the bedridden patient.


It is used in many different situations for both stabilization and pressure relief as well as in combination with other LEJRELET products.

Examples for use of LEJRELET Wedge:


- Stabilizing behind the back in lateral position
- Stabilizing in front of the body/trunk in lateral position
- Support of arm in lateral position
- Support of legs in lateral and supine position
- Positioning in a 30 degree angle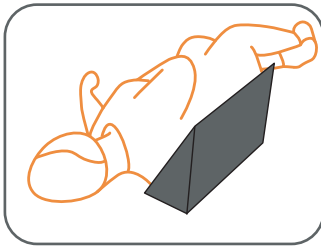


Stabilizing behind the back in lateral position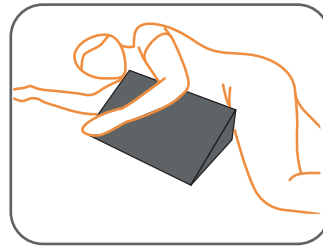

Stabilizing in front of the body in lateral position and support of arm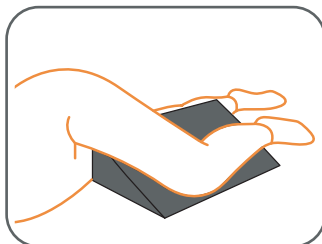

Support of leg in lateral position

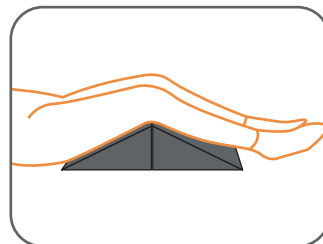Support of legs in supine position

Technical Information

Filling:

LEJRELET Wedge is pressure relieving and made with a core of poly-urethane foam covered with 2 cm of flame retardant viscoelastic foam on the top.

This combination means, that the cushions are firm and supportive, yet they are still flexible and provide effective pressure relief and comfort.

Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

Size: H 15 x W 30 x L 50

Item no.: 5000002

Weight: 550 g

Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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This Page is Left Intentionally Blank

This Page is Left Intentionally Blank

Moving Health Forward

PRESSURE ULCER PREVENTION
SAFE MOVING & HANDLING
SPECIALIST THERAPIES
BATHROOM SAFETY
RENTAL & SERVICE SOLUTIONS

Direct Healthcare Group Ltd
Withey Court, Western Industrial Estate
Caerphilly, United Kingdom
CF83 1BF

T: +44 (0) 800 043 0881
F: +44 (0) 845 459 9832
E: info@directhealthcaregroup.com

EC **REP**

EU Authorised Representative, Importer, and Distributor:

Direct Healthcare Group Sverige AB,
Torshamnsgatan 35, SE-164 40, Kista
Sweden
T: +46 (0) 8 557 200

Australian Sponsor:

Direct Healthcare Group Pty Ltd.,
68 Howe Street, Osborne Park,
Western Australia 6017
T: +61 (0) 423 852 810

Issue 3
Date: June 2022

DHG-HEALTHCARE.COM