EasyRoll PositioningBelt

SystemRoMedic™

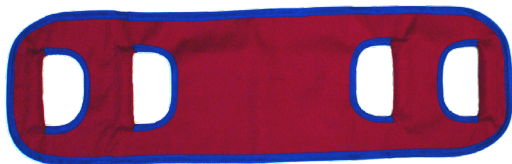
Instructions for use – English

Bruksanvisning – Svenska

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REF

9007

9007S

9007L

9007XL

Maximum patient weight: 200 kg/440 lbs

Table of contents

Product information	3
English	4
Svenska	6
Norsk	8
Suomi	10

Product information/ Produktinformation/ Produktinformasjon/ Tuotetiedot

Art. no.	Product/Produkt/ Tuote	Size/ Storlek/ Størrelse/ Koko
9007	Positioning Belt	17x110cm (WxL)
9007S	Positioning Belt	20x70cm (WxL)
9007L	Positioning Belt	17x120cm (WxL)
9007XL	Positioning Belt	17x240cm (WxL)

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code
	Maximum patient weight		

EasyRoll Positioning Belt are used in different transfer and repositioning situations such as turning in bed, repositioning higher up in bed. The choice depends on the user's need for support and required number of caregivers.

Maximum patient weight: 200 kg

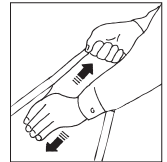
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded

Mechanical load

Apply pressure to handles and verify that they will not open or break under heavy loads.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

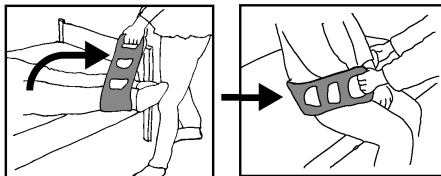
Placement of PositioningBelt

The PositioningBelt can be placed under legs, calves, feet, hips, shoulders or under the heaviest body part that will be transferred.

From laying to standing


To assist the patient in transfer from laying to standing position, PositioningBelt can be used to move the legs over the edge of the bed and later as a support behind the lower part of the back.

If the patient needs two caregivers, one caregiver can assist with moving the legs and the other with moving the upper body. At the sit to stand transfer, the caregivers need to stand on both sides of the patients and assist with holding on the handles that give the best working position.

Higher up in bed

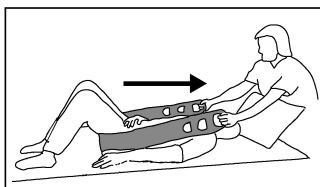One caregiver works on either side of the bed. Repositioning is easier if the user is able to bend his or her knees to push higher up in bed. Work in a walking stance with straight arms and straight back. Caregivers should coordinate their efforts to perform a smooth transfer.

Material

Nylon, Polyester

Care of the product

Read the product label.

Cleaning: Do not use rinse/ fabric soft-ener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com