




GOTO

User manual - for Size 1 & 2

For French / Italian / Spanish / Mandarin / Portuguese / Swedish / Japanese please refer to our website.
www.fireflyfriends.com

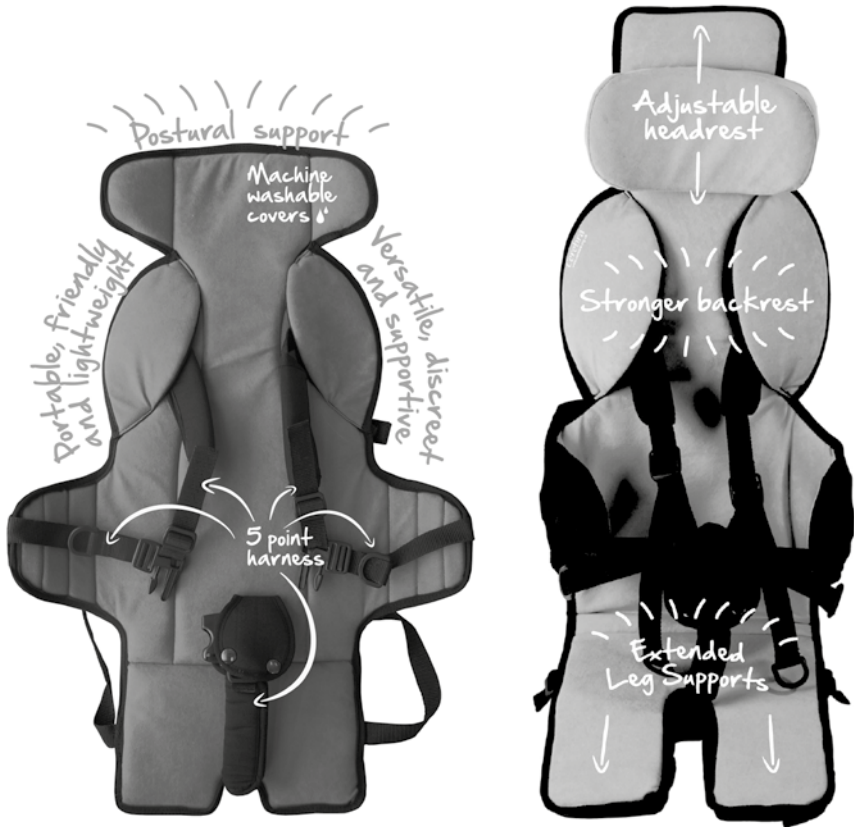

The Firefly GoTo is the goto portable seat for enhancing lives through participation.

It's a safe, supportive multipurpose seat, great for your home or on the go.

SECTION 1. Intended Use

The Goto seat has been designed to provide additional upper body support for children with disabilities. The seat must be attached to a secondary rigid support for stability and safety at all times. The Goto seat is for occasional use and therefore is not intended to be used as the primary seating system for a disabled child. The GoTo Seat must be securely attached to a rigid support by a minimum of one back strap and we recommend also using the seat base strap. If the secondary rigid support is not stable then the child should not be placed in to the seat. We cannot accept responsibility for any use of the product outside of its intended use. The GoTo Seat is available in 2 size options. Size 1 is intended for children aged approximately 1 - 2 years and a maximum weight of 13kg and size 2 is intended for children aged approximately 2 - 5 years and a maximum weight of 20kg.



Please note however that the weight limit for a child being placed into a standard shopping trolley seat is 13kg maximum.

Daily activities tested and approved by Leckey for the use of the GoToSeat are:

1. Attached to a hard back/ restaurant chair for dining with family and friends.
2. For use in a shopping trolley allowing the parent/ guardian to shop with greater ease.
3. For use in a Cradle swing type only, enabling the child to enjoy everyday activities.

The GoTo Seat must fit within the edges of the cradle swing seat base. If the Goto Seat protrudes outside of the cradle swing seat base then the GoTo Seat cannot in any circumstances be used in a cradle swing seat.

If in any doubt please contact Leckey Customer Services on UK 0800 318265 or ROI 1800 626020

SECTION 2. Declaration of Conformity

James Leckey Design Ltd; as manufacturer with sole responsibility for the Firefly brand, declares that the GoTo Seat conforms to the requirements of the 93/42/EEC Guidelines and EN12182 Technical aids for disabled persons, general requirements and test methods.

SECTION 3. Terms of Warranty

The warranty for the GoTo Seat applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturers recommendations (also see general terms of sales, delivery and payment at www.fireflyfriends.com). A one year warranty is provided with this Leckey manufactured product. To register your warranty please visit www.fireflyfriends.com.

SECTION 4. Safety Information

1. Always read user instructions fully before use.

2. Users should not be left unattended at any time in this product.

3. Always assess chair/trolley/cradle swing for suitability and stability before attaching GoTo Seat.

4. Always ensure the GoTo Seat is firmly secured before use and the straps are tight.

5. The GoTo Seat has not been crash tested therefore cannot be used in transport.

6. Never use the GoTo Seat in a shopping trolley on a sloping surface greater than 10 degrees.

7. If a GoTo Seat is used in a cradle swing seat; do not swing/push with excessive force.

8. The GoTo Seat contains parts which could present a choking hazard to small children. Check all components are tight and securely fastened before use.

9. Do not modify the GoTo Seat. Failure to follow the instructions may put the child at risk.

10. The GoTo Seat should be stored carefully and should not be used if any parts are faulty.

11. The GoTo Seat should be kept away from all direct sources of heat including naked flames, cigarettes, electric and gas heaters.

12. Supervision of the child is required when used in a cradle-type swing.

13. Size 2 has a maximum weight limit of 13kg when used in a shopping trolley.

14. The GoTo seat can be used in a disabled shopping trolley with a 700mm underseat extension strap. Check disabled trolley seat for user weight limit.

15. If in any doubt about the safe use of the GoTo Seat or if any parts should fail, please cease using the product and contact Leckey Customer Services on UK 0800 318265. Leckey recommend that a written record is maintained of all those who have been trained in the correct use of this product.

SECTION 5. Cleaning & Care Information

How to Maintain

When cleaning the plastic support we recommend that you use only warm water and a non-abrasive detergent. Never use organic solvents or dry cleaning fluids.

Upholstery and Fabric

1. Clean and care for your fabric on a regular basis. Simply vacuum to remove dust particles and wipe with a damp cloth for general cleaning.

2. Microfibre cloths are extremely effective without soap which is helpful as soap can clog your fabric.

3. The seat covers can be removed from the hard plastic back and machine washed at 40°C. Line drying is recommended however they can be tumble dried at a low temperature.

4. Upholstery can be placed into the washing machine intact, after removing straps, buckles and fastenings. Store these in a safe place and out of reach of children.

5. Fabric can also be cleaned by hand whilst in place. When cleaning, we recommend you use a non-abrasive detergent.

6. Staining should be removed as quickly as possible with absorbent cloth, towels or a sponge. Routine soap and warm water sponging is effective for ordinary soiling and minor spills. Be careful not to saturate the fabric as this will cause the stain to spread.

7. Antiseptic cleaning agents can be used on more stubborn stains. These may require a safe solvent such as Isopropyl Alcohol or Mineral Spirit. Ensure that cleaning products are fully removed.

SECTION 6. Product Servicing

For all service related issues please refer to the firefly website www.fireflyfriends.com or contact the Leckey Customer Service team on UK 0800 318265 or ROI 1800626020.

SECTION 7. Important guidelines

Placing of Child into GoTo Seat

1. The GoTo Seat must be securely fastened in place for its intended use before placing a child into the seat.

2. Ensure the harness straps are not obstructing placing a child into the GoTo Seat; shoulder straps and side lateral straps should be very slack.

3. If placed in a trolley: guide the child's legs slowly / gently through leg gaps in shopping trolley; do not force legs through.

4. Once legs are safely through, gently fit child's arms through shoulder straps – the shoulder straps should be very slack. Ensure shoulder strap pads are on child's shoulders.

5. Close centre buckle and adjust side lateral straps and shoulder straps where necessary to ensure child is secure in seat harness.

6. Close buckle strap.

Removal of Child from GoTo Seat

1. Slacken shoulder straps and side lateral straps.

2. Unlock central buckle flap and buckle.

3. Slide shoulder straps off child's shoulders and gently remove child's arms from harness.

4. If placed in a trolley: lift child up gently guiding their legs out from the leg gaps in the trolley. Do not let their feet catch the GoTo Seat as this may pull the GoTo Seat up and trap their feet.

SECTION 8. Daily Product Inspection & Servicing

We recommend that various checks of the equipment are carried out before use to ensure the product is safe. The recommended checks are detailed below.

1. Check harness is in place and secure.

2. Check all upholstery for signs of wear and tear.

3. Check all buckles to ensure straps secure firmly.

4. Check plastic support for cracks or damage.


If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our customer services department as soon as possible.

SECTION 9. Reissuing Leckey Products

Ensure the product has been cleaned thoroughly in accordance with section 5 of this manual.

Ensure a copy of the user manual is supplied with the product. If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our customer service department as soon as possible on UK 0800 318265 or ROI 1800 626020.

Section 10(a) Hardback Seat Attachment and removal

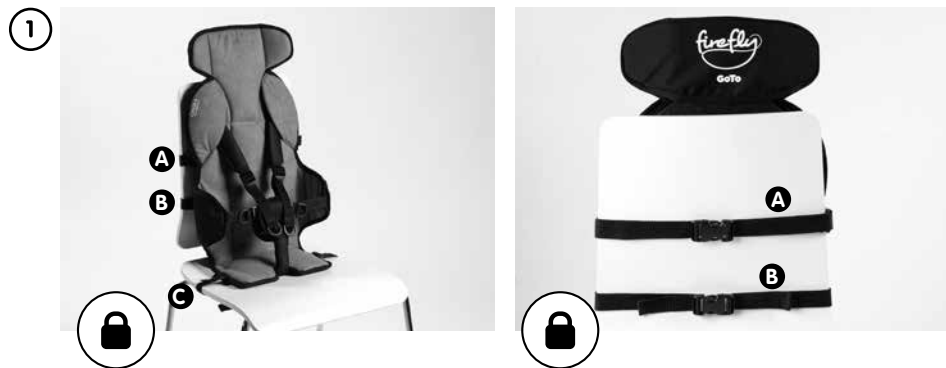

Clip in buckles to secure.
Unclip to remove.







Pull straps outward to tighten



Ensure either strap A or B and strap C are attached and fastened securely at all times before placing the child in the GoTo Seat.


If strap C cannot be attached please ensure the child is secure and has support at either side.

Section 10(b) Strap Adjustment & Securement

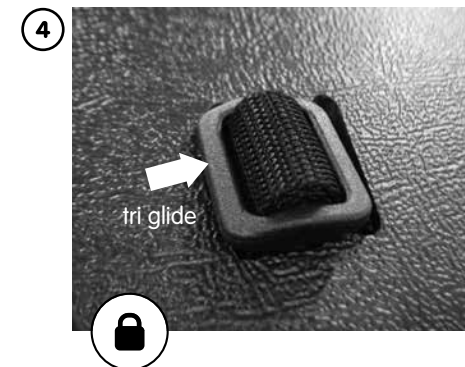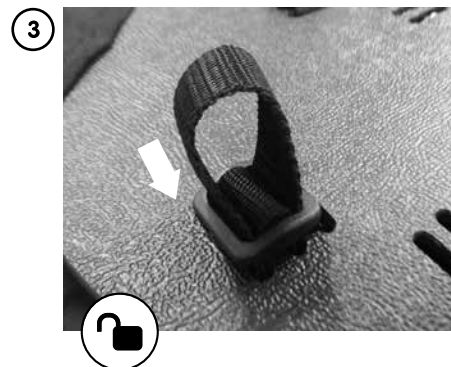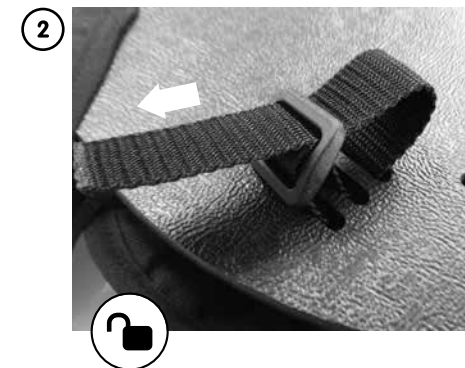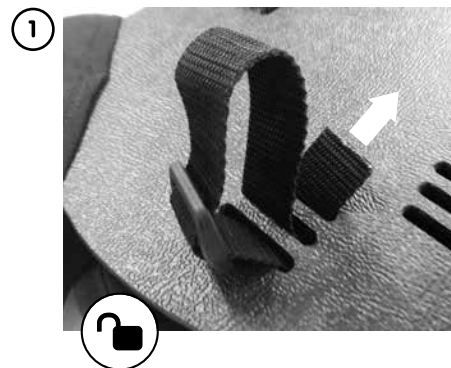

Ensure tri glides are secure

Section 10(c) 5 Point Buckle Opening and Closing

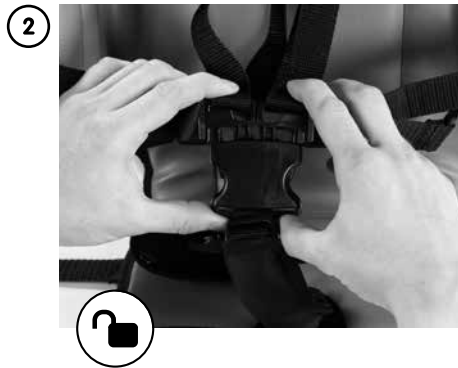

Section 10(d) How to attach and remove from shopping trolley



Clip in buckles to secure.
Unclip to remove.



Pull straps outward to tighten. For Size 1
go straight to point 5.



Feed leg straps through bars of
shopping trolley



Close straps securely as shown




Section 10(e)

How to remove and fit the upholstery (Size 1 Only)



GoTo Seat secure and ready to use. (Size 1 shown)

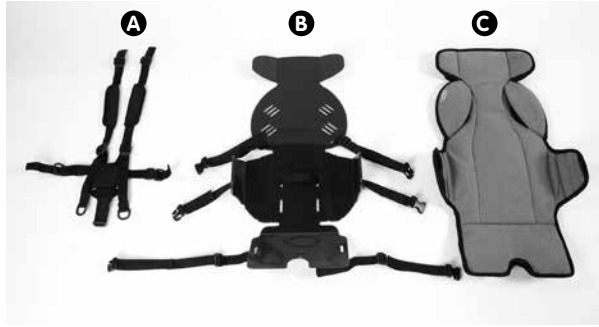

GoTo Seat dismantled for servicing or cleaning. (Size 1 shown)


Step 1



Removing the 5 point harness



Step 2

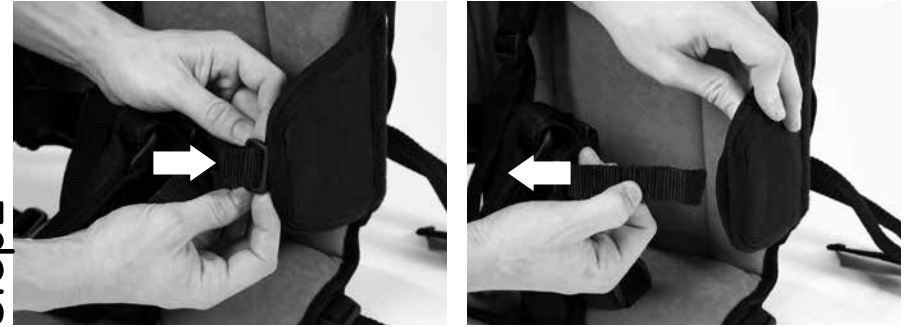

Removing the fabric cover from the plastic support.



To fit straps follow above steps in reverse. (Use section 10c to assist).



Ensure all straps and tri glides are secure and test before placing child in seat.

Section 10(f)

How to remove and fit the upholstery (Size 2 Only)

①



Unclip poppers and remove headrest

②



Undo front harness buckle and slacken straps

③



Undo tri-glide and remove lateral strap

④



Remove upholstery from plastic

⑤



Undo seat base straps and crotch strap

⑥



Slide upholstery off plastic

⑦



GoTo seat dismantling for servicing or cleaning. Leave upholstery to dry flat. Do not tumble dry. **(See SECTION 5)**

⑧



GoTo seat strapping, buckles and webbing covers

Section 10(g) Adjusting the Headrest (Size 2 Only)

①



Remove popper from back of head rest to access straps

②



Undo webbing straps and adjust headrest accordingly. Close webbing straps securely when adjustment is complete.

③



Secure poppers closed when finished adjusting headrest position

Section 10(h) Attaching to a Disabled Trolley (Size 2 Only)



Pass the webbing through wire frame and pass round back to trolley



Clip buckles to secure. Unclip to remove.



Secure poppers closed when finished adjusting headrest position



Clip buckles to secure. Unclip to remove.



Pull straps outward to tighten.
ENSURE GOTO SEAT IS SECURELY ATTACHED BEFORE PLACING CHILD INTO SEAT



GoTo size 2 should look like above once properly fitted to a disabled trolley

Section 10(i)

Removal / Attachment of Headrest Foam (Size 2 Only)

①



Unzip back of headrest and removed foam before washing. Leave to dry flat - **DO NOT TUMBLE DRY (See Section 5)**

Section 11

Technical information

Product Codes - Seats			
	Blue Fabric	Green Fabric	Pink Fabric
Size 1	157-1660-01	157-1660-02	157-1660-03
Size 2	157-2660-01	157-2660-02	157-2660-03

Technical Table

Size	One	Two
Max Weight	13kg	20kg
Age Range	1 – 2 yrs	2-5 yrs
Seat height	557mm	645mm
Seat depth	154mm	265mm
Seat width	240	280mm
Lateral chest distance	240mm – 350mm	200-340mm
Hip to shoulder distance	317mm – 380mm	390-460mm

Notes

Notes



www.fireflyfriends.com

Firefly
19C Ballinderry Road, Lisburn, BT28 2SA, Northern Ireland
UK 0800 318265 ROI 1800 626020



LS312-02
