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EN - Please read the instructions before using the product.
These instructions apply to the headrest standard Combi chairs.

Purpose

The Headrest can be used on all Combi chair models. The contoured padding lends support to the user's head and neck. The headrest is angle and height adjustable.

Assembly

Before the headrest can be assembled the black covers in the backrest need to be removed. Screw the cylindrical metal bracket onto the backrest pushbar using the 2 screws (3) and metal bushes (4) provided. Slot the tubing into the bracket and tighten the hand lever.

Adjustment

The Headrest can be height adjusted by loosening and tightening the lower hand lever (1). Angle adjustment can be achieved by loosening and tightening the upper hand lever (2).

Maintenance

Clean with regular cleaning agents without corrosive or abrasive properties. We advise against the use of chlorine and solvents.

Caution!

- Use only as described under "Purpose"
- Make sure to tighten the hand levers after adjustment.
- A headrest should always be used on a tilt chair.
- The end stop/clip at the headrest may not be removed. This end stop ensures the safety and the maximum height of the headrest.

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Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com