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EN - Instructions for use

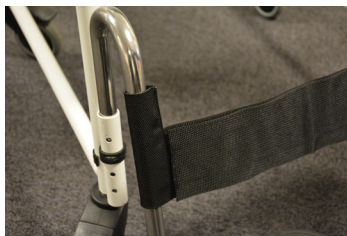
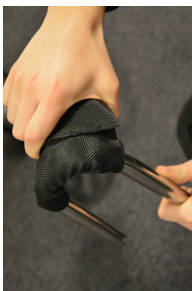
Please read the instructions before using the product.

Heel and calf strap is used together with the Footrests accessory. Heel and calf strap prevents the user's feet from getting caught behind the footrests and thereby provides even further increased safety.

Heel and calf strap consists of two pieces made from a nylon material. The pieces are easily attached to the left and right footrests and then fixed together with Velcro.

Heel and calf strap can be used with all models and variants of Combi chairs and it is recommended for use with both variants of Combi, tilt.

Attach the Heel and calf strap to the front legs of the chair/stool

Repeat the procedure on the opposite Footrest.

Maintenance

Read the product label.

To be wiped clean after use. Use domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied. The product can be detached for cleaning. Do not use fabric softener. It should not be dried in a tumble dryer.

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Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com